

Getinge Assured

Wash Monitor R Holder

The Getinge Assured Wash Monitor R Holder is an auxiliary device for routine monitoring of automated wash processes for hollow instruments in washer-disinfectors and ultrasonic washers.

Wash Monitor R Holder should only be used with Wash Monitor F/R indicators (in washer-disinfectors) or Wash Monitor U/L indicators (in ultrasonic washers). Wash Monitor F/R or Wash Monitor U/L is inserted into the Wash Monitor R Holder, simulating contaminated rigid lumen instruments.

Performance Characteristics

The Wash Monitor F/R or Wash Monitor U/L is inserted into the Wash Monitor Holder R. The Holder is then attached to a luer lock connector located on the washer rack. A standard load of hollow lumen instruments is processed as per manufacturer's recommendations. After wash cycle is complete, results of the cleaning efficacy are displayed on the Wash Monitor F/R and Wash Monitor U/L. The removal of all test soil indicates effective cleaning.



Physical Characteristics

Wash Monitor R Holder made of polished stainless steel.

- **Dimensions:** 197 mm x 11 mm
- **Weight:** 55 g
- **Substrate Material:** Electro polished stainless steel AISI 304

Shelf Life

Not applicable.

Ordering Information

Getinge Assured	Ordering number
Wash Monitor R Holder	503865000