

Getinge Assured

Test Soil

The Getinge Assured Test Soil has been carefully designed to simulate the soiling that occurs during theatre use.

The Getinge Assured Test Soil is supplied in individual pots.



Performance Characteristics

Water is added to the pot then shaken vigorously to form a paste which is then applied to a test load with the brush provided. After a drying period the test load should be cleaned using the normal procedures and inspected for residual soil. The bright red colour of the Getinge Assured Test Soil allows easy identification of areas that have not been properly cleaned.

Physical Characteristics

- **Dimensions:** H 63mm x Ø 43mm
- **Substrate Material:** Plastic vial with a lid with integrated brush

Environmental Limitations

Storage: 0° - 5°C (32° - 41°F). 30 – 60 % RH.

Shelf Life

12 months.

Standards Classification

Consistent with BS2745:1993 specification for Washer-disinfectors for medical purposes.
EN ISO 15883-5. HTM2030 Washer-disinfectors.

Ordering Information

Getinge Assured	Ordering number
Test Soil 10 pcs/case	504228300
Test Soil 1 pc	504228300/1