

Getinge Check

Bowie-Dick Mini Pack 134°C

The Getinge Check Bowie-Dick Mini Pack 134°C is for daily monitoring of pre-vacuum sub-atmospheric, trans-atmospheric and super-atmospheric steam sterilizers operating at 134°C - 137°C for up to 3.5 minutes.

The test pack is packaged in an easy to open box and designed to meet EN ISO 11140-4 requirements.



Performance Characteristics

If no appreciable air is present in the sterilizer chamber after vacuums are drawn, the steam will penetrate to the middle of the test card inside of the pack and turn the indicator card a uniform green color. If a small amount of air is present, sufficient enough to cause a two-degree temperature drop in the middle of an ISO 11140-4 Bowie-Dick towel pack, the air will migrate to the middle leaving the center of the card a purple color rather than uniform green.

Physical Characteristics

- **Lamination:** Top and bottom
- **Dimensions Card:** L 101mm x W 63mm
- **Packaging:** 30 test packs

Environmental Limitations

Storage: In provided sealed packaging at room temperature 10°C - 38°C and 10-60% Relative Humidity. Protect from moisture and humidity.

Shelf Life

24 months. The expiry date is printed on the product packaging.

Standards Classification

ISO 11140-4, ISO 11140-1 Type 2 indicators for use in specific tests.

Ordering Information

Getinge Check

Bowie-Dick Mini Pack 134°C

Order Number

6001155600