

# Getinge Clean

## Acid Detergent

Organic acid, instrument cleaner detergent. It provides superior removal of inorganic residues. Acid Detergent is especially effective against minerals and scale build-up.



### Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

### Key Features

- Has a strong dissolving action against minerals and scale
- Can be used in laboratories, hospitals and other institutions; on instruments, utensils, glassware and animal cages
- Removes detergent residuals from washer-disinfectors

### Physical and Chemical Properties

- Clear, colorless, thin liquid
- Complete water solubility
- Characteristic fragrance
- pH: <2.0
- Store below 120°F (49°C)

*NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling and storage and waste disposal methods.*

### Expiration/Use-By Date

35 Months from date of manufacture.

**Quality Statement:** Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

## Usage Information

Acid Detergent should be used at a dose of 0.2 to 1 oz./gal. (2 to 8 mL/L) depending on water quality. Always follow manufacturer's directions for the particular automatic washer being used. If you have additional questions, consult your Getinge representative.

### For an Automated Dosing System — First Time Use with New Product

1. It is important that the product delivery line is flushed before use.
  2. Prime the line to ensure that there is no/minimal air in the line.
  3. Ensure the dosing is set to the indicated amount of 0.2 to 1 oz./gal. (2 to 8 mL/L).
- **NOTE:** Follow the instrument manufacturer's IFU for Decontamination/Cleaning/Sterilization.
  - **NOTE:** Follow hospital policy for appropriate PPE to be worn during reprocessing.
  - **NOTE:** Decontamination should be performed in accordance with AAMI ST79, AORN, and CDC guidelines. Please consult your Getinge Representative for additional information.
  - **CAUTION:** Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.
  - **CAUTION:** Some plastics and rubbers might not be compatible with acid detergents or high water temperatures. Check with manufacturer for suitability.

## Ordering Information

Getinge Clean	Order Number
3 x 5L	6036000007
5 Gallons	61301600042
15 Gallons	61301600043
30 Gallons	61301602229

---

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries.  
Copyright 2021 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

**Manufacturer:** Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110  
**Sales Office, US:** 1 Geoffrey Way, Wayne, NJ 07470, USA  
**Sales Office, Canada:** 90 Matheson Blvd West, Suite 300, Mississauga, Ontario L5R 3R3, Canada  
[www.getinge.com](http://www.getinge.com)



MCV00044162 REVC  
EXP 11/23