

Getinge Clean

Alkaline Detergent

Alkaline Detergent is an economical powerful blend of alkaline builders, water conditioners and surfactants designed for use in washer-disinfectors, cart or rack washers.

Alkaline Detergent is ideal for removing protein, blood and fatty soils from instruments, labware, and glassware.



Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

Key Features

- Superior cleaning capability
- Cleans surgical instruments, labware, and glassware

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild fragrance
- pH: 12.5 to 14.0
- Store below 120°F (49°C)

NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling and storage and waste disposal methods.

Expiration/Use-By Date

35 Months from date of manufacture.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Usage Information

Alkaline Detergent should be used at a dose of 0.2 to 1 oz./gal. (2 to 8 mL/L) depending on water quality. Always follow manufacturer's directions for the particular automatic washer being used. If you have additional questions, consult your Getinge representative.

For an Automated Dosing System — First Time Use with New Product

1. It is important that the detergent delivery line is flushed before use.
 2. Prime the line to ensure that there is no/minimal air in the line.
 3. Ensure the dosing is set to the indicated amount of 0.2. to 1 oz./gal. (2 to 8 mL/L).
- **NOTE:** Follow the instrument manufacturer's IFU for Decontamination/Cleaning/Sterilization.
 - **NOTE:** Follow hospital policy for appropriate PPE to be worn during reprocessing.
 - **NOTE:** Decontamination should be performed in accordance with AAMI ST79, AORN, and CDC guidelines. Please consult your Getinge Representative for additional information.
 - **CAUTION:** Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.
 - **CAUTION:** Some plastics and rubbers might not be compatible with certain detergents or high water temperatures. Check with manufacturer for suitability.

Ordering Information

| Getinge Clean | Order Number |
|---------------|--------------|
| 3 x 5L | 6036000008 |
| 5 Gallons | 61301600040 |
| 15 Gallons | 61301600041 |
| 30 Gallons | 61301602228 |

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries. Copyright 2021 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

Manufacturer: Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office, US: 1 Geoffrey Way, Wayne, NJ 07470, USA
Sales Office, Canada: 90 Matheson Blvd West, Suite 300, Mississauga, Ontario L5R 3R3, Canada
www.getinge.com



MCV00044164 REVD
EXP 11/23