

Getinge Clean

PowerCon Acid Detergent Concentrate

PowerCon Acid Detergent Concentrate combines organic and inorganic acids with surfactants in a proprietary blend. Achieves excellent results in a wide variety of operating conditions.

PowerCon Acid Detergent Concentrate is especially effective against minerals and scale build-up.



Application

Can be used in laboratories, hospitals and other institutions on instruments, utensils, and glassware.

Key Features

- Economical
- Powerful and convenient
- High-performance cleaning
- Strong dissolving action against minerals and scale

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild, characteristic fragrance
- pH: <1.0
- Store below 120°F (49°C)

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal.

Expiration/Use-By Date

35 Months from date of manufacture.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Usage Information

Always follow manufacturer's directions for the particular automatic washer being used. Calibrate dispensing equipment to deliver a dose of 0.17 to 0.33 oz./gal. (1.3 to 2.6 mL/L). Poor water quality may require an increased product dosage. Consult your Getinge representative for appropriate chemical concentration and machine cycle information.

For an Automated Dosing System — First Time Use with New Product

1. It is important that the product delivery line is flushed before use.
 2. Prime the line to ensure that there is no/minimal air in the line.
 3. Ensure the dosing is set to the indicated amount of 0.17 to 0.33 oz./gal. (1.3 to 2.6 mL/L).
- **NOTE:** Follow the instrument manufacturer's IFU for Decontamination/Cleaning/Sterilization.
 - **NOTE:** Follow hospital policy for appropriate PPE to be worn during reprocessing.
 - **NOTE:** Decontamination should be performed in accordance with AAMI ST79, AORN, and CDC guidelines. Please consult your Getinge Representative for additional information.
 - **CAUTION:** Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.
 - **CAUTION:** Some plastics and rubbers might not be compatible with certain detergents or high water temperatures Check with manufacturer for suitability.

Ordering Information

Getinge Clean	Order Number
3 x 5L	6036000019
2.5 Gallons	61301605570
5 Gallons	61301606233

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries.
Copyright 2021 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

Manufacturer: Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office, US: 1 Geoffrey Way, Wayne, NJ 07470, USA
Sales Office, Canada: 90 Matheson Blvd West, Suite 300, Mississauga, Ontario L5R 3R3, Canada

www.getinge.com



MCV00044165 REV B
EXP 11/23