

# Getinge Clean

## PowerCon Alkaline Detergent Concentrate

PowerCon Alkaline Detergent Concentrate is a proprietary formula of powerful alkaline builders, water conditioners, anti-redeposition agents and surfactants.

PowerCon Alkaline Detergent Concentrate is ideal for removing protein, blood and fatty soils from instruments, labware, and glassware.



### Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

### Key Features

- Economical
- Powerful and convenient
- High-performance cleaning
- Effective for removal of protein, blood, fat, grease and oil
- Cleans surgical instruments, labware, and glassware

### Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild fragrance
- pH: 14.0
- Store below 120°F (49°C)

*NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.*

### Expiration/Use-By Date

35 Months from date of manufacture.

**Quality Statement:** Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

## Usage Information

Always follow manufacturer's directions for the particular automatic washer being used. Calibrate dispensing equipment to deliver a dose of 0.06 to 0.3 oz./gal. (0.5 to 2 mL/L). Poor water quality may require an increased product dosage. Consult your Getinge representative for appropriate chemical concentration and machine cycle information.

### For an Automated Dosing System — First Time Use with New Product

1. It is important that the detergent delivery line is flushed before use.
  2. Prime the line to ensure that there is no/minimal air in the line.
  3. Ensure the dosing is set to the indicated amount of 0.06 to 0.3 oz./gal. (0.5 to 2.3 mL/L).
- **NOTE:** Follow the instrument manufacturer's IFU for Decontamination/Cleaning/Sterilization.
  - **NOTE:** Follow hospital policy for appropriate PPE to be worn during reprocessing.
  - **NOTE:** Decontamination should be performed in accordance with AAMI ST79, AORN, and CDC guidelines. Please consult your Getinge Representative for additional information.
  - **CAUTION:** Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.
  - **CAUTION:** Some plastics and rubbers might not be compatible with certain detergents or high water temperatures. Check with manufacturer for suitability.
  - **CAUTION:** Not for use on soft metals such as aluminum, brass, copper or polycarbonate/polysulfone plastics.

## Ordering Information

Getinge Clean	Order Number
3 x 5L	6036000009
2.5 Gallons	61301605567
5 Gallons	61301605800

---

Getinge is a registered trademark of Getinge AB or its subsidiaries or affiliates in the United States or other countries.  
Copyright 2022 Getinge AB or its subsidiaries or affiliates. • All rights reserved.

**Manufacturer:** Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110  
**Sales Office, US:** 1 Geoffrey Way, Wayne, NJ 07470, USA  
**Sales Office, Canada:** 90 Matheson Blvd West, Suite 300, Mississauga, Ontario L5R 3R3, Canada  
[www.getinge.com](http://www.getinge.com)



MCV00044166 REV B  
EXP 11/23