
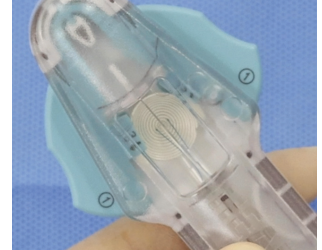


Heartstring III Proximal Seal System


Loading the Heartstring III Seal into the Delivery Device

- 1 Become familiar with the Heartstring Loading Device, including the visible number cues and verify that the seal is open and visible through the window.

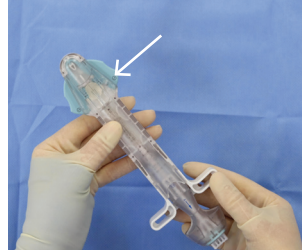

- 2 Squeeze and hold both wings of the Loading Device at the same time. Verify that the Heartstring seal is tightly rolled in the window and ① is seen in the corresponding round window.


Note: This simultaneous action is crucial for the correct loading of the seal.

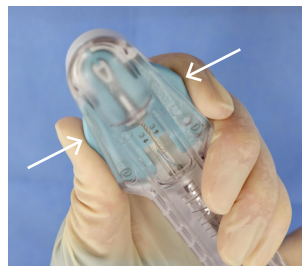


- 3 While continuing to hold the blue wings, advance the Delivery Device tube over the rolled seal. Perform this by slowly pushing on the top, blue area, of the Delivery Device until the ② visual cue appears completely in the corresponding window.



- 4 Watching the ③ arrows in the Heartstring III Proximal Seal window release the blue wings.

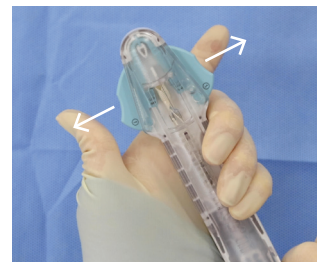

- 5 To remove the Delivery Device from the loading device, hold the loading device in the center (avoid pushing on the blue wings); pull the Delivery Device from the loading device in the direction indicated by the ④.

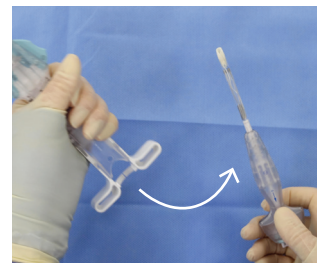

-
- 6 Visually confirm the complete loading of the Heartstring III Proximal Seal.** Check for separation, improper positioning or flaring. This step is critical for the success of the subsequent phases of the procedure.



-
- 7 Unlock the Delivery Device** lock by sliding the blue top in the direction of the arrow.



-
- 8** The Heartstring III Proximal Seal has been successfully loaded into the delivery tube and is now ready for deployment.



This guide does not replace the corresponding instructions for use. Only use the Heartstring III and all other products mentioned in this guide together with the respective instructions for use. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

GETINGE 

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Heartstring III is a registered trademark of Maquet Cardiovascular LLC • Copyright 2023 Getinge AB or its subsidiaries or affiliates • All rights reserved • ⚠ CAUTION: Federal (US) law restricts this device to sale by or on the order of a physician. Refer to Instructions for Use for current indications, warnings, contraindications, and precautions.

Manufacturer • Maquet Cardiovascular LLC • 94 Barbour Pond Drive, Wayne NJ 07470 USA

www.getinge.com

MCV00111114 REVA