

# Getinge Clean

## Neutrawash Plus

Foam controlled, liquid detergent designed for high-performance cleaning under the widest range of soil and water conditions.



### Application

An instrument cleaner detergent designed for use in mechanical cleaning equipment, cart washers, washer-disinfectors, and ultrasonic cleaners.

### Key Features

- Superior cleaning capabilities
- Works well in a range of water conditions
- Ideal for use in automated/mechanical washing applications
- Can be used with conductivity systems
- Combines detergents, emulsifiers, penetrants, cleaning agents, water softeners and corrosion inhibitors for economical cleaning
- Foam controlled and safe on all metals at use concentrations
- Incorporates a rinse aid to eliminate spotting and residual film even with various organic and inorganic soils

**Quality Statement:** Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

## Usage Information

Neutrawash Plus should be used at a dose of 0.2–1.0 oz/gal (2–8 ml/L) depending on water quality. Always follow manufacturer's directions for the particular automatic washer being used.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

## Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild fragrance
- pH value: 7.0–9.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

---

## Ordering Information

Getinge Clean	Order Number
4 x 1 Gallon	61301605889
5 Gallons	61301605890
15 Gallons	61301605891
30 Gallons	61301605892

---

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2018 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044143 REV B

**Manufacturer:** Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110  
**Sales Office:** Getinge USA Sales, LLC, 1777 East Henrietta Road, Rochester, NY 14623

[www.getinge.com](http://www.getinge.com)

**GETINGE** 