

Getinge Clean

Tec Wash III

Tec Wash III is an aluminum safe, alkaline detergent designed for use in a wide variety of operating conditions.



Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

Key Features

- Highly effective cleaner, especially on proteins
- Leaves no film to stain instruments, utensils or glassware
- Safe on aluminum and stainless steel

Usage Information

Tec Wash III should be used at a dose of 0.2–1.0 oz/gal (2–8 ml/L) depending on water quality. Follow manufacturer's directions for the particular automatic washer being used. If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild fragrance
- pH value: 11.0–13.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling, storage and waste disposal methods.

Ordering Information

Getinge Clean	Order Number
4 x 1 Gallon	61301605275
5 Gallons	61301667776
15 Gallons	61301667777
30 Gallons	61301600844

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2018 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044145 REV B

Manufacturer: Intercon Chemical Co. 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge USA Sales, LLC, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 