

# Getinge Clean

## PowerCon Triple Enzyme Detergent Concentrate

PowerCon Triple Enzyme Detergent Concentrate is a highly concentrated liquid combination of surfactants, conditioners, builders, stabilizers and a proprietary blend of enzymes.



### Application

For use in mechanical cleaning equipment, cart washers, instrument washers, washer-disinfectors, sinks and ultrasonic baths.

### Key Features

- Economical
- Delivers high-performance cleaning
- Combines power and convenience
- Multiple-enzymatic compound designed to quickly and effectively remove blood, tissue, mucous, fats and other soils

### Usage Information

Always follow manufacturer's directions for the particular automatic washer being used. Calibrate dispensing equipment to deliver a minimum of 1/12 of an ounce per gallon of water (0.65 ml/L) up to 1/2 of an ounce per gallon (3.9ml/L). Poor water quality may require an increased product dosage. Consult your Getinge representative for appropriate chemical concentration and machine cycle information.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

**Quality Statement:** Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

## Physical and Chemical Properties

- Clear, pink color
- Peach fragrance
- pH value: 7.5–8.5

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

---

## Ordering Information

Getinge Clean	Order Number
2 x 1 Gallon	61301605563
2.5 Gallons	61301605564
5 Gallons	61301605803
15 Gallons	61301606581

---

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044172 REVA

**Manufacturer:** Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110  
**Sales Office:** Getinge, 1777 East Henrietta Road, Rochester, NY 14623

[www.getinge.com](http://www.getinge.com)

**GETINGE** 