

T-DOC Sterile Supply Workflow

Learn how T-DOC manages the sterile supply flow

Use → and ← to navigate through the entire workflow in this presentation or go to specific area/location:



Soiled Area

Return units

Create washer batch

Clean Area

Handle washer batch

Pack units

Create sterilizer batch

Sterile Area

Handle sterilizer batch

Dispatch

Order Pick

Surgical Dep.

Operation

Create order

Office/Admin Area


T-DOC Overviews

T-DOC Admin

T-DOC Reports

The sterile supply flow


Learn how T-DOC manages the sterile supply flow

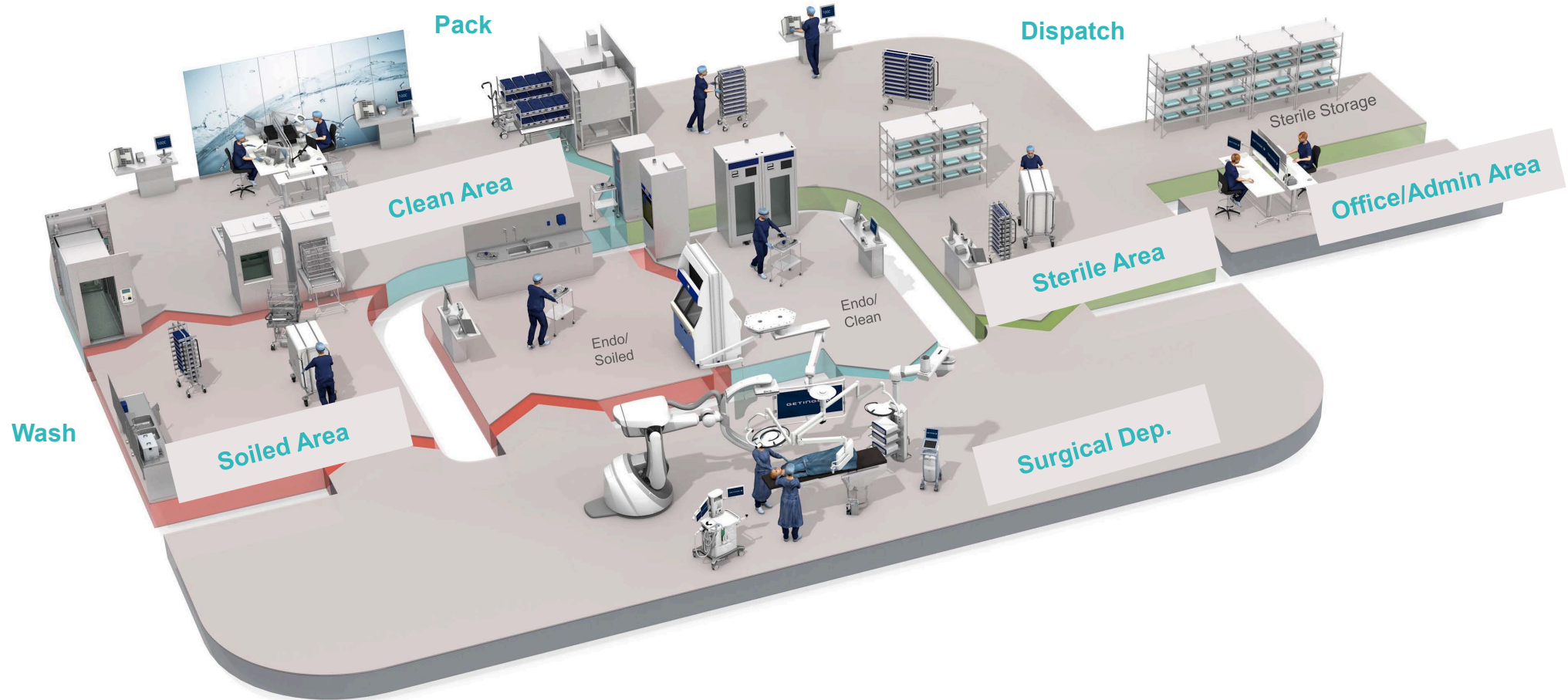

Interactive – Click to Jump to a new Area!

Soiled Area


Learn how T-DOC manages the sterile workflow in the Soiled Area

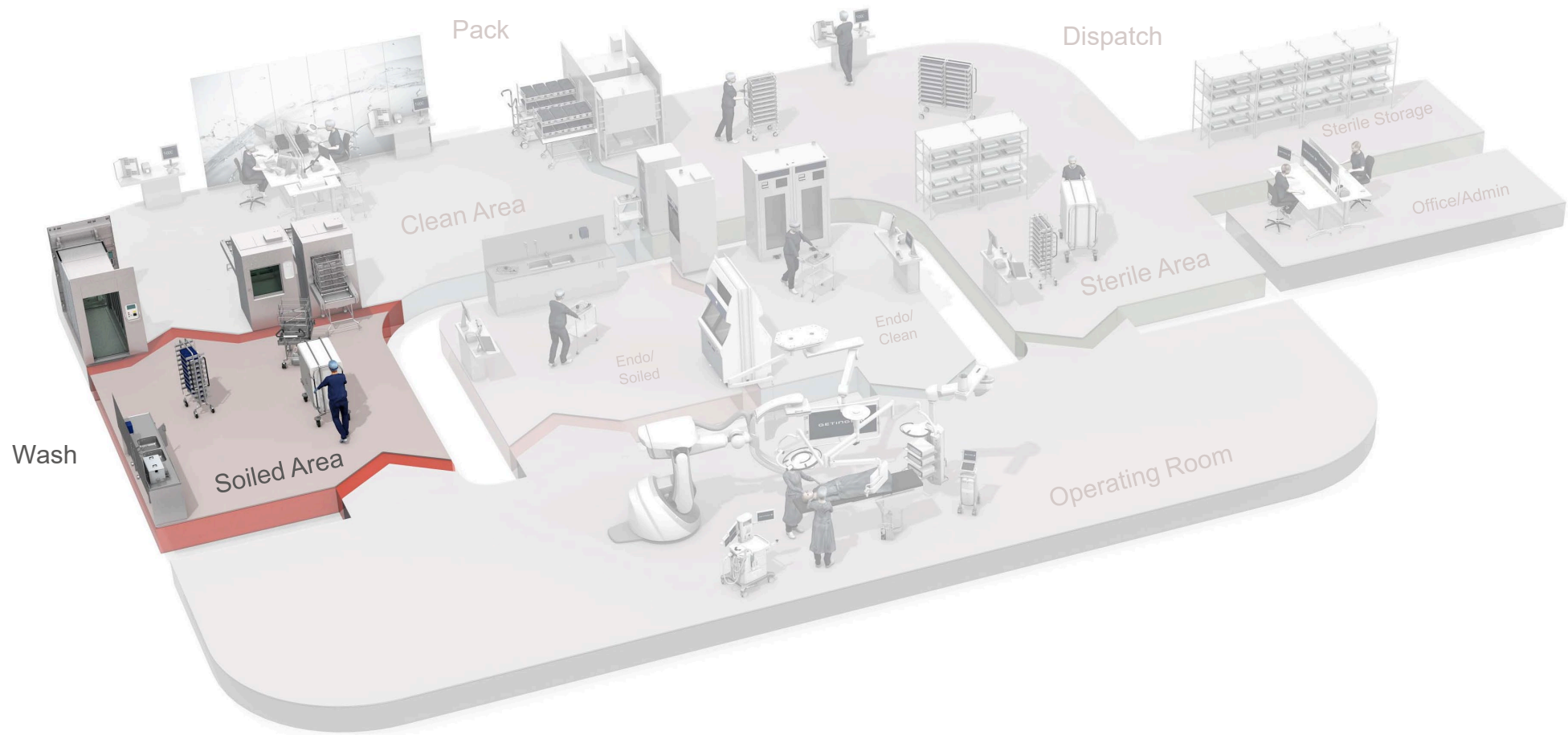


Soiled Area

Learn how T-DOC manages the sterile workflow in the Soiled Area

- Return of used goods
- Support from external documents
- Handling of fast-tracked instruments
- Comments from the OR
- Display of a trigger
- Creation of washer batch

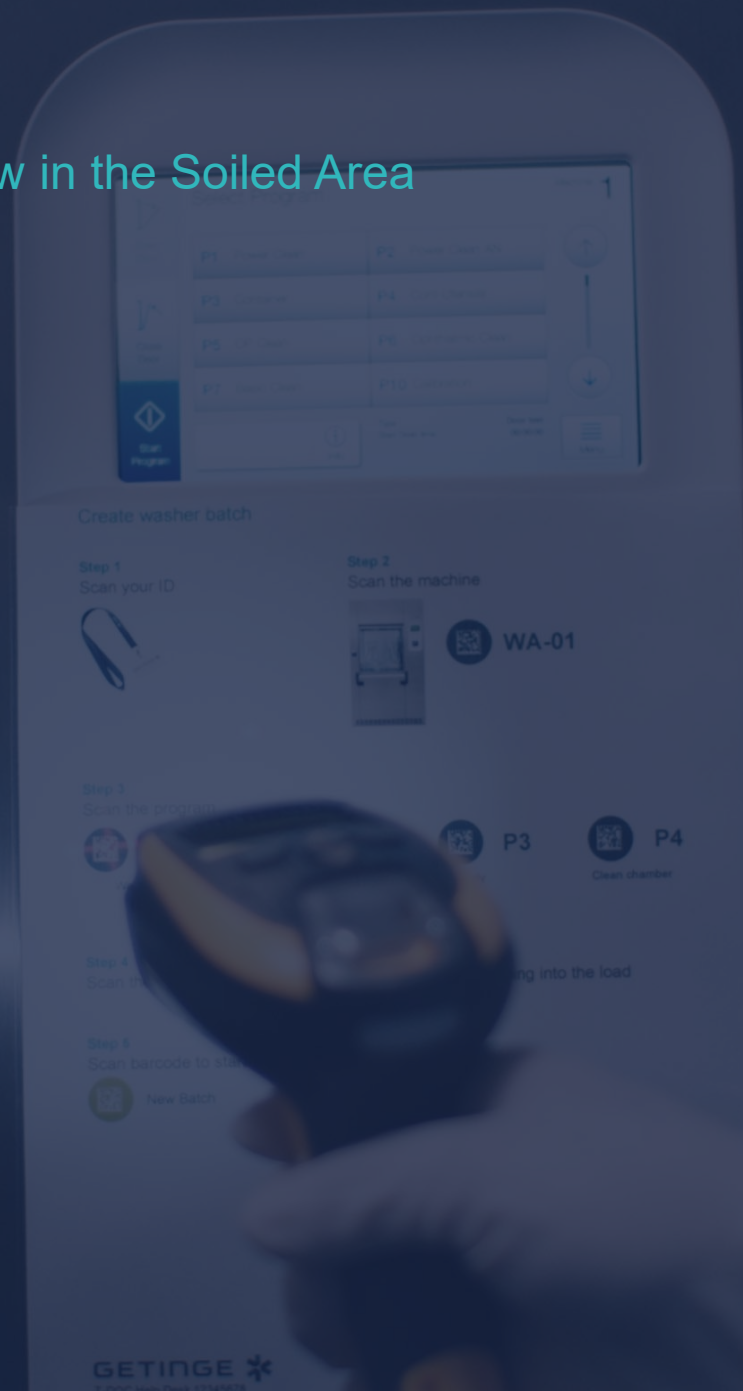

Return units

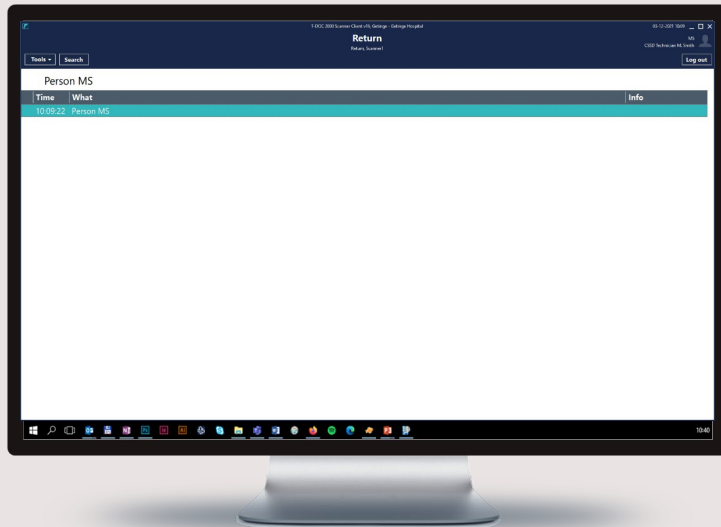

A barcode template assists the user in scanning the correct barcodes in the correct order

Return
Return instruments to decontamination

ReturnArea
scan barcode to enter area

Step 1
Scan your ID

Step 2
Scan the tray barcode

Step 3
Scan the issue

Step 4
Scan the tray barcode to assign the issue

Log out
scan barcode to log out

GETINGE
T-DOC Help Desk 22399060

Log out
scan barcode to log out

If there is an issue with a tray, follow the steps below


- Cancelled
- Cleanup
- Batch registration error
- Discontinued logging
- Protocol error
- General Washer error



Return

Return, Scanner1

Log out

 Scan user

Tools ▾

Search

Person MS

Time

What

Info


10:09:22 Person MS



Return

Return, Scanner1

Log out

 Scan return goods from the OR

Tools ▾

Search

Return 1532672/CVOR-04615 Dilator Savary 60FR/20mm

Time	What	Info
10:12:38	Return 1532672/CVOR-04615 Dilator Savary 60FR/20mm	
i 10:12:38	Unit has error assigned 1532672 50 Lid Popped on Shelf	
10:12:02	Return 1532676/P-01825 C-Section Tray (7BOB)	
10:11:41	Return 1532796/4298 Pelviscopy Tray	
10:09:22	Person MS	

Easy access to external documents, e.g., supplier instructions for assembly

Number	Info	Document
 1	Product	Savary-Gilliard Dilators.PDF

MCV00109020 REVA

Page 7



Return

Return, Scanner1


MS

CSSD Technician M. Smith



Log out

Tools ▾

Search

 Scan fast-tracked unit

 Unit is fast tracked!! 1533235 ReturnToPack URGENT: Prioritized equipment for Ortho. Dep.

Time	What	Info
 10:15:53	Unit is fast tracked!! 1533235 ReturnToPack URGENT: Prioritized equipment for Ortho. Dep.	
10:15:53	Return 1533235/P-57332 HALL SAW KEY (CVOR)	
10:12:38	Return 1532672/CVOR-04615 Dilator Savary 60FR/20mm	
 10:12:38	Unit has error assigned 1532672 50 Lid Popped on Shelf	
10:12:02	Return 1532676/P-01825 C-Section Tray (7BOB)	
10:11:41	Return 1532796/4298 Pelviscopy Tray	
10:09:22	Person MS	

Number	Info	Document
 1	Product	Savary-Gilliard Dilators.PDF

MCV00109020 REVA

Page 8



Scan a unit with triggers and comments and the notification list displays

Return

Return, Scanner1

Log out

Tools Search

Notification list

Unit 1533258 - Basic Total Hip

OR-10: Hole In Wrap

4147: Basic Total Hip
0989: Please remember to oil before assembly

Handled elements

OR-10: Needs sharpening

Wrapping material was torn. Please ensure cautious handling. Thank you.

Do not show again at any location

Displayed comment with text and/or images

Unhandled trigger

Handled comment



Confirm

Cancel

Return

Return, Scanner1

Log out

Tools Search Transfer

Trigger with visual handling instruction


Notification list

Unit 1533291 - Basic Total Hip

4147: Basic Total Hip
0912433: Disassemble in 4 parts.

Handled elements

! 4147: Basic Total Hip
0912433: Disassemble in 4 parts.
Unit 1533291 - Basic Total Hip

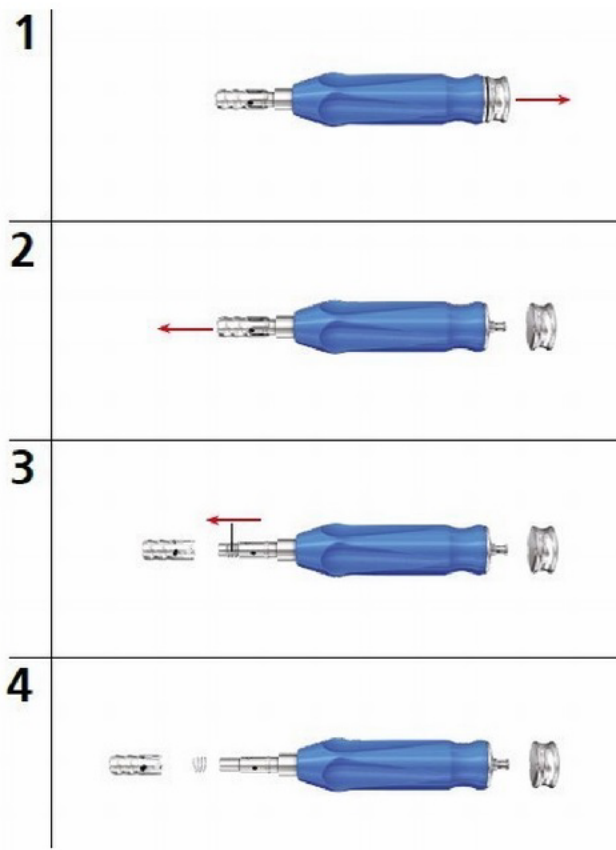


Reject

Confirm


Cancel

Create washer batch

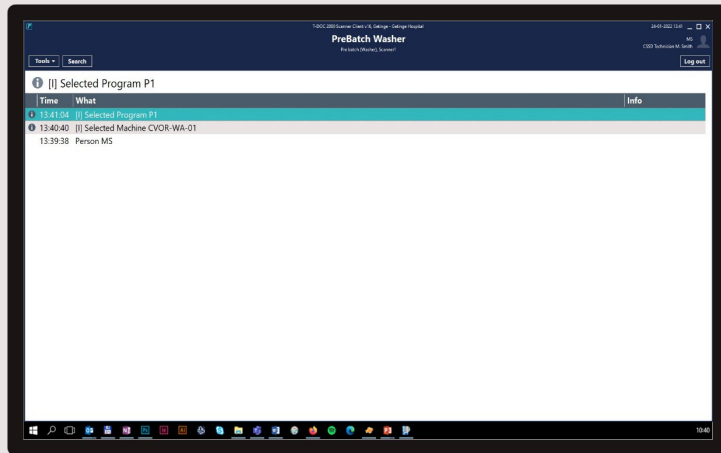


A barcode template assists the user in scanning the correct barcodes in the correct order


Washer-Disinfecter 1
Create washer batch


WashPreBatchArea 
scan barcode to enter area

Step 1
Scan your ID







Step 2
Scan the machine



 **WA-01**


Step 3
Scan the program


 **SH**  **P3**  **G3**  **P1**


Short Program Manual LRT Leak test program POROUS LOAD


Step 4
Scan the barcodes of all units and biological indicators going into the load

Step 5
Scan barcode to create the washer batch

 **New Batch**

Log out 
scan barcode to log out

GETINGE 
T-DOC Help Desk 22399060



PreBatch Washer


Pre batch (Washer), Scanner1




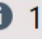
Log out

Tools ▾

Search

 Scan user, machine, and program

[I] Selected Program P1


Time	What	Info
 13:41:04	[I] Selected Program P1	
 13:40:40	[I] Selected Machine CVOR-WA-01	
13:39:38	Person MS	



PreBatch Washer

Pre batch (Washer), Scanner1

Log out

 Scan units


Unit 1532117 TRAY CRABTREE VIDEO MEDIASTINOSCOPE (CVOR) put in Wash load 1

Time	What	Info
09:57:04	Unit 1532117 TRAY CRABTREE VIDEO MEDIASTINOSCOPE (CVOR) put in Wash load 1	
09:57:01	Unit 1532152 Delicate Plastic Tray put in Wash load 1	
09:56:57	Unit 1532248 Basic Tray put in Wash load 1	
09:56:49	Unit 1532384 Camera, Stryker AIM w/light Cord put in Wash load 1	
<i>i</i> 09:56:42	[I] Selected Program P1	
<i>i</i> 09:56:40	[I] Selected Machine CVOR-WA-01	
09:56:37	Person MS	

MCV00109020 REVA

PreBatch Washer



Pre batch (Washer), Scanner1

 Create batch

Log out

Tools ▾ Search Transfer


New Batch created 398814

Time	What	Info
11:20:41	New Batch created 398814	
11:20:32	Unit 1532117 TRAY CRABTREE VIDEO MEDIASTINOSCOPE (CVOR) put in Wash load 1	
11:20:24	Unit 1532152 Delicate Plastic Tray put in Wash load 1	
11:20:11	Unit 1532248 Basic Tray put in Wash load 1	
11:19:59	Unit 1532384 Camera, Stryker AIM w/light Cord put in Wash load 1	
 11:19:32	[I] Selected Program P1	
 11:19:30	[I] Selected Machine CVOR-WA-01	
11:19:19	Person MS	

MCV00109020 REVA

Clean Area

Learn how T-DOC manages the sterile workflow in the Clean Area

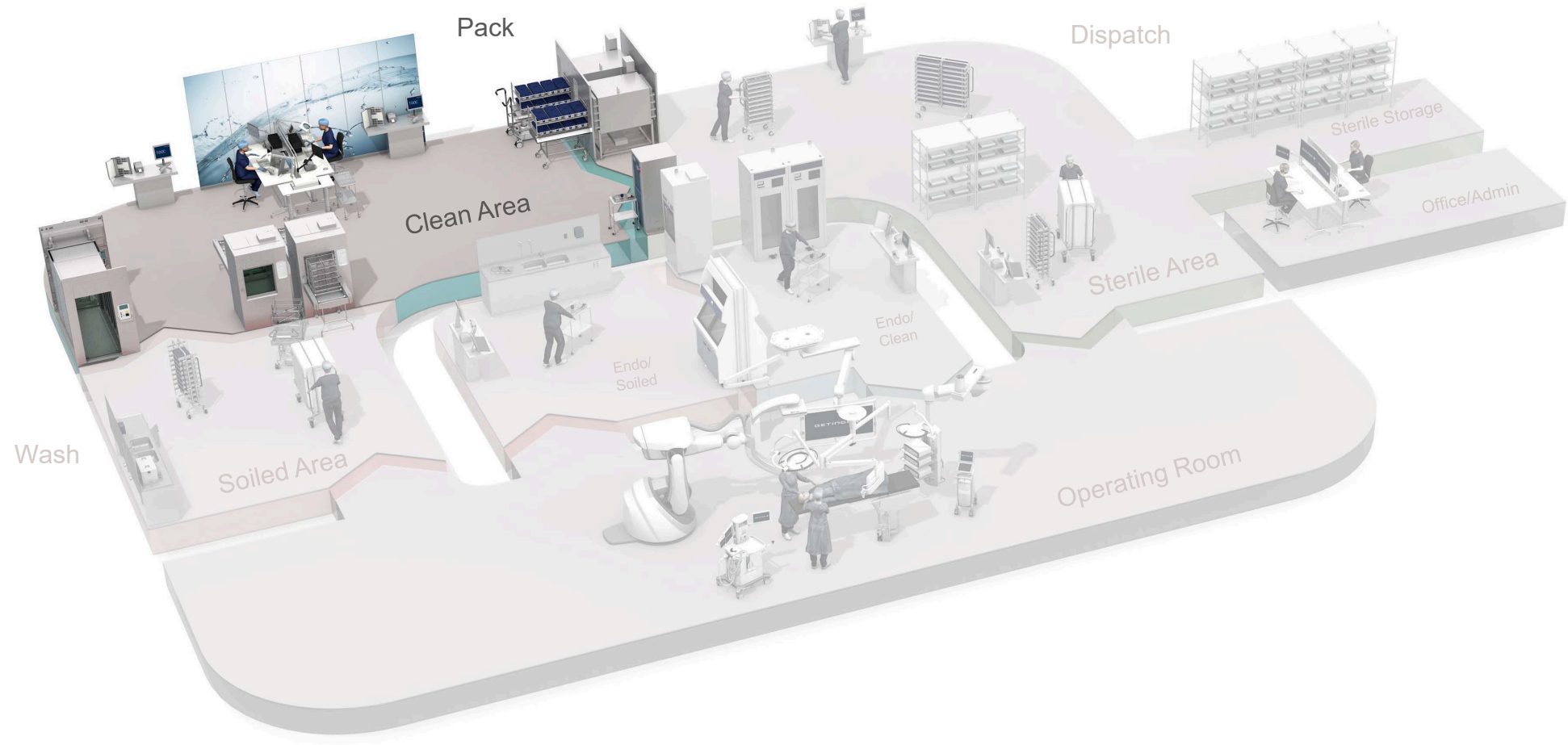


Clean Area

Learn how T-DOC manages the sterile workflow in the Clean Area

- Handling and approval of washer batch
- How to search and locate items
- Triggers
- Registration of instrument count
- Creation of sterilizer batch



Handle washer batch

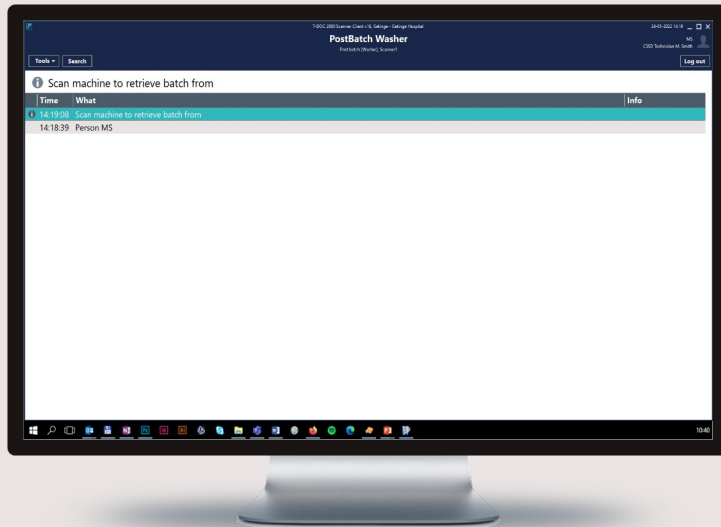

A barcode template assists the user in scanning the correct barcodes in the correct order

Approve a batch
Approve a load from washers

WashPostBatchArea
scan barcode to enter area

Step 1
Scan your ID

Step 2
Scan barcode to use last batch

Step 3
Scan the machine

Step 4
Scan barcode to approve batch

GETINGE
T-DOC Help Desk 22399060

Disapprove a batch
Disapprove a washer batch

WashPostBatchArea
scan barcode to enter area

Step 1
Scan your ID

Step 2
Scan barcode to use last batch

Step 3
Scan the machine

Step 4
Scan reason for disapproval

Step 5
Scan barcode to disapprove batch

GETINGE
T-DOC Help Desk 22399060

Log out
scan barcode to log out



PostBatch Washer


Post batch (Washer), Scanner1




Log out

Tools ▾

Search

 Scan machine

 Scan machine to retrieve batch from

Time

What

Info

 14:19:08 Scan machine to retrieve batch from


14:18:39 Person MS





PostBatch Washer

Post batch (Washer), Scanner1




 Retrieve process.
Batch includes fast tracked units

Tools ▾

Search

Log out

Process 398806

Time	What	Info
 14:20:22	Process 398806	
 14:20:22	[I] Selected Machine CVOR-WA-01	
 14:19:08	Scan machine to retrieve batch from	
14:18:39	Person MS	



PostBatch Washer

Post batch (Washer), Scanner1

Automatic pop-up of graph with equipment process for visual approval


Tools ▾

Search

Log out

Process 398806

WA-1 Washer progress

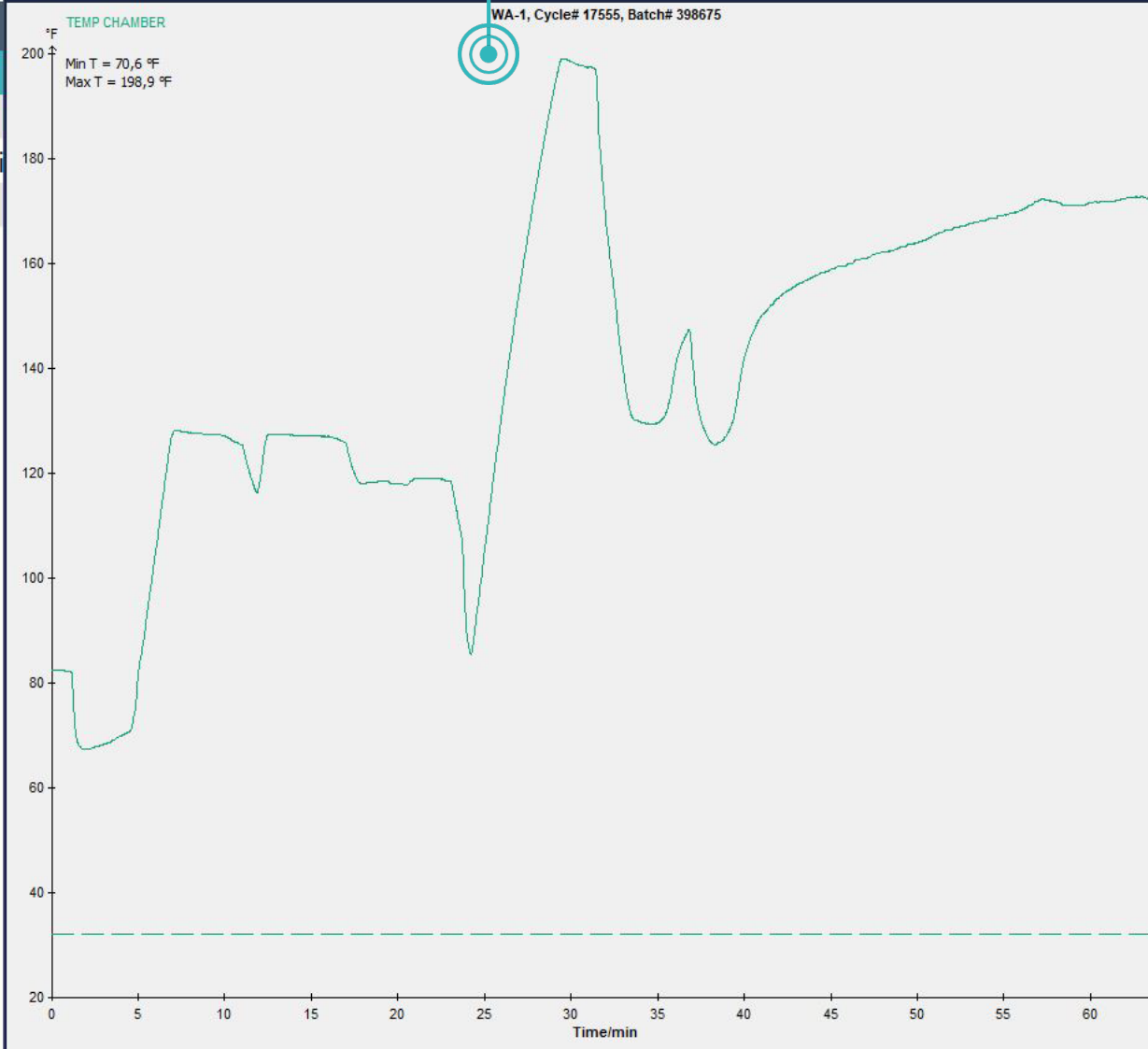

Time	Temp (°F)
DRAINING SHORT	
0:00:01	82,3
PRE RINSE CW	
0:00:11	82,3
0:01:48	67,2
0:02:00	67,2
DRAINING LONG	
0:03:16	68,4
0:04:00	69,8
WASHING	
0:04:25	70,5
0:04:30	70,5
0:05:00	81,1
0:05:30	92,7
0:06:00	104,6
0:06:30	116,9
0:07:00	127,6
0:07:07	128,0
0:07:30	127,8
0:08:00	127,7
0:08:30	127,5
0:09:00	127,3
0:09:30	127,3
DRAINING LONG	
0:09:43	127,2
0:10:00	127,0
0:10:30	126,1
WASHING 2	
0:10:56	125,3
0:11:00	125,2
0:11:30	119,1
0:11:49	116,3
0:12:00	118,0
0:12:30	127,2
0:12:36	127,4
0:13:00	127,3
0:13:30	127,3
0:14:00	127,0
0:14:30	127,0
0:15:00	127,0
0:15:30	127,0
DRAINING LONG	
0:15:45	127,0
0:16:00	127,0
0:16:30	126,5
RINSE 1	
0:16:54	125,8
0:17:00	125,5
0:17:30	119,7
0:18:00	118,0
0:18:31	118,2
DRAINING LONG	
0:18:46	118,2
0:19:00	118,5
0:19:30	118,2

```

PROGRAM: P1 INSTRUMENTS
DATE: 04/12/2020
PROCESS START: 22:20:26
MACHINE NAME: S8666-7
MACHINE ID: 1
CYCLE COUNTER: 17555

SIGNALS
AI07 TEMP INDEP CHAMB
-----
PROGTIME AI07
-----
PH1-START
00:00:00 82.5

PH2-DRAINING SHORT
00:00:00 82.1

PH220-PRE RINSE CW
00:00:11 82.1

PH20-DRAINING LONG
00:03:16 68.0

PH25-WASHING
00:04:25 70.1
00:05:26 90.3
DOS START
00:06:18 111.4
DOS STOP
00:06:42 121.1
00:06:42 121.4
00:07:42 127.5
00:08:42 127.2
00:09:42 127.0
00:09:43 127.0

PH20-DRAINING LONG
00:09:43 127.0

PH30-WASHING 2
00:10:56 125.2
00:11:47 116.4
DOS START

IDENTIFICATION: S8666-7
-----
PROGTIME AI07
00:12:44 127.2
DOS STOP
00:12:45 127.1
00:12:45 127.1
00:13:45 127.0
00:14:45 126.7
00:15:45 126.7
00:15:46 126.7

PH20-DRAINING LONG
00:15:46 126.7

PH330-RINSE 1
00:16:55 125.7

PH20-DRAINING LONG
00:18:47 118.0
  
```

Start date: 05-12-2020

Start time: 09:05:01

Program: P1 - INSTRUME

Internal program: 83

Cycle: 17555

Batch: 398675

Exp.Temp: []

Exp. Time: []

- Log
- Full Log
- Native
- 2nd
- Set Ref
- Ref
- Log(x)
- Close

MCV00109020 REVA

PostBatch Washer


Post batch (Washer), Scanner1





 Approve batch

Tools ▾

Search


Log out

 Batch approved 398806/MS

Time	What	Info
 14:31:30	Batch approved 398806/MS	
 14:20:22	Process 398806	
 14:20:22	[I] Selected Machine CVOR-WA-01	
 14:19:08	Scan machine to retrieve batch from	
14:18:39	Person MS	




Pack units

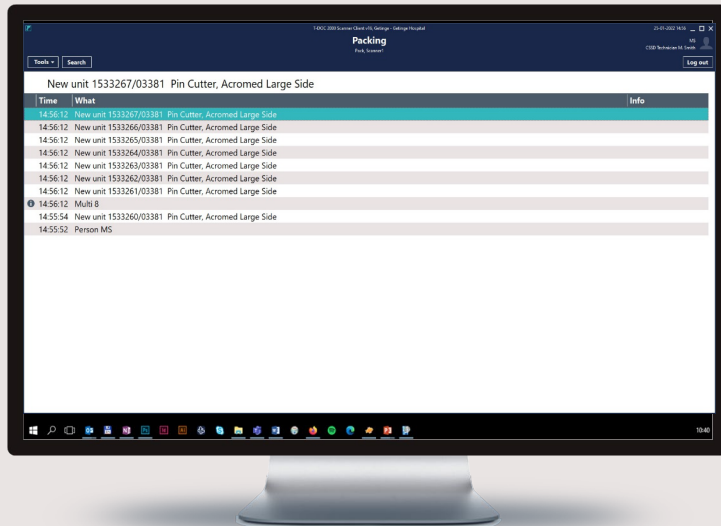


A barcode template assists the user in scanning the correct barcodes in the correct order


Packing
Create units

Pack area 
scan barcode to enter area

Step 1
Scan your ID




Step 2
Trays
Scan the tray barcode




For non-scannable barcodes search for the serial number


Single items
Search for the product




Use "Numbers" template for creating multiple units

Step 3
Use CountAssure to complete packing




GETINGE 
T-DOC Help Desk

Log out 
scan barcode to log out



Goods can be packed in various ways.
This window shows how to pack multi single instruments

Packing
Pack, Scanner1


 Scan multi single instruments

Log out

Tools ▾ Search

New unit 1533267/03381 Pin Cutter, Acromed Large Side



Time	What	Info
14:56:12	New unit 1533267/03381 Pin Cutter, Acromed Large Side	
14:56:12	New unit 1533266/03381 Pin Cutter, Acromed Large Side	
14:56:12	New unit 1533265/03381 Pin Cutter, Acromed Large Side	
14:56:12	New unit 1533264/03381 Pin Cutter, Acromed Large Side	
14:56:12	New unit 1533263/03381 Pin Cutter, Acromed Large Side	
14:56:12	New unit 1533262/03381 Pin Cutter, Acromed Large Side	
14:56:12	New unit 1533261/03381 Pin Cutter, Acromed Large Side	
 14:56:12	Multi 8	
14:55:54	New unit 1533260/03381 Pin Cutter, Acromed Large Side	
14:55:52	Person MS	



Packing

Pack, Scanner1

Log out

Search for units, serial numbers, products, and items,
and find their location

Tools ▾

Search

Products

Units


Serial numbers

Customers

Locate product

Locate item

Locate product: 4147 - Basic Total Hip, count 14



Selected fields

Product search

4147

Ready for use - 8

Packed - 1

Returned - 4

Used - 1

Other - 1

Drag a column header here to group by that column

Unit	Serial no..Serial no.	Location.Name	Location time	Status	Customer.Customer	Customer.Name	Tag
1512833	4147-06	R6K2	03-12-2020 03:13:54	On stock			
1514222	4147-04	R6K2	30-10-2020 06:04:01	On stock	ELL OR	ELL OR	
1514224	4147-01	R6K2	30-10-2020 06:04:02	On stock	ELL OR	ELL OR	
1517485	4147-02	R6K2	05-11-2020 13:01:12	On stock	ELL OR	ELL OR	
1523024	4147-09	R6K2	17-11-2020 01:46:32	On stock	ELL OR	ELL OR	
1527883	4147-03	R6K2	25-11-2020 06:19:16	On stock	ELL OR	ELL OR	
1530736	4147-07	R6K2	02-12-2020 20:11:54	On stock	ELL OR	ELL OR	
1531112	4147-08	R6K2	03-12-2020 12:26:47	On stock	ELL OR	ELL OR	

Goods can be packed in various ways.
The following flow shows how to pack a tray

Packing

Pack, Scanner1

Trigger with handling instruction displays
when tray is scanned

Log out

Tools Search

Notification list

Product 4147 - Basic Total Hip

4147: Basic Total Hip
0121: Blades are not interchangeable

4147: Basic Total Hip
0121: Blades are not interchangeable

Handled elements

Blades are not interchangeable with old set. Check to make sure blades fit.



Confirm

Cancel

Instrument count window opens

Basic tray data

Expand to view and address comments, triggers, repairs, and missing/surplus items

Packing

Pack, Scanner1

Tray item count

Supportive media

MS

CSSD Technician M. Smith

Log out

Tools

Search

Current contents list: 4147-14 - 4147 Basic Total Hip

Products.Product 4147

Program groups.Name Prevac 4/30/275°

Products.Color Code


Packings.Description Closed Container

Count changes Notifications Missing Surplus Replace Repair View

	Item	Item	Count	Present		
1	✓ S-07955	Retractor, Hohmann Extra Large (Pineda)	1 (1)	→ 1	▼	▲
2	✓ S-13188	Retractor, Small Muller	1 (1)	→	1	▼
3	✓ S-01152	Retractor, Ribbon 2"	1 (1)	→	1	▼
4	✓ S-09284	Retractor, Acetabular Left	2 (2)	→	2	▼
5	✓ S-08015	Extractor, Femoral Head (Corkscrew)	1 (1)	→	1	▼
6	✓ S-08013	Caliper, Femoral Head	3 (2)	→	3	▼
7	✓ S-09994	Retractor, Medium Sharp Hohmann	1 (1)	→	1	▼
8	✓ S-09142	Retractor, Hohmann Sharp	1 (1)	→	1	▼
9	✓ S-06452	Skid, Murphy-Lane Bone	1 (1)	→	1	▼
10	✓ S-08548	Retractor, Hip Bar	1 (1)	→	1	▼
11	✓ S-08991	Retractor, Large "J" or ("L") Hook w/Lock	1 (1)	→	1	▼
12	✓ S-13060	Retractor, Small "J" Hook	1 (1)	→	1	▼
13	✓ S-08988	Retractor, Large "J" (or "L") Hook	1 (1)	→	1	▼
14	✓ S-13061	Retractor, Small "L" Hook w/lock	1 (1)	→	1	▼
15	✓ S-23307	Kelly Clamp	2 (1)		2	+

Serial numbers:

S-23307-01, S-23307-02

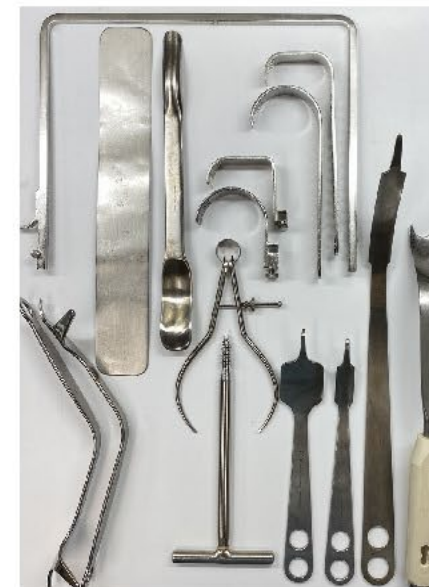

Items.Local name

MCV00109020.REVA

Suspend

OK

Cancel

Critical count is met: Standard: 3 pcs.
Acceptable: 2 pcs.
 Unacceptable: 1 pcs.

Packing

Pack, Scanner1

Log out

Tools ▾ Search

Current contents list: 4147-14 - 4147 Basic Total Hip

Products.Product 4147

Program groups.Name Prevac 4/30/275°

Products.Color Code

Packings.Description Closed Container

Count changes Notifications Missing Surplus Replace Repair ▾ View ▾

	Item	Item	Count	Present	
1	✓ S-07955	Retractor, Hohmann Extra Large (Pineda)	1 (1)	➡	1 ▾ ▲
2	✓ S-13188	Retractor, Small Muller	1 (1)	➡	1 ▾ ▲
3	✓ S-01152	Retractor, Ribbon 2"	1 (1)	➡	1 ▾ ▲
4	✓ S-09284	Retractor, Acetabular Left	2 (2)	➡	2 ▾ ▲
5	✓ S-08015	Extractor, Femoral Head (Corkscrew)	1 (1)	➡	1 ▾ ▲
6	✓ S-08013	Caliper, Femoral Head	3 (2)	➡	2 ▾ ▲
7	✓ S-09994	Retractor, Medium Sharp Hohmann	1 (1)	➡	1 ▾ ▲
8	✓ S-09142	Retractor, Hohmann Sharp	1 (1)	➡	1 ▾ ▲
9	✓ S-06452	Skid, Murphy-Lane Bone	1 (1)	➡	1 ▾ ▲
10	✓ S-08548	Retractor, Hip Bar	1 (1)	➡	1 ▾ ▲
11	✓ S-08991	Retractor, Large "J" or ("L") Hook w/Lock	1 (1)	➡	1 ▾ ▲
12	✓ S-13060	Retractor, Small "J" Hook	1 (1)	➡	1 ▾ ▲
13	✓ S-08988	Retractor, Large "J" (or "L") Hook	1 (1)	➡	1 ▾ ▲
14	✓ S-13061	Retractor, Small "L" Hook w/lock	1 (1)	➡	1 ▾ ▲
15	✓ S-23307	Kelly Clamp	2 (1)		2 + -

Serial numbers:
 S-23307-01, S-23307-02

MCV00109020.REVA



Press OK when counting is done

Items.Local name

Suspend

OK

Cancel

Print label and pack list



P0034 OP Sheet **GETINGE** ✱

Small Basic Tray 1

Customer: **8348** **P0034-001** Packed: **24-01-22** by **SYS**

Prog: **Expire date: 23-02-22**

Barcode 1: **8348** **P0034** **Small Basic Tray 1**

Barcode 2: **8348** **P0034** **Small Basic Tray 1**


Order no.: 49931 IS01140-1 Type 1
LOT: 993150523

Label and pack list with tray content are printed and put on/in the respective tray

Packing list

Bookwalter Retractor, Kit III

Details

Product no. P0001	Serial no. P00001-02	Packing material Container
Customer OR Core	Sterilization Steam	Packed by Ana C
Batch Barcode 	Unit no. 11	Created January 20, 2022 3:12 PM




Contents

Item name	Supplier no.	Count
Table Post, 12"	50-4581	1
Horizontal Flex Bar, 360° Rotation	50-4582	1
Lightweight Post Coupling, 1" x 4"	50-4554	1
Oval Ring, Medium, 10.5" x 14.5"	50-4557	1
Tilt Ratchet Mechanism	50-4580	7
Blade, Malleable, 3" x 6"	50-4567	1
Blade, Malleable, 2" x 6"	50-4561	2
Blade, Malleable, 1.5" x 6"	50-4562	1
Blade, Harrington, 2.5" x 5 5/8"	50-4566	1
Retractor, Kelly, 8"	67-7497	1

Total number of items: 17




Create sterilizer batch

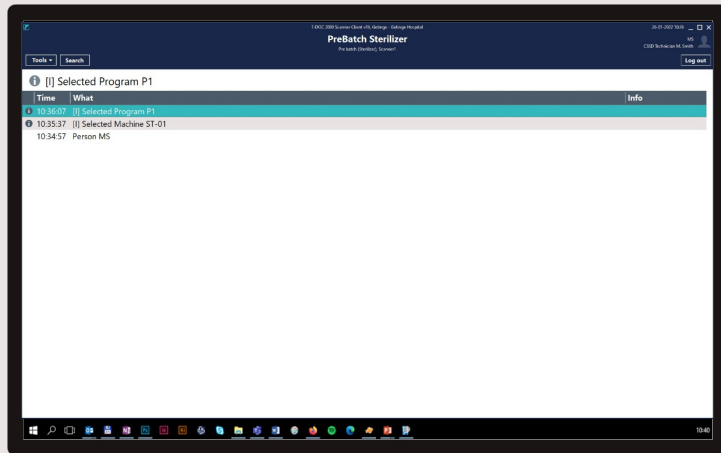


A barcode template assists the user in scanning the correct barcodes in the correct order

Sterilizer 01


Create load

Pre sterilization  scan barcode to enter area

Step 1
Scan your ID 

Step 2
Scan the machine   S1
Sterilizer 01

Step 3
Scan the program

 P1 4M 30D 275F	 P2 8M 30D 275F	 P3 10M 30D 275F	 P4 5M 30D 275F
 P5 4M 30D 275F	 P6 8M 30D 275F	 P7 10M 30D 275F	 P8 5M 30D 275F

Step 4
Scan all units and biologicals going into sterilizer load

Step 5
Scan barcode to start load  New batch

Log out  scan barcode to log out

GETINGE 
T-DOC Help Desk 886-333-3862







PreBatch Sterilizer

Pre batch (Sterilizer), Scanner1

Log out

 Scan machine and program



 [I] Selected Program P1

Time	What	Info
------	------	------

10:36:07	[I] Selected Program P1	
----------	-------------------------	--

10:35:37	[I] Selected Machine ST-01	
----------	----------------------------	--

10:34:57	Person MS	
----------	-----------	--




PreBatch Sterilizer

Pre batch (Sterilizer), Scanner1



Log out

Tools ▾

Search

 Scan all units


Unit 1533269 Clamp, Allis 6" put in Load 1

Time	What	Info
10:48:20	Unit 1533269 Clamp, Allis 6" put in Load 1	
10:47:09	Unit 1533268 DaVinci Robotic Lap Tray put in Load 1	
10:45:54	Unit 1533183 DaVinci Robotic Lap Tray put in Load 1	
10:45:52	Unit 1533250 HANDLE, BLUE GRAFT KNIFE (ARTHREX) put in Load 1	
10:45:47	Unit 1533267 Pin Cutter, Acromed Large Side put in Load 1	
 10:45:39	[I] Selected Program P1	
 10:45:37	[I] Selected Machine ST-01	
10:45:36	Person MS	

MCV00109020 REVA

PreBatch Sterilizer


Pre batch (Sterilizer), Scanner1

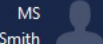


Log out




Tools ▾

Search

 Error when trying to scan a unit already connected to another batch




 Unit assigned to batch 398804

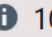Time	What	Info
 10:41:07	Unit assigned to batch 398804	
10:40:54	Unit 1533250 HANDLE, BLUE GRAFT KNIFE (ARTHREX) put in Load 1	
10:40:42	Unit 1533267 Pin Cutter, Acromed Large Side put in Load 1	
10:39:04	Unit 1533259 Basic Total Hip put in Load 1	
 10:36:07	[I] Selected Program P1	
 10:35:37	[I] Selected Machine ST-01	
10:34:57	Person MS	



PreBatch Sterilizer



Pre batch (Sterilizer), Scanner1

 Scan all units in the batch

Log out

Tools ▾ Search Transfer

Unit 1533188 Clamp, Allis 6" put in Load 1

Time	What	Info
10:05:15	Unit 1533188 Clamp, Allis 6" put in Load 1	
10:05:06	Unit 1533223 HANDLE, BLUE GRAFT KNIFE (ARTHREX) put in Load 1	
10:04:42	Unit 1533263 Pin Cutter, Acromed Large Side put in Load 1	
10:04:34	Unit 1533284 Retractor, Dull Pedi Weitlander 4 3/8" put in Load 1	
 10:03:02	[I] Selected Program P1	
 10:02:31	[I] Selected Machine ST-01	
10:00:40	Person MS	

MCV00109020 REVA

PreBatch Sterilizer

Pre batch (Sterilizer), Scanner1

Log out

Tools Search Transfer

Create batch


New Batch created 398813

Time	What	Info
10:06:52	New Batch created 398813	
10:05:15	Unit 1533188 Clamp, Allis 6" put in Load 1	
10:05:06	Unit 1533223 HANDLE, BLUE GRAFT KNIFE (ARTHREX) put in Load 1	
10:04:42	Unit 1533263 Pin Cutter, Acromed Large Side put in Load 1	
10:04:34	Unit 1533284 Retractor, Dull Pedi Weitlander 4 3/8" put in Load 1	
10:03:02	[I] Selected Program P1	
10:02:31	[I] Selected Machine ST-01	
10:00:40	Person MS	

MCV00109020 REVA

Sterile Area

Learn how T-DOC manages the workflow in the Sterile Area

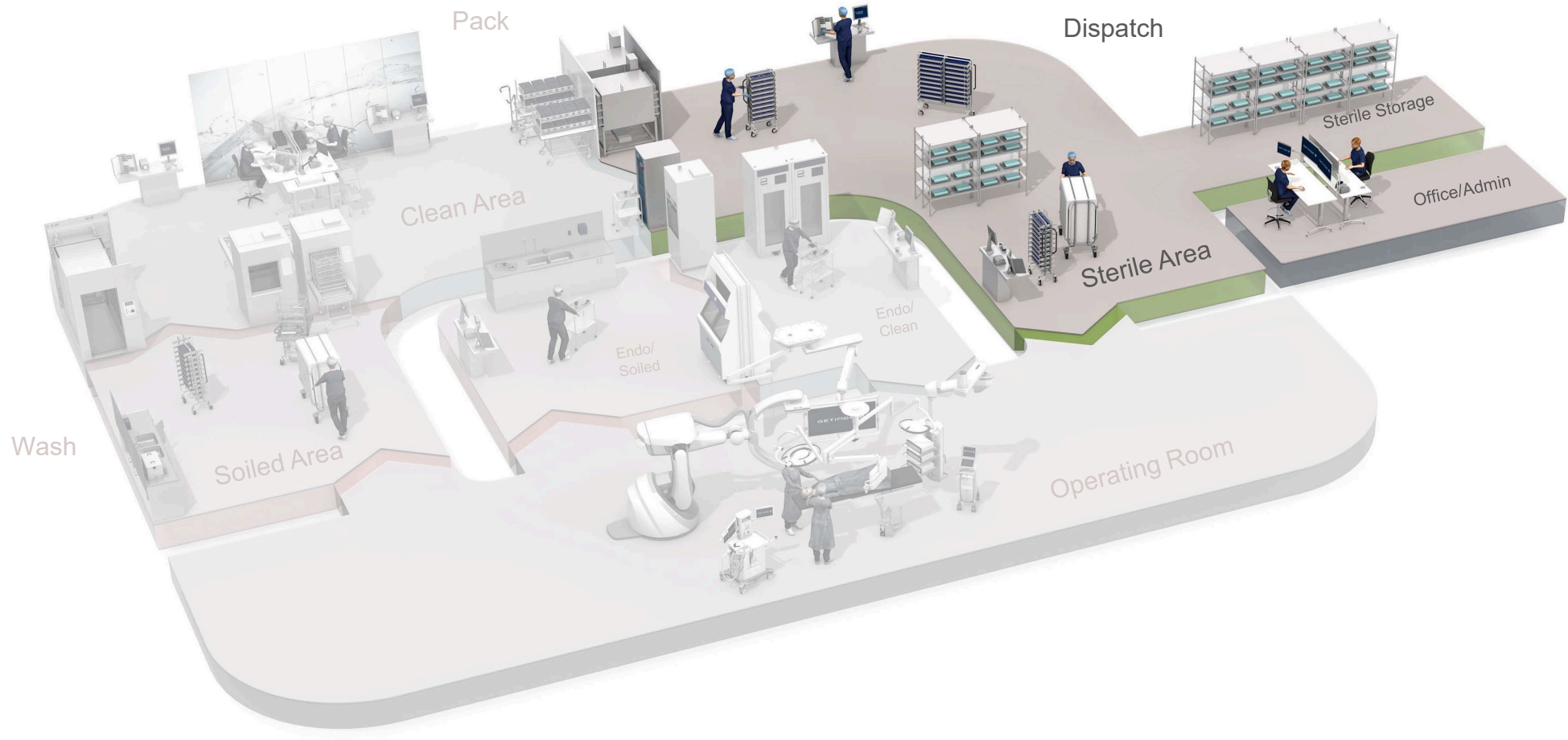


Sterile Area

Learn how T-DOC manages the sterile workflow in the Sterile Area

- Handling and approval of sterilizer batch
- Dispatch
- Order picking




Handle sterilizer batch

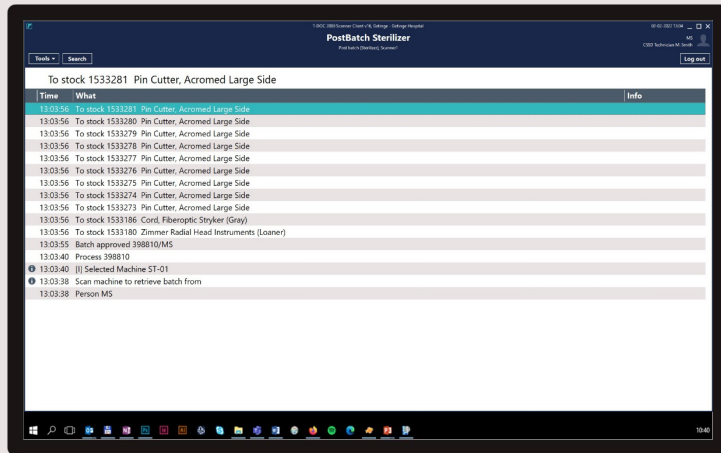


A barcode template assists the user in scanning the correct barcodes in the correct order


Approve a batch
Approve sterilizer batch

PostBatchArea 
scan barcode to enter area




Step 1
Scan your ID





Step 2
Scan barcode to use last batch


 Use last batch


Step 3
Scan the machine

  **ST-01**
PACS3000 Sterilizer 1 **ST-02**
PACS3000 Sterilizer 2

Step 4
Scan barcode to approve batch

 Approve batch


Log out 
scan barcode to log out

GETINGE 
T-DOC Help Desk



PostBatch Sterilizer

Post batch (Sterilizer), Scanner1

 Retrieve the latest processed batch from the machine

Log out

Tools ▾

Search

Process 398812

Time	What	Info
13:11:21	Process 398812	
i 13:11:21	[I] Selected Machine ST-01	
i 13:11:18	Scan machine to retrieve batch from	
13:11:13	Person MS	



PostBatch Sterilizer

Post batch (Sterilizer), Scanner1

MS

CSSD Technician M. Smith

Equipment process for visual approval

Log out

Tools

Search

Process 398812

Time	What	Info
------	------	------


13:11:21 Process 398812

13:11:21 [I] Selected Machine ST-01

13:11:18 Scan machine to retrieve batch from

13:11:13 Person MS

ST-01 Sterilizer progress

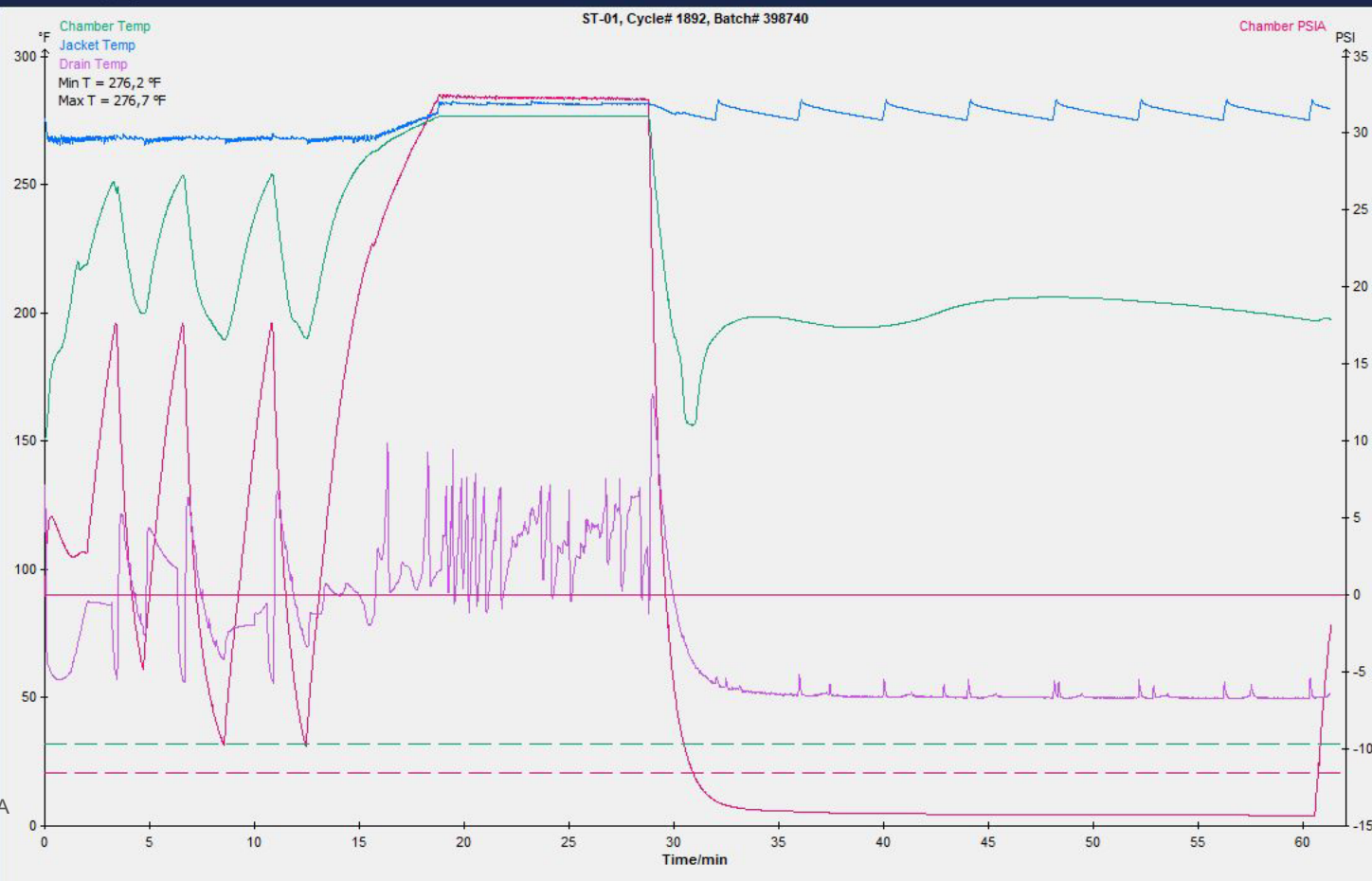

Time	PSI	°F
0:00:01	-0,074	150,6
0:00:01	0,316	150,7
0:00:02	0,906	150,9
0:00:03	1,536	151,3
0:00:04	2,136	151,9
0:00:05	2,656	152,9
0:00:06	3,116	154,1
0:00:07	3,516	155,7
0:00:08	3,846	157,4
0:00:09	4,106	159,4
0:00:10	4,306	161,5
0:00:11	4,466	163,6
0:00:12	4,626	165,6
0:00:13	4,776	167,5
0:00:14	4,876	169,2
0:00:15	4,936	170,7
0:00:16	4,976	172,0
0:00:17	5,016	173,4
0:00:18	5,056	174,7
0:00:19	5,076	175,8
0:00:20	5,066	176,9
0:00:21	5,046	177,8
0:00:22	5,026	178,5
0:00:23	5,006	179,2
0:00:24	4,986	179,9
0:00:25	4,946	180,5
0:00:26	4,886	180,9
0:00:27	4,816	181,4
0:00:28	4,776	181,7
0:00:29	4,736	182,0
0:00:30	4,686	182,3
0:00:31	4,616	182,7
0:00:32	4,546	183,0
0:00:33	4,496	183,2
0:00:34	4,456	183,4
0:00:35	4,406	183,7
0:00:36	4,346	183,9
0:00:37	4,276	184,1
0:00:38	4,206	184,3
0:00:39	4,166	184,5
0:00:40	4,116	184,6

DATE	PROCESS START	MACHINE NAME	CYCLE COUNTER
12/06/20	12:16:29	# 1	1892

PARAMETERS	Value	Unit
Exposure Temp	275.0	F
Exposure Time	00:10:00	
Drying Time	00:30:00	

SIGNALS	Value	Unit
AI05 Chamber PSIA	17.18	218.7
AI00 Chamber Temp	28.32	245.6

PROGRAM	AI05	AI00
00:00:00	14.40	150.6
00:00:00	14.41	150.6
00:01:00	17.59	191.0
00:02:00	17.19	218.7
00:02:01	17.18	218.7
00:03:00	28.32	245.6
00:03:24	32.09	248.9
00:04:00	17.34	220.2
00:04:43	9.67	199.8
00:04:43	9.65	199.8
00:05:00	14.06	207.7
00:06:00	27.04	243.3
00:06:35	32.08	253.4
00:07:00	19.83	228.2
00:08:00	7.32	195.8
00:08:34	4.72	189.4
00:08:34	4.71	189.3
00:09:00	11.17	199.6
00:10:00	23.88	236.6
00:10:50	32.08	253.6
00:11:00	27.72	247.1
00:12:00	8.12	196.3
00:12:28	4.69	190.1
00:12:28	4.65	190.1
00:13:00	12.98	205.6
00:14:00	25.68	241.1
00:15:00	34.05	257.2
00:16:00	38.36	264.2
00:17:00	41.81	269.5
00:18:00	44.67	273.5

Start date: 06-12-2020

Start time: 12:16:34

Program: P3 - vac 10/30/3

Internal program: 3

Cycle: 1892

Batch: 398740

Exp.Temp: []

Exp. Time: []

Log

Full Log

Native

2nd

Set Ref

Ref

Log(x)

Close


MCV00109020 REVA

PostBatch Sterilizer

Post batch (Sterilizer), Scanner1

MS

CSSD Technician M. Smith

 Approve batch and units change status

Log out


Tools ▾

Search

To stock 1533281 Pin Cutter, Acromed Large Side

Time	What	Info
13:03:56	To stock 1533281 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533280 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533279 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533278 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533277 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533276 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533275 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533274 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533273 Pin Cutter, Acromed Large Side	
13:03:56	To stock 1533186 Cord, Fiberoptic Stryker (Gray)	
13:03:56	To stock 1533180 Zimmer Radial Head Instruments (Loaner)	
13:03:55	Batch approved 398810/MS	
13:03:40	Process 398810	
i 13:03:40	[I] Selected Machine ST-01	
i 13:03:38	Scan machine to retrieve batch from	
13:03:38	Person MS	


Dispatch

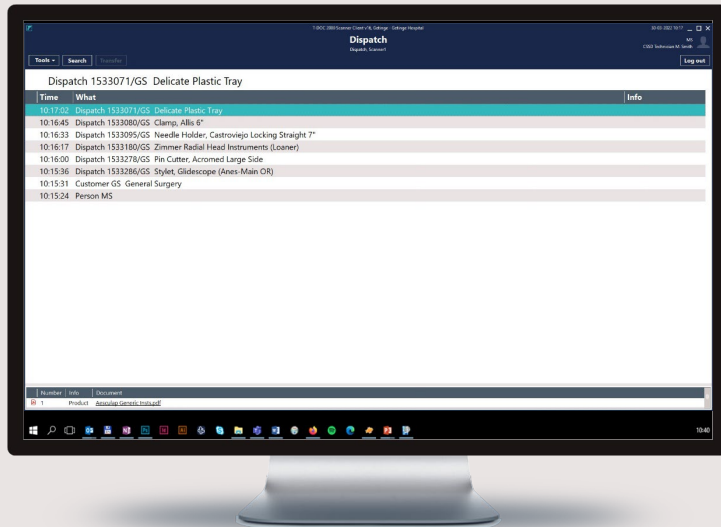


A barcode template assists the user in scanning the correct barcodes in the correct order

Dispatch

Deliver units to customers


Dispatch 
scan barcode to enter area


Step 1
Scan your ID




Step 2
Scan the customer barcode

Step 3
Scan the barcodes of all units being dispatched

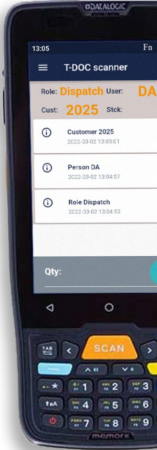

Log out 
scan barcode to log out

GETINGE 
T-DOC Help Desk



Dispatch

Dispatch, Scanner1



Scan customer and units to be dispatched

Tools ▾

Search

Transfer

Log out

Dispatch 1533071/GS Delicate Plastic Tray

Time	What	Info
10:17:02	Dispatch 1533071/GS Delicate Plastic Tray	
10:16:45	Dispatch 1533080/GS Clamp, Allis 6"	
10:16:33	Dispatch 1533095/GS Needle Holder, Castroviejo Locking Straight 7"	
10:16:17	Dispatch 1533180/GS Zimmer Radial Head Instruments (Loaner)	
10:16:00	Dispatch 1533278/GS Pin Cutter, Acromed Large Side	
10:15:36	Dispatch 1533286/GS Stylet, Glidescope (Anes-Main OR)	
10:15:31	Customer GS General Surgery	
10:15:24	Person MS	


MCV00109020 REVA



Member | Info | Document

Product [Aesculap Generic Insts.pdf](#)

Order Pick

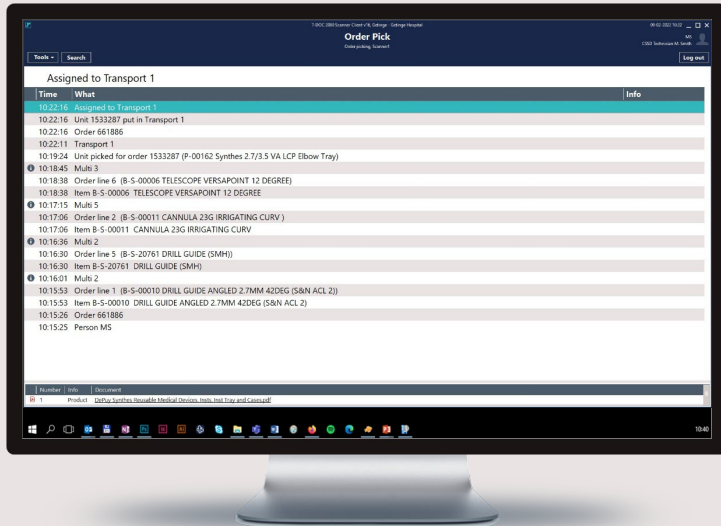


The printed pick list with 2D codes and quantity assists the user when picking the goods.

The printed delivery note lists the goods contents and is included with the goods to be delivered.

Delivery note

General Surgery

Details

Customer code GS	Customer General Surgery
Order no. 661894	Delivered by CSSD Technician M. Smith
Batch barcode 	Delivered 30-03-2022 10:27

Delivered from
Memorial Medical Center

Delivered to
General Surgery

Contents delivered

Article	Ordered
Stylet, Glidescope (Anes-Main OR)	1
Product no. Serial no. Unit no.	
P-03451	1533286
Pin Cutter, Acromed Large Side	1
Product no. Serial no. Unit no.	
03381	1533278
Zimmer Radial Head Instruments (Loaner)	1
Product no. Serial no. Unit no.	
P-57654	1533180
Needle Holder, Castroviejo Locking Straight 7"	1
Product no. Serial no. Unit no.	
4863	1533095
Clamp, Allis 6"	1
Product no. Serial no. Unit no.	
2158	1533080
Delicate Plastic Tray	1
Product no. Serial no. Unit no.	
4891	4891-08 1533071

GETINGE *

Pick list


GS General Surgery

Details

Order no. 661888	Customer code GS General Surgery	Delivered by System Administrator
Order barcode 	Customer GS General Surgery	Delivered 09-02-2022 12:42

Delivered from
Memorial Medical Center


Delivered to
GS General Surgery

Item	Name	Ord.	Unit measure
B-10016	CANNULA, LIPOSUCTION MERCEDES BLUE 	10	
B-22213	ARCH BAR, RUBBER BANDS 	10	
B-3005	BONE TAMP 6MM 	15	
B-3007	BONE TAMP 7MM 	25	

Product	Name	Ord.
03643	Extractor Pliers	2
03686	Tissue Forcep, Dura w/ Teeth (7BOB)	5
03688	Retractor, Serris (Cat Paws) Sharp (7BOB)	3
03690	Knife Handle, #3 (ER)	12
03773	Medtronic Metrx Clamp	20

GETINGE *

Page: 1 of 1

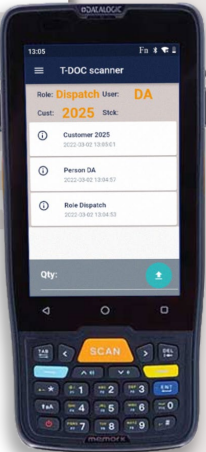



Scan the location, user, order to be picked,
and all items

Order Pick

Order picking, Scanner1

Log out

Unit picked for order 1533287 (P-00162 Synthes 2.7/3.5 VA LCP Elbow Tray)


Time	What	Info
10:19:24	Unit picked for order 1533287 (P-00162 Synthes 2.7/3.5 VA LCP Elbow Tray)	
i 10:18:45	Multi 3	
10:18:38	Order line 6 (B-S-00006 TELESCOPE VERSAPOINT 12 DEGREE)	
10:18:38	Item B-S-00006 TELESCOPE VERSAPOINT 12 DEGREE	
i 10:17:15	Multi 5	
10:17:06	Order line 2 (B-S-00011 CANNULA 23G IRRIGATING CURV)	
10:17:06	Item B-S-00011 CANNULA 23G IRRIGATING CURV	
i 10:16:36	Multi 2	
10:16:30	Order line 5 (B-S-20761 DRILL GUIDE (SMH))	
10:16:30	Item B-S-20761 DRILL GUIDE (SMH)	
i 10:16:01	Multi 2	
10:15:53	Order line 1 (B-S-00010 DRILL GUIDE ANGLED 2.7MM 42DEG (S&N ACL 2))	
10:15:53	Item B-S-00010 DRILL GUIDE ANGLED 2.7MM 42DEG (S&N ACL 2)	
10:15:26	Order 661886	
10:15:25	Person MS	

MCV00109020 REVA



Member | Info | Document

Product [DePuy Synthes Reusable Medical Devices, Insts, Inst Tray and Cases.pdf](#)

 All scanned items and units are assigned to trolley

Order Pick

Order picking, Scanner1

Log out


Assigned to Transport 1

Time	What	Info
10:22:16	Assigned to Transport 1	
10:22:16	Unit 1533287 put in Transport 1	
10:22:16	Order 661886	
10:22:11	Transport 1	
10:19:24	Unit picked for order 1533287 (P-00162 Synthes 2.7/3.5 VA LCP Elbow Tray)	
i 10:18:45	Multi 3	
10:18:38	Order line 6 (B-S-00006 TELESCOPE VERSAPOINT 12 DEGREE)	
10:18:38	Item B-S-00006 TELESCOPE VERSAPOINT 12 DEGREE	
i 10:17:15	Multi 5	
10:17:06	Order line 2 (B-S-00011 CANNULA 23G IRRIGATING CURV)	
10:17:06	Item B-S-00011 CANNULA 23G IRRIGATING CURV	
i 10:16:36	Multi 2	
10:16:30	Order line 5 (B-S-20761 DRILL GUIDE (SMH))	
10:16:30	Item B-S-20761 DRILL GUIDE (SMH)	
i 10:16:01	Multi 2	
10:15:53	Order line 1 (B-S-00010 DRILL GUIDE ANGLED 2.7MM 42DEG (S&N ACL 2))	
10:15:53	Item B-S-00010 DRILL GUIDE ANGLED 2.7MM 42DEG (S&N ACL 2)	
10:15:26	Order 661886	
10:15:25	Person MS	

MCV00109020 REVA

Order Pick

Order picking, Scanner1

 Order is delivered

Log out


Order delivered 661886

Time	What	Info
10:23:52	Order delivered 661886	
10:23:52	Dispatch 1533287/ORTHO Synthes 2.7/3.5 VA LCP Elbow Tray	
10:22:16	Assigned to Transport 1	
10:22:16	Unit 1533287 put in Transport 1	
10:22:16	Order 661886	
10:22:11	Transport 1	
10:19:24	Unit picked for order 1533287 (P-00162 Synthes 2.7/3.5 VA LCP Elbow Tray)	
i 10:18:45	Multi 3	
10:18:38	Order line 6 (B-S-00006 TELESCOPE VERSAPOINT 12 DEGREE)	
10:18:38	Item B-S-00006 TELESCOPE VERSAPOINT 12 DEGREE	
i 10:17:15	Multi 5	
10:17:06	Order line 2 (B-S-00011 CANNULA 23G IRRIGATING CURV)	
10:17:06	Item B-S-00011 CANNULA 23G IRRIGATING CURV	
i 10:16:36	Multi 2	
10:16:30	Order line 5 (B-S-20761 DRILL GUIDE (SMH))	
10:16:30	Item B-S-20761 DRILL GUIDE (SMH)	
i 10:16:01	Multi 2	
10:15:53	Order line 1 (B-S-00010 DRILL GUIDE ANGLED 2.7MM 42DEG (S&N ACL 2))	
10:15:53	Item B-S-00010 DRILL GUIDE ANGLED 2.7MM 42DEG (S&N ACL 2)	
10:15:26	Order 661886	
10:15:25	Person MS	

MCV00109020 REVA

Surgical Department


Learn how T-DOC manages the sterile workflow in the Surgical Department

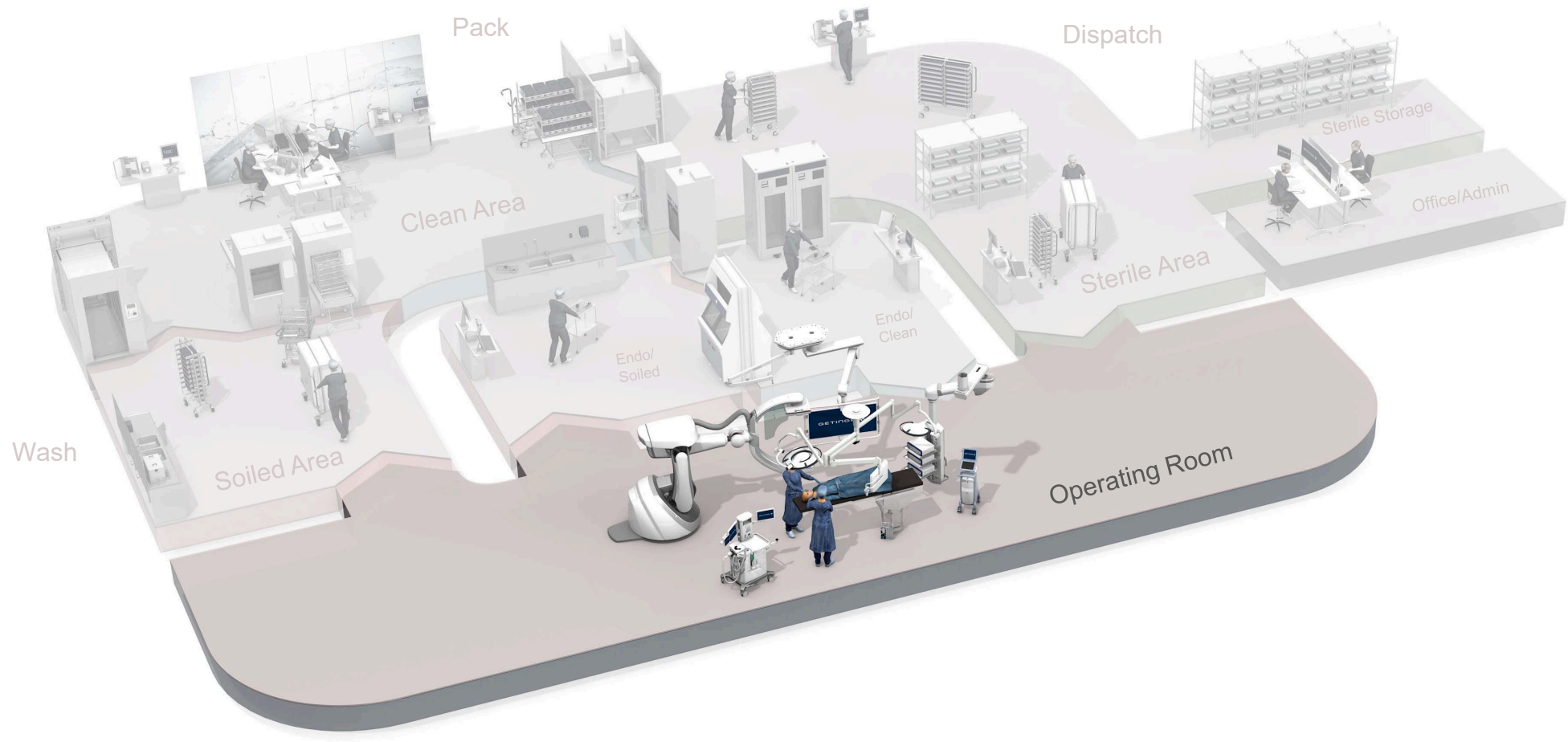


Surgical Department

Learn how T-DOC manages the sterile workflow in the Surgical Department

- Patient registration
- Instrument count
- Registration of count changes
- How to place an instrument order

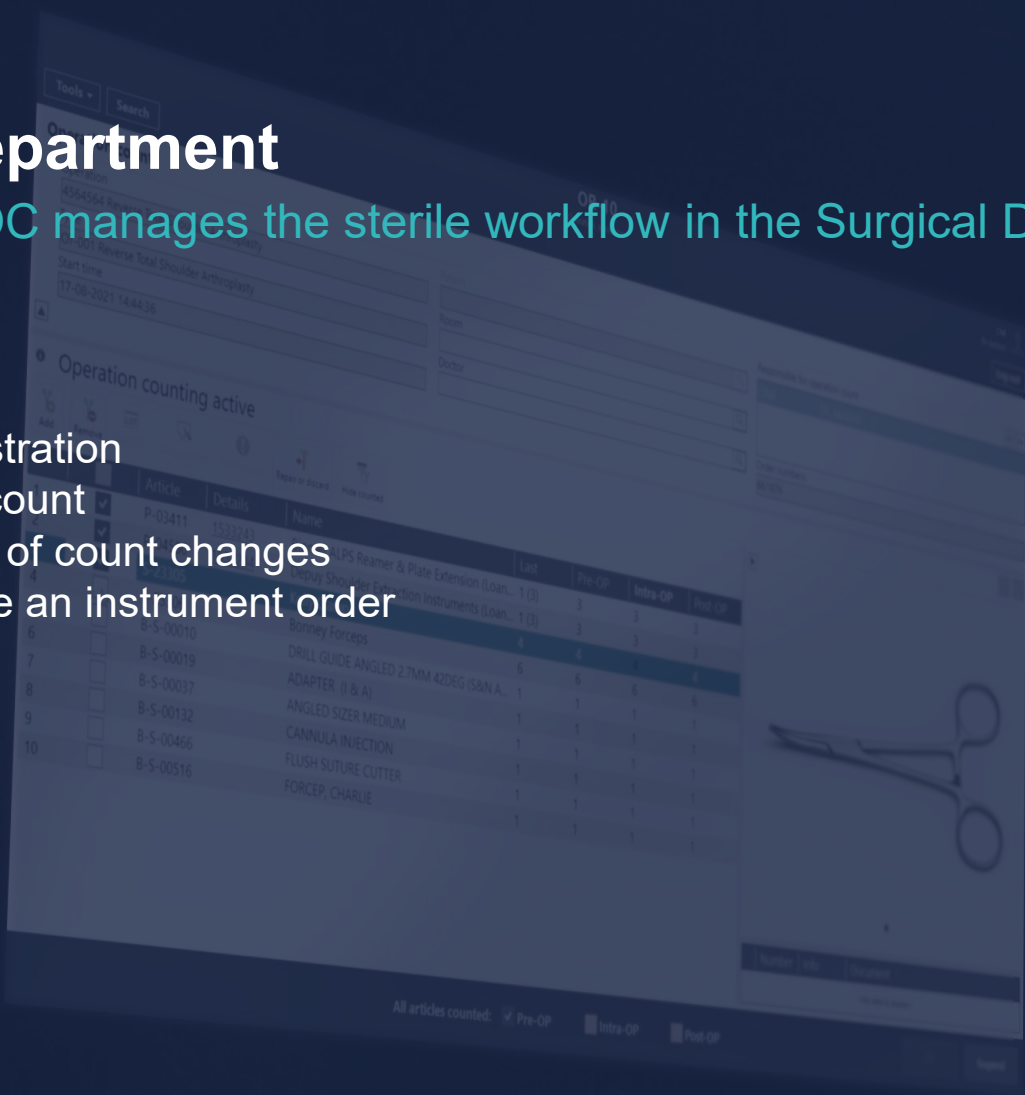


Operation

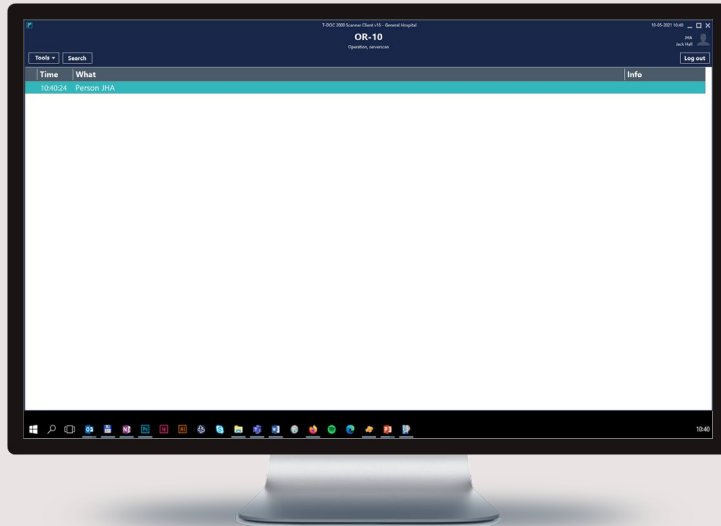


A barcode template assists the user in scanning the correct barcodes in the correct order

Assign to patient

Assign units to patient

Operation Location 
scan barcode to enter area

Step 1
Scan your ID




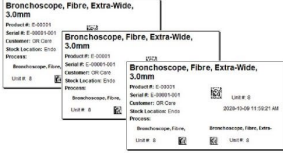
Step 2
Scan barcode to register patient


 Patient ID


Step 3
Scan the patient barcode



Step 4
Scan the barcodes of all units being assigned to the patient



Log out 
scan barcode to log out

GETINGE 
T-DOC Help Desk




OR-10

Operation, serverscan

JHA
Jack Hall

Log out

 Scan user

Tools ▾


Search

Time

What

Info

10:40:24 Person JHA

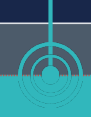


OR-10

Operation, serverscan

JHA
Jack Hall

Log out

 Scan operation ID and operation count window opens

Tools ▾

Search

Time	What	Info
10:42:56	Operation 123456 Reverse Total Shoulder Arthroplasty	
10:40:24	Person JHA	



OR-10

Operation, serverscan

Basic surgery data

Expand to administer lot numbers, comments, triggers and repairs

For patient safety, articles can be counted before, during, and after surgery

Tools

Search

Operation count

Operation

123456 Reverse Total Shoulder Arthroplasty

Type

OT-001 Reverse Total Shoulder Arthroplasty

Start time

10-05-2021 10:42:51

Patient

Room

OR-10

Doctor

Jane Smith

Responsible for operation count

Order numbers

661859

Order items bound to operation 1 / 1

- Add
- Remove
- Edit LOT
- Add comment
- Notifications
- Repair or discard
- Hide counted

		Article	Details	Name	Last	Pre-OP	Intra-OP	Post-OP
1	<input type="checkbox"/>	4405	1533204	Chisel Set, Small	1 (6)	6	6	6
2	<input type="checkbox"/>	B61112	1533205	TRAY LARGE ORTHO	1 (41)	41	41	41
3	<input type="checkbox"/>	C-P-57332	1533206	HALL SAW KEY (CVOR)	1	1	1	1
4	<input type="checkbox"/>	P-02480	1533207	Biomet Vanguard SSK Reamer & Stem Trials (Loaner)	1 (3)	3	3	3
5	<input type="checkbox"/>	P-02520	1533208	Depuy Anchor Peg Glenoid Tray (APG+) (Loaner)	1 (3)	3	3	3
6	<input type="checkbox"/>	P-03411	1533209	Biomet ALPS Reamer & Plate Extension (Loaner)	1 (3)	3	3	3
7	<input type="checkbox"/>	P-04580	1533210	Depuy Shoulder Extraction Instruments (Loaner)	1 (3)	3	3	3
8	<input type="checkbox"/>	S-18876		Biomet Primary Total Shoulder	1	1	1	1



Supportive media and documents

Counting status

Number	Info	Document
1	Product	Zimmer Orthopedic Insts.pdf

MCV00109020 REVA

OR-10

Operation, serverscan

JHA
Jack Hall

Log out

Scan patient ID and link patient with articles, surgery, and doctor

Tools Search

Operation count

Operation
123456 Reverse Total Shoulder Arthroplasty

Type
OT-001 Reverse Total Shoulder Arthroplasty

Start time
10-05-2021 10:42:51

Patient
123456-1234 John Doe

Room
OR-10

Doctor
Jane Smith


Responsible for operation count

Order numbers
661859

Operation counting active

		Article	Details	Name	Last	Pre-OP	Intra-OP	Post-OP
1	<input type="checkbox"/>	4405	1533204	Chisel Set, Small	1 (6)	6	6	6
2	<input type="checkbox"/>	B61112	1533205	TRAY LARGE ORTHO	1 (41)	41	41	41
3	<input type="checkbox"/>	C-P-57332	1533206	HALL SAW KEY (CVOR)	1	1	1	1
4	<input type="checkbox"/>	P-02480	1533207	Biomet Vanguard SSK Reamer & Stem Trials (Loaner)	1 (3)	3	3	3
5	<input type="checkbox"/>	P-02520	1533208	Depuy Anchor Peg Glenoid Tray (APG+) (Loaner)	1 (3)	3	3	3
6	<input type="checkbox"/>	P-03411	1533209	Biomet ALPS Reamer & Plate Extension (Loaner)	1 (3)	3	3	3
7	<input type="checkbox"/>	P-04580	1533210	Depuy Shoulder Extraction Instruments (Loaner)	1 (3)	3	3	3
8	<input type="checkbox"/>	S-18876		Biomet Primary Total Shoulder	1	1	1	1

Click on each tray to open the trays content list and count instruments in tray



Number	Info	Document
1	Product	Zimmer.Orthopedic Instns.pdf

MCV00109020 REVA



OR-10

Operation, serverscan

Content list for the selected tray opens.
Count and check-off instruments in the tray.

Tools ▾

Search

Log out

Content list

Product

B61112 TRAY LARGE ORTHO

Product serial


B61112-01

Unit

1533205

- LOT
- Edit LOT
- Add comment
- Repair or discard
- Hide counted

		Item	Name	Last	Pre-OP	Intra-OP	Post-OP
1	<input checked="" type="checkbox"/>	S-06406	Mallet, Large	1	1	1	1
2	<input checked="" type="checkbox"/>	S-00297	Curette, Straight 1	1	1	1	1
3	<input checked="" type="checkbox"/>	S-01145	Curette, Straight 2	1	1	1	1
4	<input checked="" type="checkbox"/>	S-00153	Curette, Straight 4	1	1	1	1
5	<input checked="" type="checkbox"/>	S-00250	Curette, Straight 0	1	1	1	1
6	<input checked="" type="checkbox"/>	S-09040	Elevator, Cobb Large	1	1	1	1
7	<input checked="" type="checkbox"/>	S-13099	Elevator, Cobb Small	1	1	1	1
8	<input checked="" type="checkbox"/>	S-07814	Retractor, Hohmann Dull	2	2	2	2
9	<input type="checkbox"/>	S-08459	Retractor, Hayes	2	2	2	2
10	<input type="checkbox"/>	S-05149	Retractor, Hohmann 6mm	1	1	1	1
11	<input type="checkbox"/>	S-05561	Retractor, Hohmann 8mm	1	1	1	1
12	<input type="checkbox"/>	S-08174	Elevator, Freer	1	1	1	1
13	<input type="checkbox"/>	S-17573	Pin Cutter	1	1	1	1
14	<input type="checkbox"/>	S-03547	Osteotome, Straight Hibbs 3/4" (square handle)	1	1	1	1
15	<input type="checkbox"/>	S-03540	Osteotome, Curved Hibbs 3/4" (square handle)	1	1	1	1
16	<input type="checkbox"/>	S-00565	Osteotome, Straight Hibbs 1/2" (square handle)	1	1	1	1
17	<input type="checkbox"/>	S-00566	Osteotome, Curved Hibbs 1/2" (square handle)	1	1	1	1
18	<input type="checkbox"/>	S-00636	Osteotome, Straight Hibbs 1/4" (square handle)	1	1	1	1
19	<input type="checkbox"/>	S-00620	Osteotome, Curved Hibbs 1/4" (square handle)	1	1	1	1
20	<input type="checkbox"/>	S-13294	Rongeur, Snub Nose	1	1	1	1
21	<input type="checkbox"/>	S-11504	Rongeur, Regular Curved	1	1	1	1
22	<input type="checkbox"/>	MCV500020 Rev A	Retractor, Geloi Small	2	2	2	2



Number	Info	Document
1	Product	Depuy Mitek Cleaning and Sterilization.pdf
2	Product	Aesculap Generic Insts.pdf



OK

Cancel

OR-10

Operation, serverscan

Count change:
Choose amount, reason, and make a comment

Tools ▾

Search

Log out

Content list

Product

B61112 TRAY LARGE ORTHO

Product serial


B61112-01

Unit

1533205

- LOT
- Edit LOT
- Add comment
- Repair or discard
- Hide counted

		Item	Name	Last	Pre-OP	Intra-OP	Post-OP
1	<input checked="" type="checkbox"/>	S-06406	Mallet, Large	1	1	1	1
2	<input checked="" type="checkbox"/>	S-00297	Curette, Straight 1	1	1	1	1
3	<input checked="" type="checkbox"/>	S-01145	Curette, Straight 2	1	1	1	1
4	<input checked="" type="checkbox"/>	S-00153	Curette, Straight 4	1	1	1	1
5	<input checked="" type="checkbox"/>	S-00250	Curette, Straight 0	1	1	1	1
6	<input checked="" type="checkbox"/>	S-09040	Elevator, Cobb Large	1	1	1	1
7	<input checked="" type="checkbox"/>	S-13099	Elevator, Cobb Small	1	1	1	1
8	<input checked="" type="checkbox"/>	S-07814	Retractor, Hohmann Dull	2	2	2	2
9	<input type="checkbox"/>	S-08459	Retractor, Hayes	2	1	2	2
10	<input type="checkbox"/>	S-05149	Retractor, Hohmann 6mm	1	1	1	1
11	<input type="checkbox"/>	S-05561	Retractor, Hohmann 8mm				
12	<input type="checkbox"/>	S-08174	Elevator, Freer				
13	<input type="checkbox"/>	S-17573	Pin Cutter				
14	<input type="checkbox"/>	S-03547	Osteotome, Straight Hibbs 3/4" (square handle)				
15	<input type="checkbox"/>	S-03540	Osteotome, Curved Hibbs 3/4" (square handle)				
16	<input type="checkbox"/>	S-00565	Osteotome, Straight Hibbs 1/2" (square handle)				
17	<input type="checkbox"/>	S-00566	Osteotome, Curved Hibbs 1/2" (square handle)				
18	<input type="checkbox"/>	S-00636	Osteotome, Straight Hibbs 1/4" (square handle)				
19	<input type="checkbox"/>	S-00620	Osteotome, Curved Hibbs 1/4" (square handle)				
20	<input type="checkbox"/>	S-13294	Rongeur, Snub Nose				
21	<input type="checkbox"/>	S-11504	Rongeur, Regular Curved				
22	<input type="checkbox"/>	MCV500020 Retractor, Gelpi Small					



Number | Info | Document

1	Product	Depuy Mitek Cleaning and Sterilization.pdf
2	Product	Aesculap Generic Insts.pdf

Count change

Article
S-08459, Retractor, Hayes

Difference
-1

Reason *
Delivered with missing instruments ▾

Comment

OK Cancel

OK Cancel



OR-10

Operation, serverscan

Log out

Tools

Search

Content list

Product

B61112 TRAY LARGE ORTHO

Product serial

B61112-01

Unit

1533205

- LOT
- Edit LOT
- Add comment
- Repair or discard
- Hide counted

		Item	Name	Last	Pre-OP	Intra-OP	Post-OP
1	✓	S-06406	Mallet, Large	1	1	1	1
2	✓	S-00297	Curette, Straight 1	1	1	1	1
3	✓	S-01145	Curette, Straight 2	1	1	1	1
4	✓	S-00153	Curette, Straight 4	1	1	1	1
5	✓	S-00250	Curette, Straight 0	1	1	1	1
6	✓	S-09040	Elevator, Cobb Large	1	1	1	1
7	✓	S-13099	Elevator, Cobb Small	1	1	1	1
8	✓	S-07814	Retractor, Hohmann Dull	2	2	2	2
9	✓	S-13099	Elevator, Cobb Small	2	1	2	2
10	✓	S-05149	Retractor, Hohmann 6mm	1	1	1	1
11	✓	S-05561	Retractor, Hohmann 8mm	1	1	1	1
12	✓	S-08174	Elevator, Freer	1	1	1	1
13	✓	S-17573	Pin Cutter	1	1	1	1
14	✓	S-03547	Osteotome, Straight Hibbs 3/4" (square handle)	1	1	1	1
15	✓	S-03540	Osteotome, Curved Hibbs 3/4" (square handle)	1	1	1	1
16	✓	S-00565	Osteotome, Straight Hibbs 1/2" (square handle)	1	1	1	1
17	✓	S-00566	Osteotome, Curved Hibbs 1/2" (square handle)	1	1	1	1
18	✓	S-00636	Osteotome, Straight Hibbs 1/4" (square handle)	1	1	1	1
19	✓	S-00620	Osteotome, Curved Hibbs 1/4" (square handle)	1	1	1	1
20	✓	S-13294	Rongeur, Snub Nose	1	1	1	1
21	✓	S-11504	Rongeur, Regular Curved	1	1	1	1
22	✓	MCV900020	Retractor, Geloi Small	2	2	2	2



When all tray items are counted, select OK and the content list closes.

Number	Info	Document
1	Product	Depuy Mitek Cleaning and Sterilization.pdf
2	Product	Aesculap Generic Insts.pdf

OK Cancel



On the Operation count list, a notification now indicates changes in instrument count.

OR-10

Operation, serverscan

Tools Search

Operation count

Operation
123456 Reverse Total Shoulder Arthroplasty

Type
OT-001 Reverse Total Shoulder Arthroplasty

Start time
10-05-2021 10:42:51

Patient
123456-1234 John Doe

Room
OR-10

Doctor
Jane Smith

Responsible for operation count
JHA Jack Hall

Order numbers
661859

Operation counting active

	<input checked="" type="checkbox"/>	Article	Details	Name	Last	Pre-OP	Intra-OP	Post-OP
1	<input checked="" type="checkbox"/>	4405	1533204	Chisel Set, Small	1 (6)	6	6	6
2	<input checked="" type="checkbox"/>	B61112	1533205	TRAY LARGE ORTHO	1 (40) i	40	40	40
3	<input checked="" type="checkbox"/>	C-P-57332	1533206	HALL SAW KEY (CVOR)	1	1	1	1
4	<input checked="" type="checkbox"/>	P-02480	1533207	Biomet Vanguard SSK Reamer & Stem Trials (Loaner)	1 (3)	3	3	3
5	<input checked="" type="checkbox"/>	P-02520	1533208	Depuy Anchor Peg Glenoid Tray (APG+) (Loaner)	1 (3)	3	3	3
6	<input checked="" type="checkbox"/>	P-03411	1533209	Biomet ALPS Reamer & Plate Extension (Loaner)	1 (3)	3	3	3
7	<input checked="" type="checkbox"/>	P-04580	1533210	Depuy Shoulder Extraction Instruments (Loaner)	1 (3)	3	3	3
8	<input checked="" type="checkbox"/>	S-18876		Biomet Primary Total Shoulder	1	1	1	1



MCV00109020 REVA



OR-10

Operation, serverscan

Log out

Tools Search

Reason for count change is displayed when notification icon is selected

Operation count

Operation
123456 Reverse Total Shoulder Arthroplasty

Type
OT-001 Reverse Total Shoulder Arthroplasty

Start time
10-05-2021 10:42:51

Patient
123456-1234 John Doe

Room
OR-10

Doctor
Jane Smith

Responsible for operation count
JHA Jack Hall

Order numbers
661859

Operation counting active

	<input checked="" type="checkbox"/>	Article
1	<input checked="" type="checkbox"/>	4405
2	<input checked="" type="checkbox"/>	B61112
3	<input checked="" type="checkbox"/>	C-P-57332
4	<input checked="" type="checkbox"/>	P-02480
5	<input checked="" type="checkbox"/>	P-02520
6	<input checked="" type="checkbox"/>	P-03411
7	<input checked="" type="checkbox"/>	P-04580
8	<input checked="" type="checkbox"/>	S-18876

Count changes for unit 1533205, B61112

	Count	Diff.	Item	Name	Reason	Comment	User
1	Pre-OP	-1	S-08459	Retractor, Hayes	Delivered with missing instruments		JHA

Number | Info | Document

<No data to display>



OR-10

Operation, serverscan

Tools Search

Operation count

Operation
123456 Reverse Total Shoulder Arthroplasty

Type
OT-001 Reverse Total Shoulder Arthroplasty

Start time
10-05-2021 10:42:51

Patient
123456-1234 John Doe

Room
OR-10

Doctor
Jane Smith

Responsible for operation count
JHA Jack Hall

Order numbers
661859

Operation counting active

	<input checked="" type="checkbox"/>	Article	Details	Name	Last	Pre-OP	Intra-OP	Post-OP
1	<input checked="" type="checkbox"/>	4405	1533204	Chisel Set, Small	1 (6)	6	6	6
2	<input checked="" type="checkbox"/>	B61112	1533205	TRAY LARGE ORTHO	1 (40)	40	40	40
3	<input checked="" type="checkbox"/>	C-P-57332	1533206	HALL SAW KEY (CVOR)	1	1	1	1
4	<input checked="" type="checkbox"/>	P-02480	1533207	Biomet Vanguard SSK Reamer & Stem Trials (Loaner)	1 (3)	3	3	3
5	<input checked="" type="checkbox"/>	P-02520	1533208	Depuy Anchor Peg Glenoid Tray (APG+) (Loaner)	1 (3)	3	3	3
6	<input checked="" type="checkbox"/>	P-03411	1533209	Biomet ALPS Reamer & Plate Extension (Loaner)	1 (3)	3	3	3
7	<input checked="" type="checkbox"/>	P-04580	1533210	Depuy Shoulder Extraction Instruments (Loaner)	1 (3)	3	3	3
8	<input checked="" type="checkbox"/>	S-18876		Biomet Primary Total Shoulder	1	1	1	1

Operation count is completed by selecting "OK"

Number | Info | Document


<No data to display>

Counting of all items in all trays is completed

When operation count is complete, all status boxes are marked

MCV00109020 REVA

Create order

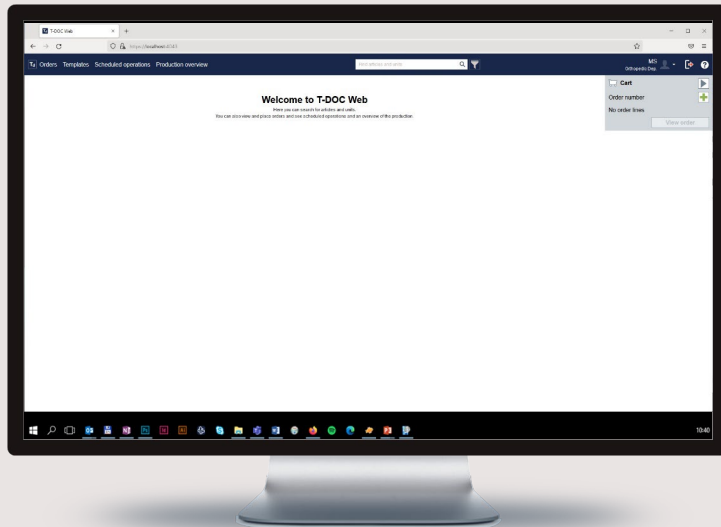


The T-DOC Web client can easily be accessed from a browser and is often used outside the CSSD.

In T-DOC Web, you can:

- Search for units, serial numbers, products and items, and find their location
- Place and amend orders
- Use standard order templates



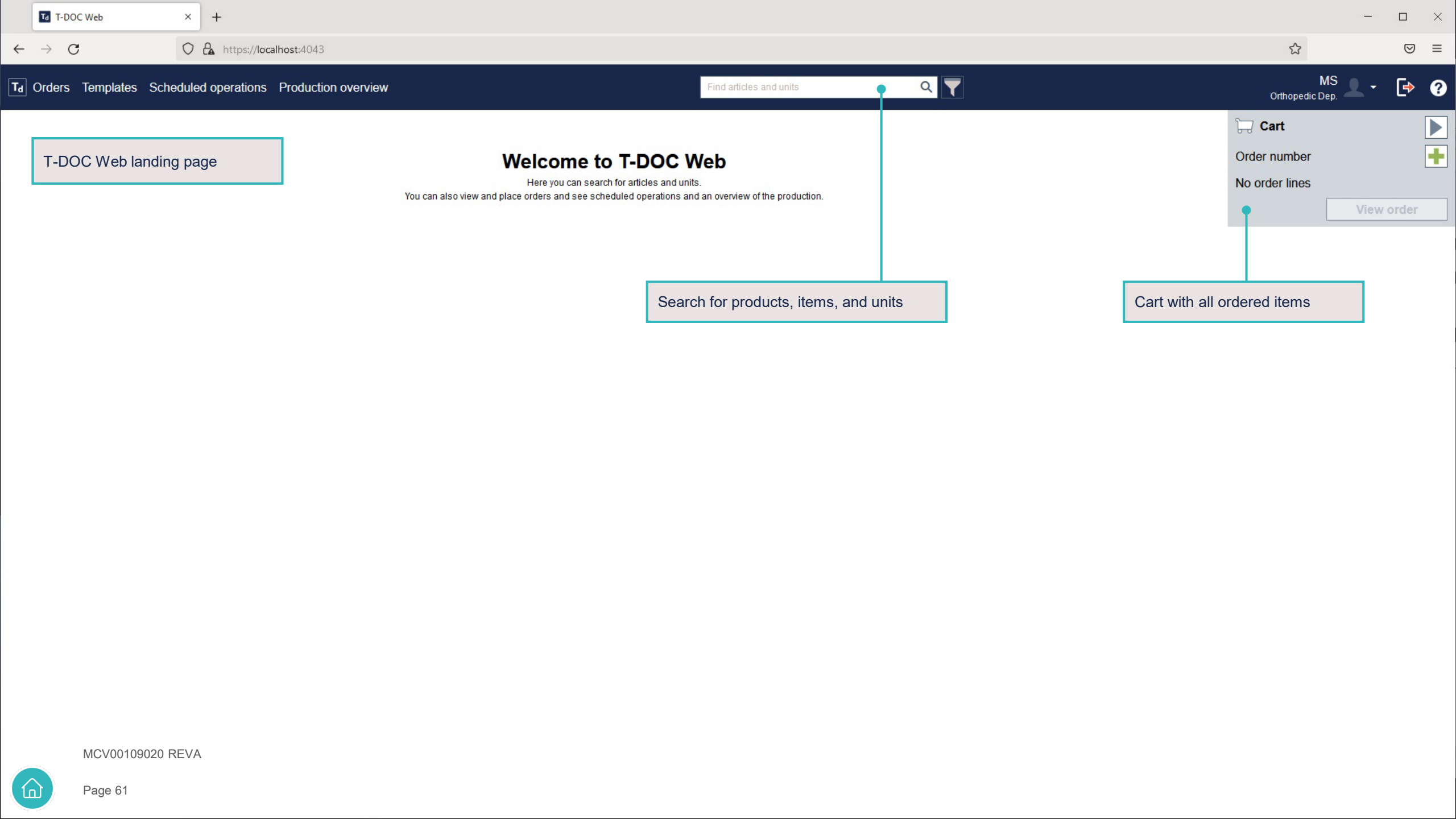

T-DOC Web landing page

Search for products, items, and units

Cart with all ordered items



Search

Search for specific products, items, and units

Search for articles and units.

basic [Search] [Filter]

Hover over icon to view an image

Add products and items to the cart

- Products (25)
- Items (17)
- Endoscopes (0)
- Units (0)

Media	Product	Item.Name	Item.Manuf. No	Item.Specialty.Name	Quantity	Add
	1743451	Basic Lacrimal Eye Tray	4516589-7	Eye	1	
	4147	Basic Total Hip	564321-98	Ortho	1	
	4282	Zimmer Basic Instruments		Ortho	1	
		Zimmer Basic Plate Set			1	
		Basic Tray		General	1	
		Synthes Basic Instruments		Ortho	1	
	6124	Basic Bone 1		Ortho	1	
	6125	Basic Bone 2		Ortho	1	
	6924	Synthes Basic Screw Rack	56451236656	Ortho	1	
	B29249	TRAY BASIC		General	1	
	C4804750	TRAY BASIC (CVOR)			1	
	C6652877	BASIC BUNDLE (CVOR)			1	
	P-00186	Synthes DHS-DCS Basic Instruments			1	
	P-00192	Synthes LCP DHHS Basic Instruments		Ortho	1	
	P-02935	Stryker Accolade II Basic Insts (Loaner)			1	
	P-03143	Zimmer Basic Fragment Plates (Loaner)			1	
	P-03347	Zimmer Basic Screw Rack (Loaner)			1	
	P-03349	Zimmer Basic Insts (Loaner)			1	
	P-03388	Stryker Matta Pelvic Basic Insts (Loaner)			1	
	P-03391	Stryker Matta Basic Insts (Loaner)			1	



Cart

Order number

No order lines

[View order](#)



Search

Search for articles and units.

basic

- Products (25)
- Items (17)
- Endoscopes (0)
- Units (0)

Media	Product	Item.Name	Item.Manuf. No	Item.Specialty.Name	Quantity	Add
	1743451	Basic Lacrimal Eye Tray	4516589-7	Eye	1	
	4147	Basic Total Hip	564321-98	Ortho	1	
	4282	Zimmer Basic Instruments		Ortho	1	
		Zimmer Basic Plate Set			1	
		Basic Tray		General	1	
		Synthes Basic Instruments		Ortho	1	
	6124	Basic Bone 1		Ortho	1	
	6125	Basic Bone 2		Ortho	1	
	6924	Synthes Basic Screw Rack	56451236656	Ortho	1	
	B29249	TRAY BASIC		General	1	
	C4804750	TRAY BASIC (CVOR)			1	
	C6652877	BASIC BUNDLE (CVOR)			1	
	P-00186	Synthes DHS-DCS Basic Instruments			1	
	P-00192	Synthes LCP DHHS Basic Instruments		Ortho	1	
	P-02935	Stryker Accolade II Basic Insts (Loaner)			1	
	P-03143	Zimmer Basic Fragment Plates (Loaner)			1	
	P-03347	Zimmer Basic Screw Rack (Loaner)			1	
	P-03349	Zimmer Basic Insts (Loaner)			1	
	P-03388	Stryker Matta Pelvic Basic Insts (Loaner)			1	
	P-03391	Stryker Matta Basic Insts (Loaner)			1	

To view product details, select the product

Products have now been added to the cart

Cart

Order number: 12


Article	Qty.
51698 Total Knee Bundle	1
S-23308 Procedure Pack Hip	5
B-P-03827 AIMER ELBOW (S&N	2
4284 Zimmer Basic Plate Set	1

View order

MCV00109020 REVA

Product details

Show physical location of a product and minimize time spent searching

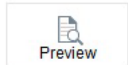

Product
1743451 - Basic Lacrimal Eye Tray

Local name

Customer
ELL OR ELL OR

Price
0,00

Stock placement
R6N1

Reservation mode
Not required



Add to cart
1

Item list	Serial numbers	Locate product	Documents	Remark						
Ready for use (1)	Packed (0)	Returned (0)	Used (0)	Repaired (0)						
Fast track	Unit	Serial no..Serial no.	Location.Name	Location time	Status	Customer.Customer	Customer.Name	Stock.Stock	Stock.Name	Tag
	1515428	1743451-01	R6N1	2020-11-01 11:31:55	40	ELL OR	ELL OR	Product Stock	Product Stock Memorial	

Cart

Order number: 12

Article	Qty.
51698 Total Knee Bundle	1
S-23308 Procedure Pack Hip	5
B-P-03827 AIMER ELBOW (S&N)	2
4284 Zimmer Basic Plate Set	1

View order

Select "View order" to see the cart details



Order details

Order details are displayed, including cart content, requested delivery date, and the department placing the order

Customer ref.no. 12

Status

Open

Select "Send" to send to the CSSD

Requested delivery date

12/02/2022 10.00

Booking mode

Active

Ordered by

ORTHO - Orthopedic Dep.

Reference text

Booking status

Undetermined

Ordered from

Memorial Medical Center - Memorial Medical Center

Reference text 2

Add article Product Item Endoscope

Please enter search text


Media	Article	Number	Name	Price	Quantity	Sum	Status
Product		51698	Total Knee Bundle	0,00	1	1	?
Item		S-23308	Procedure Pack Hip	0,00	5	5	?
Product		B-P-03827	AIMER ELBOW (S&N ACL 1)	0,00	2	2	?
Product		4284	Zimmer Basic Plate Set	0,00	1	1	?

Page 1 of 1 (4 results) 1

Delete selected

MCV00109020 REVA

Order template details

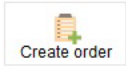

Template

001

From customer:

ORTHO - Orthopedic Dep.

Org. stock

Own reference text

Own reference text 2

Name

Replenish OR cabinet

To factory:

Memorial Medical Center - Memorial Medical Center

Supp. stock

Factory reference text

Factory reference text 2

Orders can also be created by using an order template.
 Frequent orders can quickly be created using an order template, saving the staff time and minimizing the risk of ordering incorrect items or quantity.

Articles				
Art. type	Article	Name	Price	Count
Item	B-10016	CANNULA, LIPOSUCTION MERCEDES BLUE (BYRON)	0,00	10
Item	B-22213	ARCH BAR, RUBBER BANDS	0,00	10
Item	B-3005	BONE TAMP 6MM	0,00	15
Item	B-3007	BONE TAMP 7MM	0,00	25
Product	03643	Extractor Pliers	0,00	2
Product	03686	Tissue Forcep, Dura w/ Teeth (7BOB)	0,00	5
Product	03688	Retractor, Senns (Cat Paws) Sharp (7BOB)	0,00	3
Product	03690	Knife Handle, #3 (ER)	0,00	12
Product	03773	Medtronic Metrx Clamp	0,00	20

10 20 40 80


Page 1 of 1 (9 results) 1

MCV00109020 REVA



Office/Admin Area


Learn how T-DOC manages the sterile workflow in the Office/Admin Area

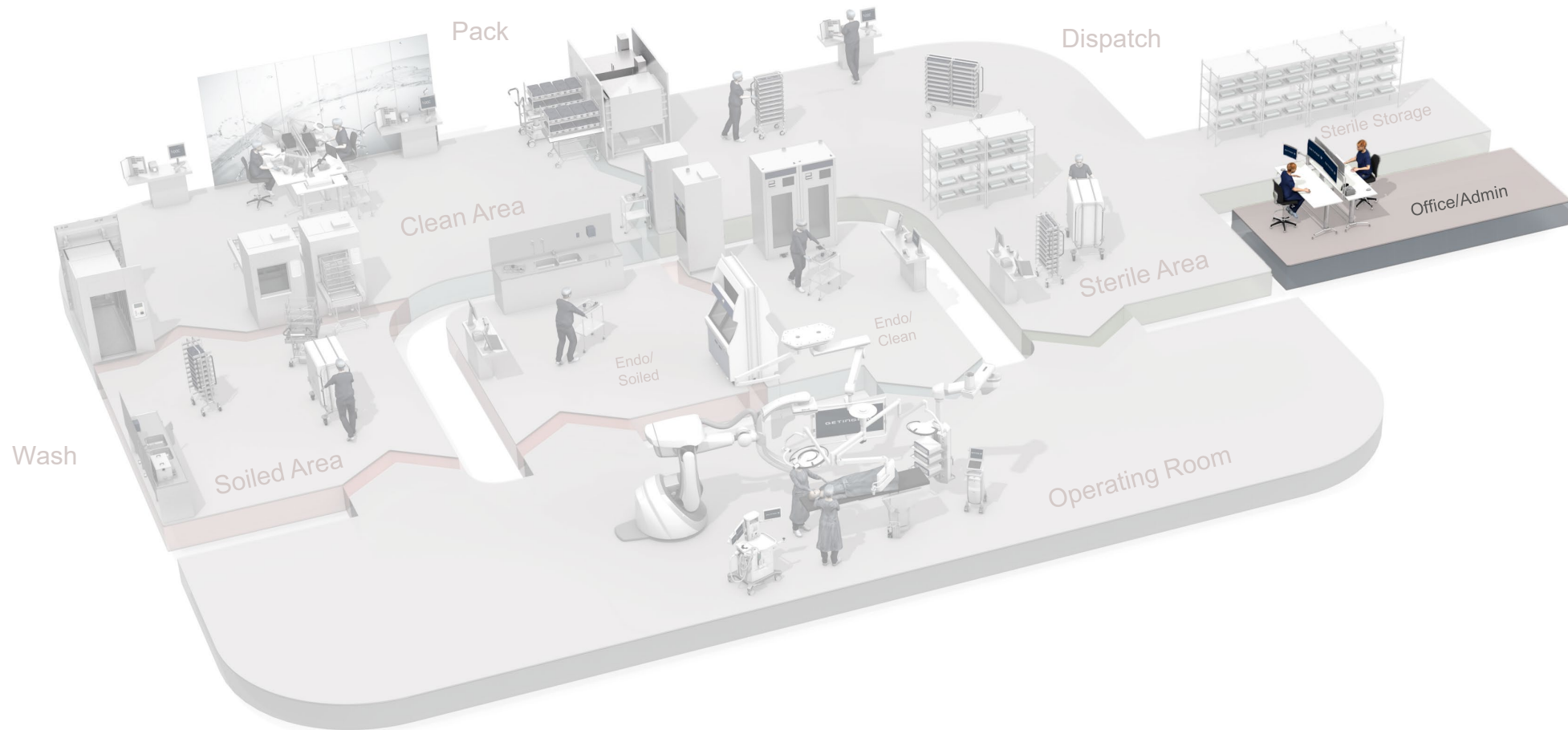

T-DOC Info Overviews

Quick insights on CSSD production, surgeries, resources and goods

- T-DOC Production Overview
- T-DOC Item Exchange Info Overview
- T-DOC Fast Track Info Overview
- T-DOC Machine Viewer
- T-DOC Unit Info Overview
- T-DOC Orders Needs List
- T-DOC Scheduled Operations
- T-DOC Operation Info Overview


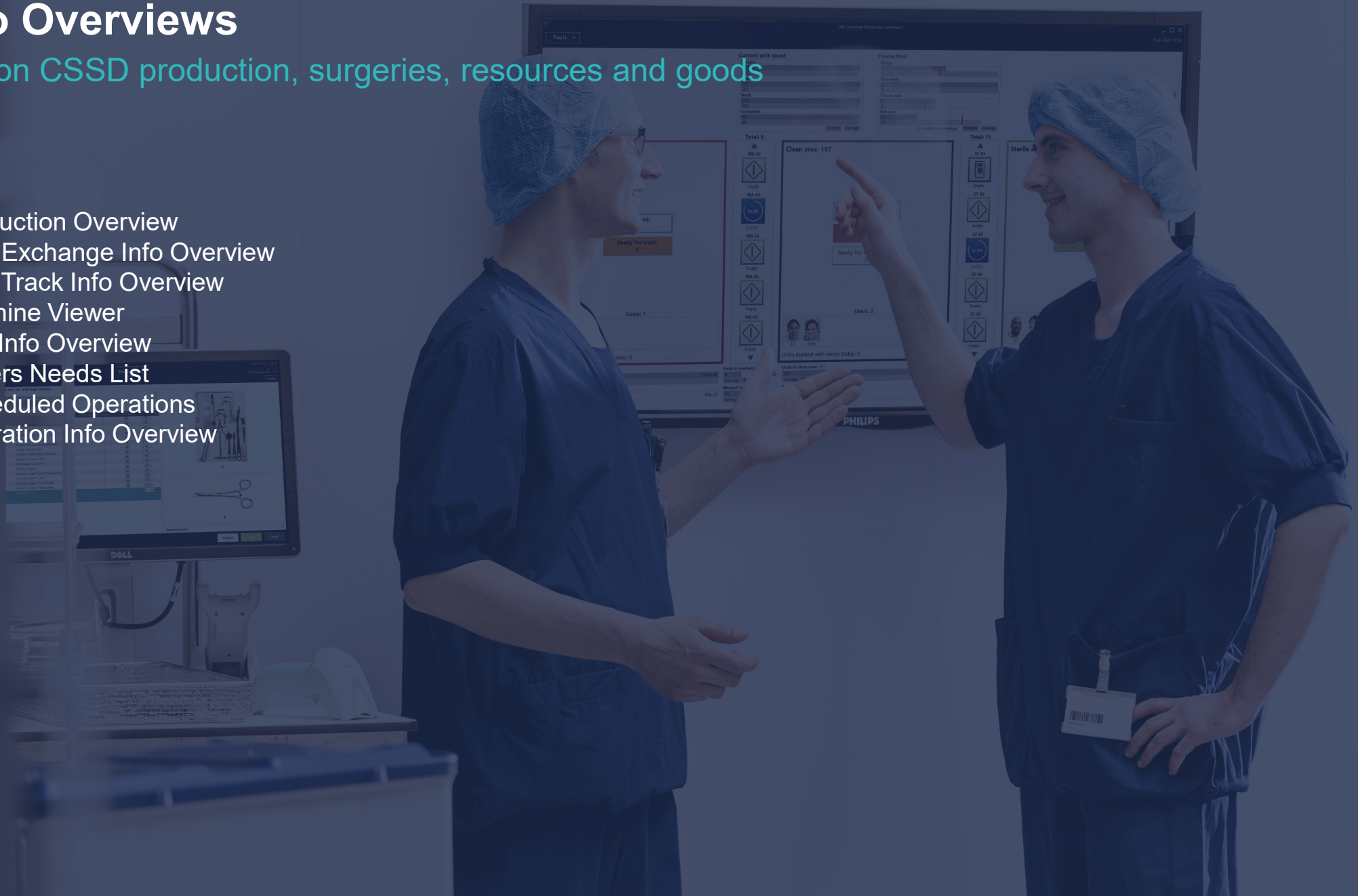


T-DOC Production Info Overview

Tools

Overview of all units, their status, count, and location.
Overview on equipment process status and staff allocation in each area.

Incoming: 52

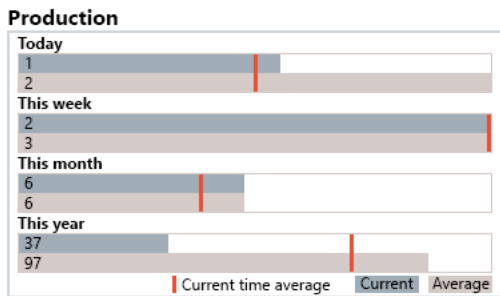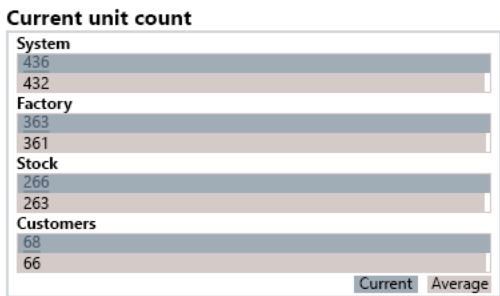

Units at customers: 20

Units on stock: 266 (4 🚫) ⚠️

Unclean area: 45

Returned: 45

Ready for wash: 0

Users: 1

PW

Units marked with errors today: 0

Total: 6

- ▲ WA-01

Empty
- WA-02

6 units
- WA-03

Empty
- WA-04

Empty
- WA-05

Empty

▼

Clean area: 197

Waiting for handling: 189 (2 🚫)

Packed: 6

Ready for sterilization: 2

Users: 2

LP AMB

Units marked with errors today: 0

Total: 15

- ▲ ST-01

Empty
- ST-02

Empty
- ST-03

2 units
- ST-04

Empty
- ST-05

Empty

▼

Sterile area: 40






Ready for handling: 0

Ready for dispatch: 10

Users: 2

KT LW

Units marked with errors today: 0

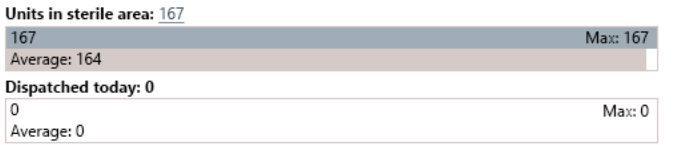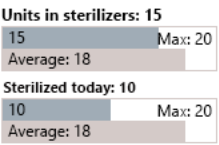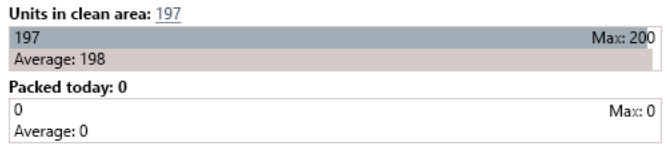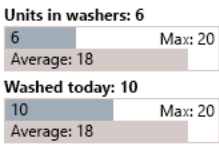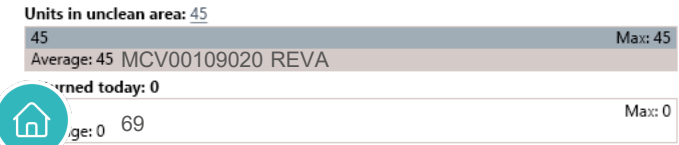

Surplus items

Item **S0002 Tweezers Surgical Fine 20 cm**User **SYS** Time **18-12 05:53** Serial no. Location **PackingArea**Item **S0001 Tweezers Surgical 20 cm**User **SYS** Time **18-12 12:38** Serial no. Location **PackingArea**

Older surplus items

Item **S0004 Tweezers Surgical Fine 15 cm**User **DB** Time **17-12 04:03** Serial no. Location **PostBatchArea**Item **S0003 Tweezers Surgical (Coarse) 15 cm**User **DA** Time **17-12 11:38** Serial no. Location **DispatchArea**

Missing items

Item **S0001 Tweezers Surgical 20 cm**User **DC** Time **18-12 10:58** Serial no. Location **PackingArea**Item **S0005 Tweezers Anatomical 20 cm**User **DA** Time **18-12 03:23** Serial no. Location **PostBatchArea**

Older missing items

Item **S0002 Tweezers Surgical Fine 20 cm**User **SYS** Time **16-12 11:23** Serial no. Location **DispatchArea**Item **S0007 Pince Adson Surgical 12 cm**User **DC** Time **17-12 07:23** Serial no. Location **PackingArea**Item **S0006 Tweezers Anatomical 15 cm**User **DB** Time **17-12 10:43** Serial no. Location **ReturnArea**

The registered missing and surplus instruments in T-DOC CountAssure appear in the T-DOC Item Exchange Info Overview. This overview is typically displayed on a large monitor in the Clean Area, providing staff a quick overview of lost and found instruments.



T-DOC Fast Track Info Overview

Tools ▾

Unit	Product	Product.Item.Name	Status	Location.Name	FT name
1533284	56850	Retractor, Dull Pedi Weitlander 4 3/8"	Packed	Packing	Pack To Dispatch
1533263	03381	Pin Cutter, Acromed Large Side	Packed	Packing	Pack To Dispatch
1533259	4147	Basic Total Hip	Packed	Packing	Pack To Dispatch
1533245	P-00104	Zimmer Nexgen Total Knee Tray	Packed	Packing	Pack To Dispatch
1533282	P-57375	Medtronic Infinity Cervical Thoracic Implant Module	Packed	Packing	
1533272	P-04206	SPA PICK TICKET ITEM – DO NOT USE	Packed	Admin (System)	
1533265	03381	Pin Cutter, Acromed Large Side	Packed	Packing	
1533264	03381	Pin Cutter, Acromed Large Side	Packed	Packing	
1533262	03381	Pin Cutter, Acromed Large Side	Packed	Packing	
1533261	03381	Pin Cutter, Acromed Large Side	Packed	Packing	
1533260	03381	Pin Cutter, Acromed Large Side	Packed	Packing	
1533258	4147	Basic Total Hip	Returned	Return	
1533257	4147	Basic Total Hip	Returned	Return	
1533256	4147	Basic Total Hip	Returned	Admin (System)	
1533253	P-57603	Scissors, Mayo Curved 6" (Outpatient Speech Therapy)	Returned	Return	
1533251	P-04580	Depuy Shoulder Extraction Instruments (Loaner)	Packed	Packing	
1533249	P-00104	Zimmer Nexgen Total Knee Tray	Returned	Admin (System)	
1533248	P-00104	Zimmer Nexgen Total Knee Tray	Packed	Packing	
1533247	P-00104	Zimmer Nexgen Total Knee Tray	Packed	Packing	
1533246	P-00104	Zimmer Nexgen Total Knee Tray	Packed	Packing	
1533241	P-57797	comp with item serial	Packed	Admin (System)	
1533235	P-57332	HALL SAW KEY (CVOR)	Returned	Return	
1533229	P-57332	HALL SAW KEY (CVOR)	Returned	Return	
1533228	B61112	TRAY LARGE ORTHO	Returned	PostBatch Washer	
1533223	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
1533222	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
1533221	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
1533220	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
1533219	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
1533218	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
1533217	MCV00109020 REV B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
16	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	
15	B3966185	HANDLE, BLUE GRAFT KNIFE (ARTHREX)	Packed	Packing	

This overview provides staff with quick insights on which units are marked for fast tracking, the receiving customer, and the reason for the fast tracking.



This overview shows equipment status for all reprocessing equipment in the CSSD and the amount of time left on the equipment process.

All machines

▲ Sterilizers

Sterilizer 1: 0:06

Sterilizer 2:

Sterilizer 3:

Sterilizer 4:

Sterilizer 5:

Sterilizer 6:

▲ Washers

Washer 7: 0:05

Washer 8:

Washer 9:

Washer 10:

▼ Incubators

MCV00109020 REVA

Unit	Product	Product name
72134	PR-01	Large needle holder
72146	9000	Battery box
72145	96520	Metzenbaum Scissors 18 cm
72144	9005	Alligator forceps
72149	91174	Hammer
72148	9000	Battery box
72147	96562	Babcocks, long 9½
72160	93082	Bakes dilators (9)
72159	96562	Babcocks, long 9½
72158	91147	Richardson, small
72157	91124	Bakes dilators (9)
72156	10002	Basic Tray Children
72150	93082	Bakes dilators (9)
72143	000001	Basic Tray Children
72142	000001	Basic Tray Children

This overview shows all units located in the CSSD and provides staff with guidance on which units to process first.

High, medium and low priority are assigned to units based on their defined turnaround time.

Turnaround time definitions (e.g. from location A to B) is particularly relevant for CSSDs that have service-level agreements with their customers.



T-DOC Orders Needs List

Tools ▾

Qty.	Not ready	Type	Number	Name	Order.Exp. del. time	Org. customer.Customer	Operation	Operation type.Name	Room.ID	Doctor.Name
4	4	Item	S-23305	Kelly Clamp	20:52	ORTHO	097124987	Reverse Total Shoulder Arthroplasty	OR 11	Grant^W^Gonz...
6	6	Item	S-23306	Bonney Forceps	20:52	ORTHO	097124987	Reverse Total Shoulder Arthroplasty	OR 11	Grant^W^Gonz...
+ 1	1	Product	P-03411	Biomet ALPS Reamer & Plate Extension (Loaner)	20:52	ORTHO	097124987	Reverse Total Shoulder Arthroplasty	OR 11	Grant^W^Gonz...
+ 1	1	Product	P-04580	Depuy Shoulder Extraction Instruments (Loaner)	20:52	ORTHO	097124987	Reverse Total Shoulder Arthroplasty	OR 11	Grant^W^Gonz...
10		Item	B-10016	CANNULA, LIPOSUCTION MERCEDES BLUE (BYRON)	19:00			Spine Works Center		
10	9	Item	B-22213	ARCH BAR, RUBBER BANDS	19:00			Spine Works Center		
15		Item	B-3005	BONE TAMP 6MM	19:00			Spine Works Center		
25		Item	B-3007	BONE TAMP 7MM	19:00			Spine Works Center		
+ 2	1	Product	03643	Extractor Pliers	19:00			Spine Works Center		
+ 5	3	Product	03686	Tissue Forcep, Dura w/ Teeth (7BOB)	19:00			Spine Works Center		
- 3	2	Product	03688	Retractor, Senns (Cat Paws) Sharp (7BOB)	19:00			Spine Works Center		
Status		Type	Number	Unit	Serial no..Serial no.	Location.Name				
On stock		Original	03688	1187496		PostBatch Sterilizer				
Packed		Original	03688	446740		Packing				
Packed		Original	03688	446742		Packing				
Packed		Original	03688	1196605		Packing				
+ 12	11	Product	03690	Knife Handle, #3 (ER)	19:00	Spine Works Center				
+ 20	20	Product	03773	Medtronic Metrx Clamp	19:00	Spine Works Center				

This info overview provides CSSD staff with insights into what needs to be produced for stock to be able to fulfill the open orders.

It shows all orders, including status, delivery time, customer, and surgery information.

The overview has numerous filtering options to match the user's specific criteria.

MCV00109020 REVA



T-DOC Orders Needs List

Tools ▾

Qty.	Not ready	Type	Number	Name	Order.Exp. del. time	Org. customer.Customer	Operation	Operation type.Name	Room.ID	Doctor.Name
4	4	Item	S-23305	Kelly Clamp	20:52	ORTHO	*****	*****	*****	Grant^W^Gonz...
6	6	Item	S-23306	Bonney Forceps	20:52	ORTHO	*****	*****	*****	Grant^W^Gonz...
+ 1	1	Product	P-03411	Biomet ALPS Reamer & Plate Extension (Loaner)	20:52	ORTHO	*****	*****	*****	Grant^W^Gonz...
+ 1	1	Product	P-04580	Depuy Shoulder Extraction Instruments (Loaner)	20:52	ORTHO	*****	*****	*****	Grant^W^Gonz...
10		Item	B-10016	CANNULA, LIPOSUCTION MERCEDES BLUE (BYRON)	19:00	Spine Works Center	*****	*****	*****	
10	9	Item	B-22213	ARCH BAR, RUBBER BANDS	19:00	Spine Works Center	*****	*****	*****	
15		Item	B-3005	BONE TAMP 6MM	19:00	Spine Works Center	*****	*****	*****	
25		Item	B-3007	BONE TAMP 7MM	19:00	Spine Works Center	*****	*****	*****	
+ 2	1	Product	03643	Extractor Pliers	19:00	Spine Works Center	*****	*****	*****	
+ 5	3	Product	03686	Tissue Forcep, Dura w/ Teeth (7BOB)	19:00	Spine Works Center	*****	*****	*****	
- 3	2	Product	03688	Retractor, Senns (Cat Paws) Sharp (7BOB)	19:00	Spine Works Center	*****	*****	*****	

Status	Type	Number	Unit	Serial no..Serial no.	Location.Name				
On stock	Original	03688	1187496		PostBatch Sterilizer				
Packed	Original	03688	446740		Packing				
Packed	Original	03688	446742		Packing				
Packed	Original	03688	1196605		Packing				
+ 12	11	Product	03690	Knife Handle, #3 (ER)	19:00	Spine Works Center	*****	*****	*****
+ 20	20	Product	03773	Medtronic Metrx Clamp	19:00	Spine Works Center	*****	*****	*****

The overview is configurable and can hide sensitive information to increase the level of privacy. This setting can be applied to all T-DOC info overviews.





Scheduled operations

This overview provides staff with insights on upcoming surgeries, their order status, and location.

Search for

From To

	Orders	Start time	End time	Operation	Operation type...	Name	Room.Name	Status	Patient.ID	Patient.Name	Doctor.Name	Booking status	Create order
	1	08/03/2022 12.54	08/03/2022 13.54	OP056	*****	*****	General Surgery ...	Accepted	56842	*****	Brian Allan	OK	
		15/09/2022 09.04	15/09/2022 12.14	OP055	*****	*****	OR 1	Accepted	69841	*****	Carole Eyre	OK	
	1	16/06/2022 09.13	16/06/2022 09.52	OP050	*****	*****	OR 4	Done	87439	*****	Raymond Hales	OK	
	2	05/06/2022 10.46	05/06/2022 11.46	OP049	*****	*****	Cardiac Room 2	Done	31429	*****	Tara Washer	OK	
	1	01/06/2022 10.33	01/06/2022 11.33	OP048	*****	*****	OR 3	Accepted	87417	*****	Michael Hope	OK	
	2	28/04/2022 12.01	28/04/2022 12.01	OP046	*****	*****	General Surgery ...	Accepted	12384	*****	Garry Colman	OK	
	1	02/02/2022 09.52	02/02/2022 10.52	OP039	*****	*****	OR 2	Accepted	74815	*****	Brian Allan	OK	
	2	30/01/2022 11.15	30/01/2022 12.00	OP043	*****	*****	Cardiac Room 1	Started	36471	*****	Carole Eyre	OK	
	1	20/01/2022 10.42	20/01/2022 11.42	OP037	*****	*****	OR 1	Accepted	64578	*****	Raymond Hales	Error	
		04/12/2022 15.37	04/12/2022 15.58	OP036	*****	*****	OR 4	Accepted	36143	*****	Tara Washer	OK	
		02/12/2022 14.00	02/12/2022 14.30	OP035	*****	*****	OR 3	Accepted	36124	*****	Michael Hope	OK	
	2	22/02/2022 15.03	22/02/2022 16.03	OP033	*****	*****	Cardiac Room 2	Accepted	74134	*****	Garry Colman	OK	
		08/03/2022 12.54	08/03/2022 13.54	OP031	*****	*****	General Surgery ...	Accepted	87417	*****	Balbir Johal	Error	
		28/10/2022 12.09	28/10/2022 12.09	OP030	*****	*****	OR 1	Started	47169	*****	Brian Allan	OK	
	1	01/10/2022 11.23	01/10/2022 11.23	OP028	*****	*****	OR 4	Accepted	323871	*****	Carole Eyre	OK	
	1	30/09/2022 16.03	30/09/2022 16.03	OP029	*****	*****	Cardiac Room 2	Accepted	36148	*****	Raymond Hales	OK	
		22/09/2022 11.06	22/09/2022 13.16	OP027	*****	*****	OR 3	Accepted	74164	*****	Tara Washer	OK	
	1	03/08/2022 11.17	03/08/2022 11.17	OP019	*****	*****	General Surgery ...	Accepted	36174	*****	Michael Hope	OK	
	0	11/03/2022 09.09	11/03/2022 09.09	OP017	*****	*****	OR 2	Started	12416	*****	Garry Colman	OK	
	0	22/02/2022 15.03	22/02/2022 16.03	OP016	*****	*****	Cardiac Room 1	Done	74311	*****	Balbir Johal	OK	



Tools ▾

Start time	Operation	Operation type.Name	Customer.Name	Location
- OR 11				
- 02-05 20:26	89799879899	Reverse Total Shoulder Arthroplasty	Orthopedic Dep.	Arrived at OR
Goods		Location.Name		
Case cart 139		Arrived at OR		
- 03-05 10:15	651651321	Knee Replacement	Orthopedic Dep.	In Transit
Goods		Location.Name		
Case cart 100		In Transit		
Case cart 94		In Transit		
- 04-05 11:22	654684235222	Knee Replacement	Orthopedic Dep.	Not dispatched
Goods		Location.Name		
Order 661918		Not dispatched		
- OR 12				
+ 03-05 10:20	654651321856	Reverse Total Shoulder Arthroplasty	Orthopedic Dep.	Arrived at OR

This overview is for OR staff, notifying them of the status and location of the goods ordered for the upcoming surgeries. Easy-to-read color codes indicate status of goods and save time for the porters and OR staff who don't have to search for the needed goods.



T-DOC Operation Info Overview

Operation ID	Operation	Start time	End time	Location
--------------	-----------	------------	----------	----------

OP Room 1				
38392	OP958162	2021-11-21	2021-11-21	OR Ortho
Goods				Location name
Unit 487192				OR Ortho
Unit 572509				OR Ortho
98786	OP351840	2021-11-21	2021-11-21	OR Ortho
Goods				Location name
Unit 981329				OR Ortho
4345	OP871529	2021-11-21	2021-11-21	OR Ortho
Goods				Location name
Unit 429814				OR Ortho

The overview can be configured to show the units included in each delivery

OP Room 2				
3266	OP672094	2021-11-21	2021-11-21	In transit
Goods				Location name
Unit 715209				OR Ortho
Unit 761302				In transit
4882	OP659142	2021-11-21	2021-11-21	Not dispatched
Goods				Location name
Unit 591023				Not dispatched
4354	OP620936	2021-11-21	2021-11-21	Not dispatched
Goods				Location name
Unit 169258				OR Ortho
Unit 109234				OR Ortho
Unit 461924				OR Ortho
Unit 985126				Not dispatched

OP Room 3				
7132	OP763091	2021-11-21	2021-11-21	OR Ortho
9613	OP152386	2021-11-21	2021-11-21	Not dispatched
2394	OP412903	2021-11-21	2021-11-21	In transit

OP Room 4				
------------------	--	--	--	--

OP Room 5				
------------------	--	--	--	--

OP Room 6				
------------------	--	--	--	--



T-DOC Admin

Structured data management of all sterile and non-sterile goods

- Single items
- Composite items
- Composite items list
- Products



Manage all data related to single items and their tracking, ordering, suppliers, pricing, identical items, and user settings.

Single Items

+ New


Quick search

Full search >>

Recent Factory

Item	Name
S-18905	Biomet Vanguard SSK ...

S-23309 -

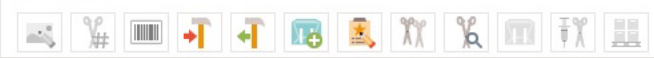

Item
S-23309

Name

Local name



General Price Ordering Material Triggers Identical User fields Events Documents Remark

Group

Specialty

Status
Normal

Trace by
Item

Complexity
0

Factories

Bound to

Community Hospital

Memorial Medical Center

Reservation mode
Not required

New

Item *
S-23309

Name *
Scissors Mayo 15 cm

Supplier

Supplier No.

Save & New Save Cancel



Manage all data related to composite items and their naming, ordering, suppliers, pricing, triggers, and user settings.

Composite Items

+ New

Quick search

[Full search >>](#)

C-09637 - Composite item list



Item: **C-09637**

Lin/Inst:

Name:

Local name:



Item	Name	Ini...
B-C-00014	ACE SCREWDRIVER SET	MS
B-C-00013	ACUFEX BITERS	MS
B-C-00012	ESOPHAGOSCOPY SET, JACKS...	MS
B-C-00010	5MM MONOPOLAR COAGULA...	MS
B-C-0001	NEIVERT SNARE - duplicate	MS
B-C-00009	5MM COAGULATING ROD & C...	MS
B-C-00008	5MM CLIP-ON LIGHT CORD	MS
B-C-00007	45deg & 90deg AWLS	MS
B-C-00005	SKIN HOOKS, DOUBLE 12MM	MS
B-C-00004	WOLF HYSTEROSCOPY 0 DEG T...	MS
B-C-00003	DILATOR, URETHRAL #28	MS
B-C-00002	Z's RETRACTORS & ELEVATORS	MS
B-C-00001	RETRACTOR, "S"	MS
B-09436	SPECULUM, NASAL 1"	MS
C-06970	Depuy Shoulder Extraction Instr...	SYS
C-00614	Medtronic PLIF Tray	SYS
B-C-00015	ADAIR CLAMPS - duplicate	SYS
B-C-00016	ADENOID CURETTES (SET of 3 - ...	SYS
B-C-00017	ADENOID CURETTES (SET of 3 - ...	SYS
B-C-00018	ADSON FORCEPS	SYS
B-C-00019	SPECULUM, EAR ADULT SET	SYS
B-C-0002	CLAMP ALLIS BABY	SYS
B-C-00020	ADULT/PEDI LEAD HANDS MCV00109020 REVA	SYS
B-C-00021	ALLEN WRENCHES	SYS
B-C-00022	ALLIGATORS?	SYS

General | Price | Ordering | Material | Triggers | User fields | Events | Documents | Remark

Group:

Specialty:

Status: **Normal**

Number of tags for washing:

Complexity mode: **Default (Max)**

Complexity:

Factories

Bound to:

Community Hospital

Memorial Medical Center

New

Item *

Name *

Manage all content in the composite item list; items, critical counts, name, and supplier

Composite Items

+ New

Quick search

Full search >>

B-C-00014 - ACE SCREWDRIVER SET Composite item list

Pic...	Item	Count	Critical	Name	Supplier No.	Placement
	S-04950	1	1	6.5 Screwdriver Shank	14616	
	S-13925	1	1	T-Handle	14316	
	B-S-01608	1	1	SCREWDRIVER, CANNULATED 3.5MM (gold)	8241-56-070	
	B-S-00079	1	1	ACE BLACK SCREWDRIVER HANDLE	8241-65-000	
	B-S-00817	2	2	K-WIRE SHORT .062	8290-16-006	
	S-10815	3	3	Pin Guide, 9" Thrd	14012-9	

Item	Name	Ini...
B-C-00014	ACE SCREWDRIVER SET	MS
B-C-00013	ACUFEX BITERS	MS
B-C-00012	ESOPHAGOSCOPY SET, JACKS...	MS
B-C-00010	5MM MONOPOLAR COAGULA...	MS
B-C-0001	NEIVERT SNARE - duplicate	MS
B-C-00009	5MM COAGULATING ROD & C...	MS
B-C-00008	5MM CLIP-ON LIGHT CORD	MS
B-C-00007	45deg & 90deg AWLS	MS
B-C-00005	SKIN HOOKS, DOUBLE 12MM	MS
B-C-00004	WOLF HYSTEROSCOPY 0 DEG T...	MS
B-C-00003	DILATOR, URETHRAL #28	MS
B-C-00002	Z's RETRACTORS & ELEVATORS	MS
B-C-00001	RETRACTOR, "S"	MS
B-09436	SPECULUM, NASAL 1"	MS
C-06970	Depuy Shoulder Extraction Instr...	SYS
C-00614	Medtronic PLIF Tray	SYS
B-C-00015	ADAIR CLAMPS - duplicate	SYS
B-C-00016	ADENOID CURETTES (SET of 3 - ...	SYS
B-C-00017	ADENOID CURETTES (SET of 3 - ...	SYS
B-C-00018	ADSON FORCEPS	SYS
B-C-00019	SPECULUM, EAR ADULT SET	SYS
B-C-0002	CLAMP ALLIS BABY	SYS
B-C-00020	ADULT/PEDI LEAD HANDS MCV00109020 REVA	SYS
B-C-00021	ALLEN WRENCHES	SYS
	ALLIGATORS?	SYS

Single item search

Item	Name
B-209...	SCISSOR, MICRO CASTRO STR
B-S-0...	HYSTEROSCOPE RIGID BX SCISSOR
B-S-0...	SCISSOR IRIS CURVED
B-S-0...	SCISSOR IRIS STRAIGHT
B-S-0...	SCISSOR ARTHROSCOPY
B-S-0...	SCISSOR FORCEP
B-S-0...	SCISSOR JOSEPH STRAIGHT
B-S-0...	SCISSOR 3 3/4" IRIS CVD w/SS TIP
B-S-0...	SCISSOR 3 3/4" IRIS STR w/SS TIP
B-S-0...	SCISSOR 4 1/2" STEVENS TENOTOMY CVD
B-S-0...	SCISSOR 4 1/2" STEVENS TENOTOMY STR
B-S-0...	SCISSOR 4 1/2" SUTURE STR w/SS TIP
B-S-0...	SCISSOR 4 1/8" IRIS CVD w/SS TIP
B-S-0...	SCISSOR 4 1/8" IRIS STR w/SS TIP
B-S-0...	SCISSOR 4 3/4" SUTURE CVD w/SS TIP
B-S-0...	SCISSOR 4 3/4" SUTURE STR w/SS TIP
B-S-0...	SCISSOR 4 3/8" IRIS STR w/SS TIP
B-S-0...	SCISSOR 4" IRIS CVD w/SS TIP
B-S-0...	SCISSOR 4" IRIS STR w/SS TIP
B-S-0...	SCISSOR 5 1/2" MAYO DISSECTING CVD
B-S-0...	SCISSOR 5 1/2" MAYO DISSECTING STR

Smart search is not currently active! (300+)

Edit Save Cancel

Manage and create products, their reprocessing, pricing, triggers, user settings, fast tracking, etc.

Products

+ New

Quick search

Full search >>

P-57798 - ACE SCREWDRIVER SET



Product
P-57798

Type
Tray

Item
B-C-00014

ACE SCREWDRIVER SET

Traceability
Full traceability

Trace type
Serial number

Recent Factory

Product	Name
C6758225	GRASPER THO...

General Packing Wash / ST Price Triggers Indicators Timing Identical User fields Fast tracking Events Documents Remark

Customer

Customer site

Dispatch restriction
Default

Reservation mode
Not required

Counting point group

Stock overview

Packed	0
On stock	0
Dispatched	0
Repair	0
Total count	0

New

Product *
P-57798

Item *
B-C-00014

Packing material *
Spec Cont

Save & New Save Cancel



Stock Placement

Loan product

Primary

er






T-DOC Reports

Gain insights and make data-driven decisions based on T-DOC Reports

- Machine utilization
- Production per customer
- Production overview – errors
- Recall of expired units
- New subscription



Machine utilisation w/percentage Sterilizer

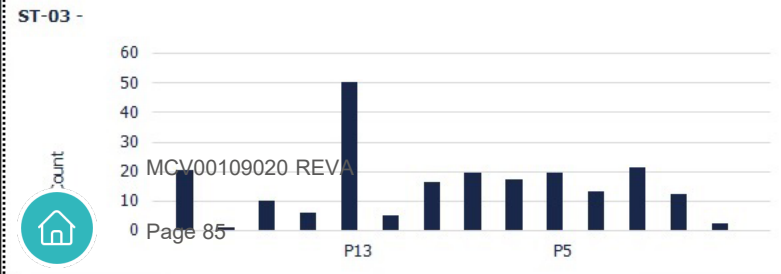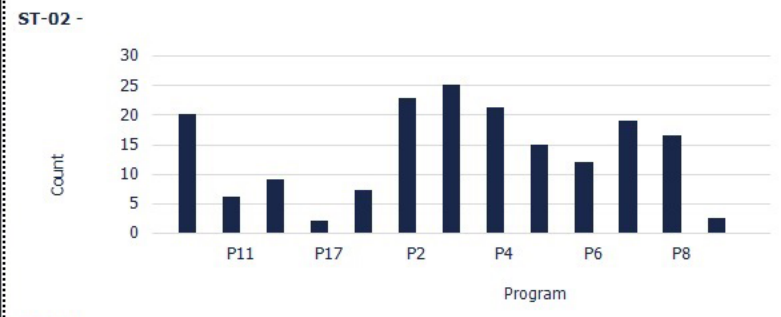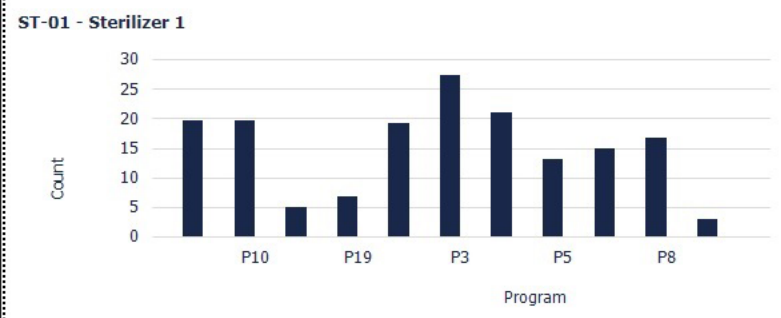


Customers "NONE",1C,1C (6030),1E,2B (615C) Created from Created to
 Factory Community Hospital,Memorial Me Order by

View Report

1 of 3 100% Find | Next


Production per customer

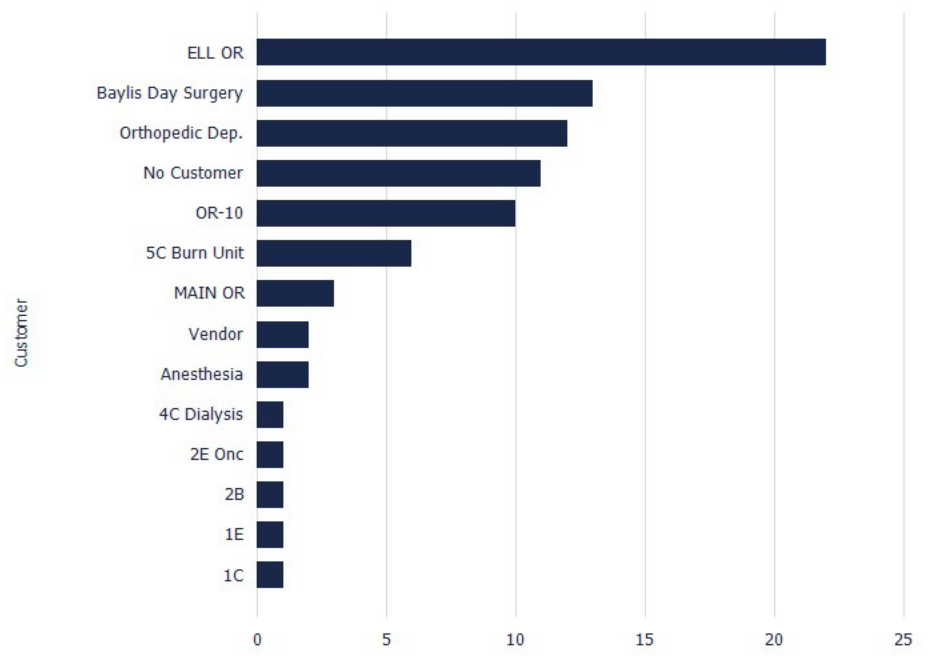


MCV00109020 REVA

	No Customer	
	Page 86	Total 11
	1C	

Production overview - Errors

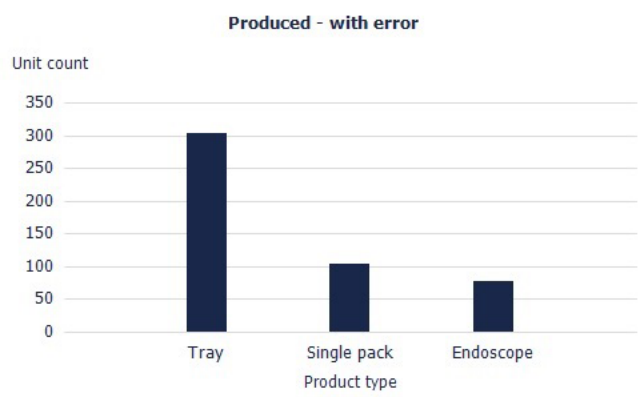

Produced products with error

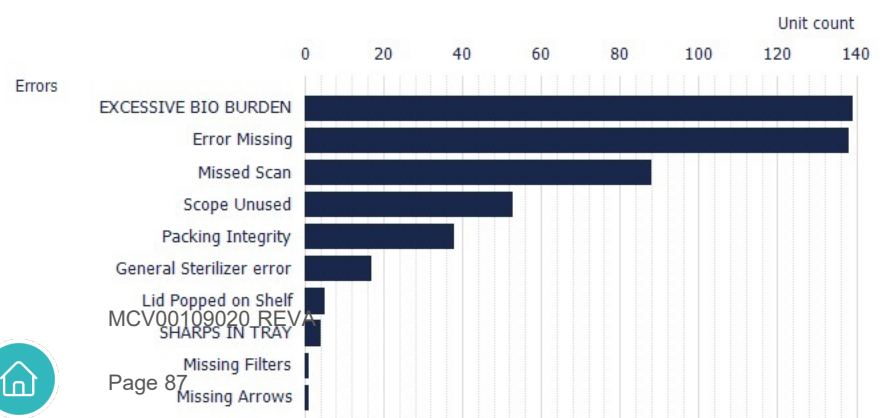

Mode: Recall Expires before: 2/22/2022 Product:

Own stock: No Customers: 1C,1C (6030),1E,2B (6150),2C (62) C6758225,03381,03622,03643,036

Type: Tray,Single pack,Endoscope Factories: Community Hospital,Memorial Me

[View Report](#)



Recall of expired units

Anesthesia Anesthesia				
CVOR CVOR				
Echo Lab (7130) Echo Lab				
Unit	Product		Stock Placement	Expiry date
1282946	ECHO-04601-01	Tee Probe X72t SN:B1TYVV	ECHO	9/23/2019
1471871	ECHO-57612-01	Tee Probe X92t SN:B3CN11	ECHO	8/27/2020
1498033	P-57776-01	Tee Probe X72t SN:B1ZKMT	ECHO	10/9/2020
1530199	Echo-04103-01	TEE Prob S/N B3NCT2	Echo	12/8/2020
1531762	Echo-03794-01	TEE Probe (for EPIQ7) S/N B1W3HB	Echo Lab	12/10/2020
1531764	ECHO-57546-01	Tee Probe X72t SN:B2VQVT	ECHO	12/10/2020
1532308	Echo-04883-01	TEE Probe Pediatric(S7-3t) S/N B20ZXT	Echo Lab	12/11/2020
1532309	Echo-03793-01	TEE Probe (for EPIQ7) S/N B18YJW	Echo Lab	12/11/2020
1532341	ECHO-57760-01	Tee Probe X82t SN:B34TM9	ECHO	12/11/2020
MAIN OR MAIN OR				
Unit	Product		Stock Placement	Expiry date
907256	Generic-ENF-GP-01	ENF-GP	Loaner	1/23/2018
MCV00109020 REVA				
OR-07 OR-07				
Laboratory Therapy Respiratory Therapy				
PA				



1 of 2 100% Find | Next

Recall of expired units

- Anesthesia Anesthesia
- CVOR CVOR
- Echo Lab (7130) Echo Lab

Unit	Product	Stock Placement
1282946	ECHO-04601-01 Tee Probe X72t SN:B1TYVV	ECHO
1471871	ECHO-57612-01 Tee Probe X92t SN:B3CN11	ECHO
1498033	P-57776-01 Tee Probe X72t SN:B1ZKMT	ECHO
1530199	Echo-04103-01 TEE Prob S/N B3NCT2	Echo
1531762	Echo-03794-01 TEE Probe (for EPIQ7) S/N B1W3HB	Echo Lab
1531764	ECHO-57546-01 Tee Probe X72t SN:B2VQVT	ECHO
1532308	Echo-04883-01 TEE Probe Pediatric(S7-3t) S/N B20ZXT	Echo Lab
1532309	Echo-03793-01 TEE Probe (for EPIQ7) S/N B18YJW	Echo Lab
1532341	ECHO-57760-01 Tee Probe X82t SN:B34TM9	ECHO

- Word
- Excel
- PowerPoint
- PDF
- TIFF file
- MHTML (web archive)
- CSV (comma delimited)
- XML file with report data
- Data Feed

Different output formats

Unit	Product	Stock Placement	Expiry date
907256	Generic-ENF-GP-01 ENF-GP	Loaner	1/23/2018

New Subscription

Home > T-DOC > Production > Statistics - Production > Production overview > Manage > Subscriptions > New Subscription

Owner

Type of subscription

- Standard subscription
Generate and deliver one report
- Data-driven subscription
Generate and deliver one report for each row in a dataset

[Learn more](#)

Schedule

Deliver the report on the following schedule:

- Shared schedule
- Report-specific schedule [Edit schedule](#)

Destination

Deliver the report to:

Report parameters

Parameter	Source of value	Value/field
Date from	<input type="text" value="Use default value"/>	<input type="text" value="Jan 1, 2022 12:00:00 AM"/> 
Date to	<input type="text" value="Use default value"/>	<input type="text" value="Jan 31, 2022 12:00:00 AM"/> 
PARAMSUMMARY	<input type="text" value="Use default value"/>	<input type="text" value="No"/>



GETINGE

PASSION FOR LIFE