

Nebulization: Aerogen with Servo Ventilator

Short Guide



1. Connect the Aerogen control cable to the ventilator system.




Servo-u and Servo-n ventilators nebulizer port is on the side of the ventilator.




Servo-air ventilators – nebulizer port is at the front of the ventilator.

2. Connect the nebulizer between the inspiratory tube and the Y piece.

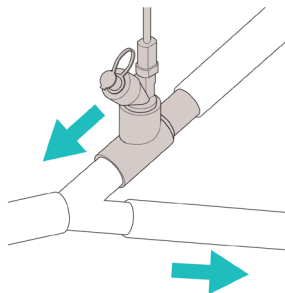


or between the inspiratory tube and the dry side of the humidifier.

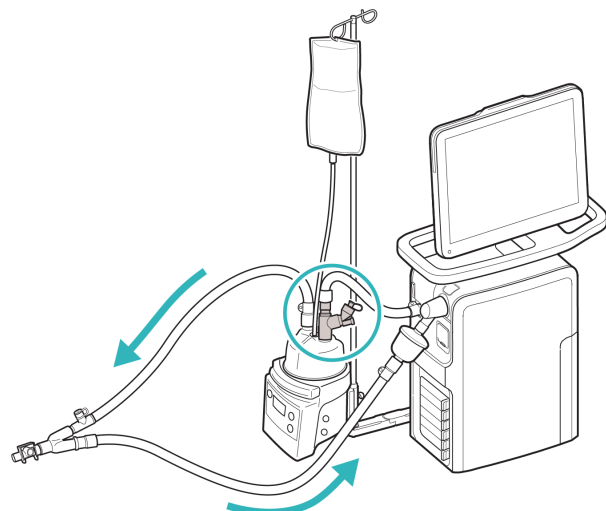

3. To operate the nebulizer.

Tap **MANEUVERS (1)** and then **NEBULIZATION (2)**

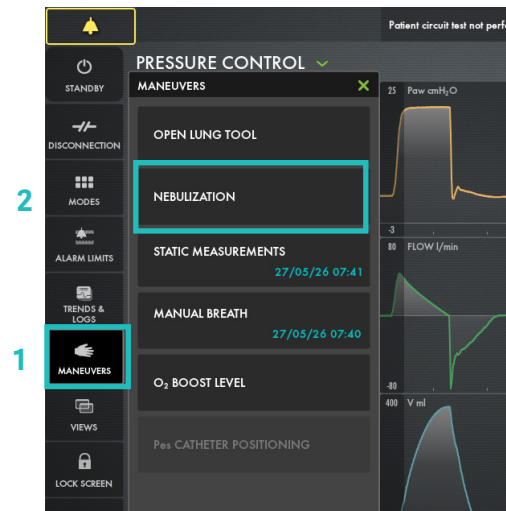

4. You can choose the nebulization period (1) or continuous nebulization (2).

The time for the nebulization period can be changed (3).

Tap start nebulization

No extra gas volume is added, i.e. ventilator system settings and values are not affected.



5. Check that nebulization is in progress and how much time remains by looking at the progress symbol at the top of the screen.

Tap the symbol for access to deactivate nebulization.



- Do not use the nebulizer without a filter, e.g. Servo Duo Guard, connected to the expiratory inlet of the ventilator system.
- During nebulization, carefully monitor the airway pressure. Increased airway pressure could result from a clogged expiratory filter. Replace the filter if the expiratory resistance increases.
- Do not use an HME during nebulization. The humidifier can become blocked or the drug can become caught in the humidifier.

IMPORTANT: Refer to the Servo-u/n/air user's manual for operation of the ventilator.