

Test Soil

WC 120

Indications for Use

The SteriTec Test Soil is designed to simulate the soiling that naturally occurs during theatre use. The Test Soil is supplied in powder form as individual test pots.

Instructions for Use

1. For safety purposes, wear appropriate personal protective gear throughout the procedure.
2. Simply add water to the fill line.
3. Replace the cap.
4. Shake vigorously until even consistency.
5. Leave standing for 10 minutes.
6. Apply liberally to test pieces in test load with the brush provided.
7. Following a 30-minute** drying period at room temperature, the test load should be cleaned using normal procedures and then inspected for residual soil.

**Drying time should not exceed 2 hours.

The bright red color of the Test Soil allows easy identification of areas that have not been properly cleaned, providing a visual analysis of equipment efficiency.

Storage

0° - 5°C (32° - 41°F). 30 – 60 % RH.

Expiry Date

The expiry date is printed on the product packaging. Do not use after the expiration date on the packaging.