

# Oversizing information

## Advanta V12 balloon expandable covered stent

5-10 mm stent oversizing information at 12 ATM rated burst pressure\*

Stent diameter/length	Stent length after oversizing to:					
	5 mm	6 mm	7 mm	8 mm	9 mm	10 mm
5X16	15.6	15.1	14.2	-	-	-
5X22	21.0	20.2	19.4	-	-	-
5X32	32.3	31.5	31.2	29.6	29.2	-
5X38	37.7	37.0	35.7	34.7	32.7	-
5X59	60.0	58.7	57.5	56.5	54.0	-
6X16	-	15.1	14.2	-	-	-
6X22	-	20.2	19.4	-	-	-
6X32	-	31.5	31.2	29.6	29.2	-
6X38	-	37.0	35.7	34.7	32.7	30.9
6X59	-	58.7	57.5	56.5	54.0	52.5
7X16	-	-	14.2	-	-	-
7X22	-	-	19.4	-	-	-
7X32	-	-	31.2	29.6	29.2	-
7X38	-	-	35.7	34.7	32.7	30.9
7X59	-	-	57.5	56.5	54.0	52.5
8X32	-	-	-	29.6	29.2	-
8X38	-	-	-	34.7	32.7	30.9
8X59	-	-	-	56.5	54.0	52.5
9X32	-	-	-	-	29.2	-
9X38	-	-	-	-	32.7	30.9
9X59	-	-	-	-	54.0	52.5
10X38	-	-	-	-	-	30.9
10X59	-	-	-	-	-	52.5

12 mm stent oversizing information at 10 ATM rated burst pressure\*

Stent diameter/length	Stent length after oversizing to:		
	12 mm	14 mm	16 mm
12X29	25.3	24.4	22.5
12X41	37.1	35.8	33.4
12X61	57.6	56.3	52.5

\* Data is provided for the maximal expansion size within the product matrix  
 - Not recommended

The Advanta V12 stent is not available in the U.S.

The Advanta V12 balloon expandable covered stent is manufactured by Atrium Medical Corporation

• Getinge and **GETINGE** \* are trademarks or registered trademarks of Getinge AB, its subsidiaries or affiliates in the United States or other countries • Copyright 2024 Atrium Medical Corp. • All rights not expressly granted are reserved. • Refer to Instructions for Use for current indications, warnings, contraindications, and precautions. • PN 012122 Rev AA 08/2024

**Manufacturer:** Atrium Medical Corporation • 40 Continental Blvd • Merrimack, NH 03054 • USA • +1 603-880-1433