

Advanta V12

covered stent system



Table of contents

Bridge with confidence.....	4-5
Fenestrated and Branched Endovascular Aortic Repair (F/BEVAR).....	6-7
Iliac Branch Device (IBD) aneurysmal repair.....	8
Bridging procedure feature and benefits.....	9
Clinical confidence in occlusive disease.....	10-11
Aortoiliac Occlusive Disease (AIOD).....	12
Occlusive disease features and benefits.....	13
Ordering information.....	14-15




Advanta V12 Bridge with confidence.¹

A balloon expandable covered stent built for consistent, lasting success in FEVAR, BEVAR, and IBD procedures.¹

Advanta V12 covered stent has been meeting the needs of surgeons and patients for 20 years, and is the only durable solution backed by decades of real-world evidence. Now approved for bridging in FEVAR, BEVAR, and IBD aneurysmal repair, the Advanta V12 brings the same trusted performance and durability with added confidence for you and your patients.¹

Choose the stent that's designed – and proven – to go the distance, giving you confidence with every crossing.¹

Supporting aneurysm coverage in fenestrated and branched configurations¹

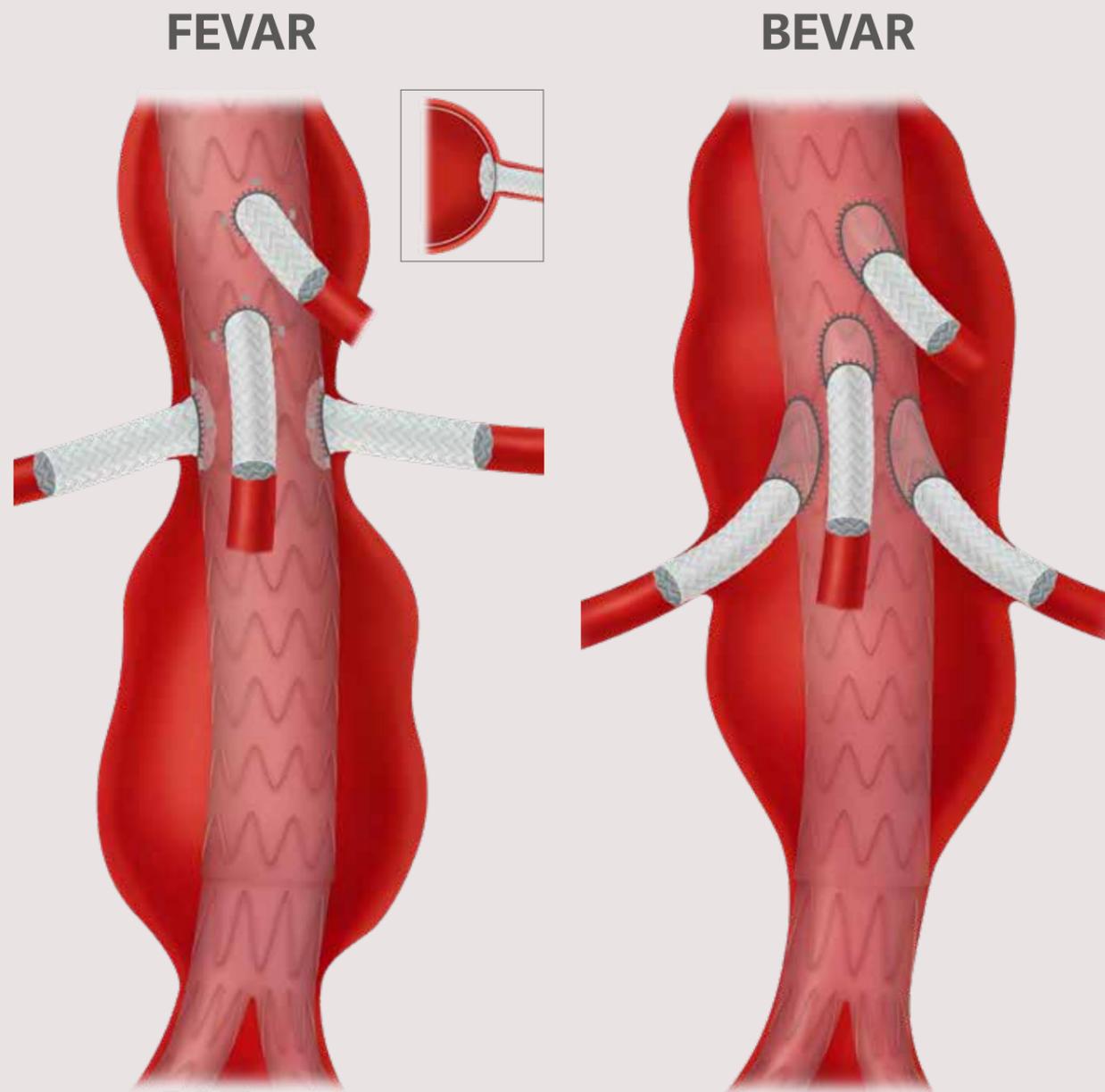


ADVANTA V12 AS BRIDGING STENT IN FEVAR

ADVANTA V12 AS BRIDGING STENT IN BEVAR

Clinical outcomes show high patency and low complication rates in FEVAR/BEVAR^{2,3}


Meta-analysis (mean follow-up ~2 years)²

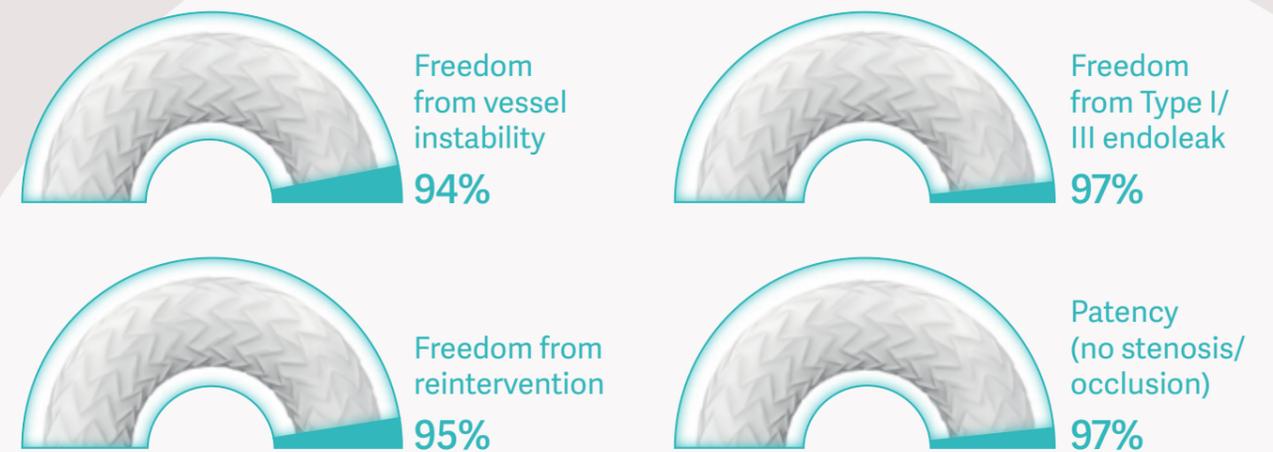

8-year outcomes (retrospective study, 636 patients/1675 target vessels)³



Supporting aneurysm coverage in IBD aneurysm repair¹

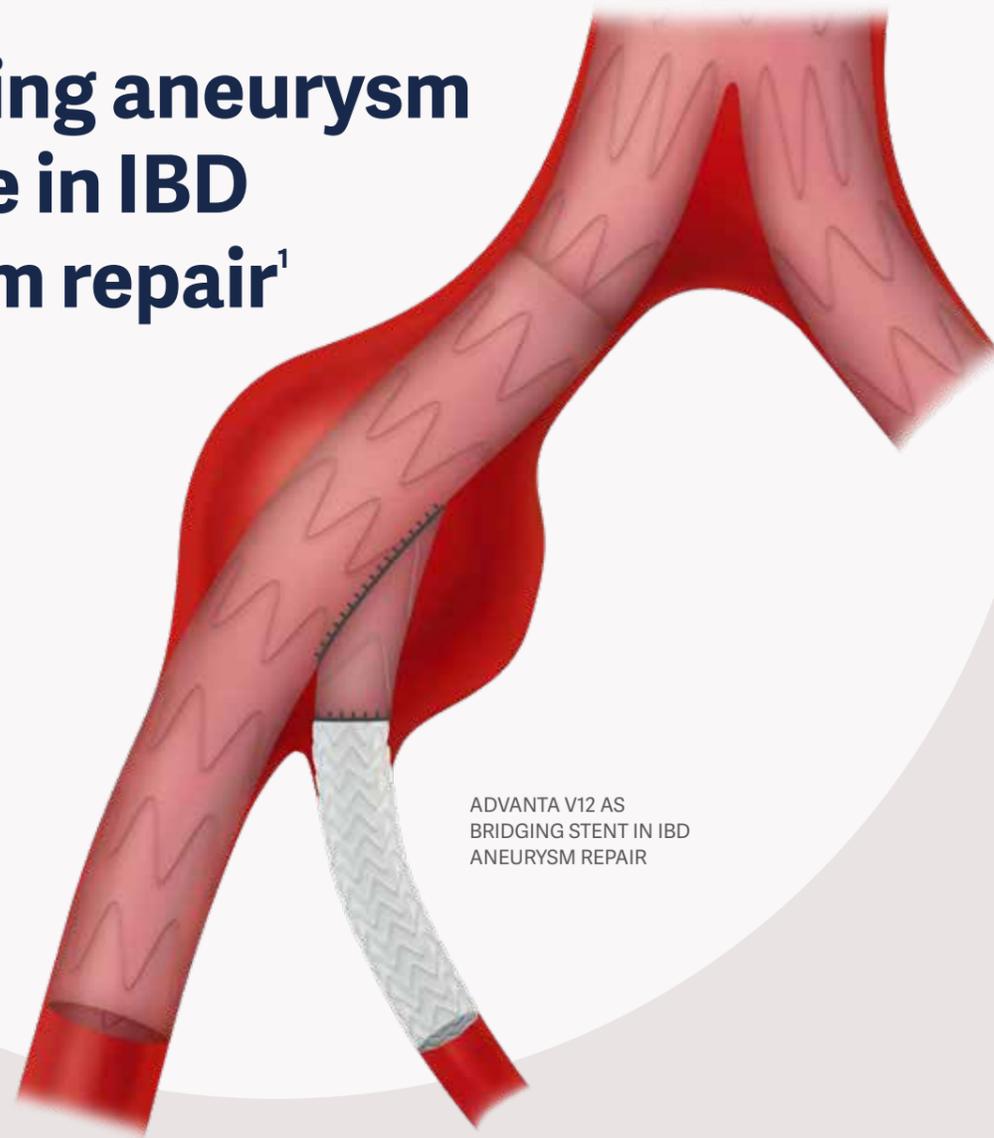

ADVANTA V12 AS BRIDGING STENT IN IBD ANEURYSM REPAIR


Durability isn't optional. It's vital.¹

Built for durability and precision, Advanta V12 lets you bridge to more treatment possibilities – confidently and reliably¹

- Full ePTFE encapsulation provides smooth inner lumen offering ease of navigation during re-intervention¹
- Predictable recoil and foreshortening promotes precise deployment¹
- Stainless steel design provides radial force to support dependable sealing¹
- Low-profile, flexible delivery and consistent deployment facilitate precise placement and optimized seal integrity¹
- Customizable to conform to the anatomy (tapered and flared)^{1,11}

5-year outcomes⁴

Prospective, multicenter, single-arm pivotal study (PRESERVE II)

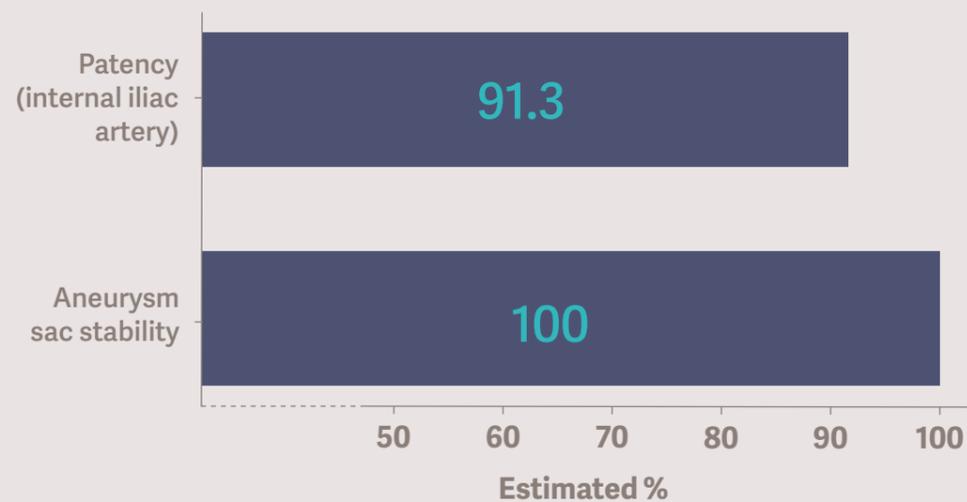

ADVANTA V12 COVERED STENT SYSTEM



Advanta V12

Clinical confidence in occlusive disease^{1,5,6,8,9,13}

The first-to-market, balloon-expandable, fully encapsulated stent, trusted by physicians with nearly 1 million units implanted worldwide.¹

A cornerstone in advanced vascular care


Advanta V12 has long delivered consistent results in the treatment of occlusive disease. Backed by decades of real-world clinical experience, it provides physicians with the reliability, precision, and durability needed to optimize patient outcomes.^{1,5,6}

Choose the stent that's designed—and proven—to go the distance, giving you confidence with every lesion.¹

Kiss with confidence^{1,5,6,8,10,15}


The only balloon expandable covered stent with real-world data spanning decades, including a reported **5-year primary patency rate of 74.7%**.⁵

High long-term patency and low reintervention rates in aortoiliac occlusive disease.⁶

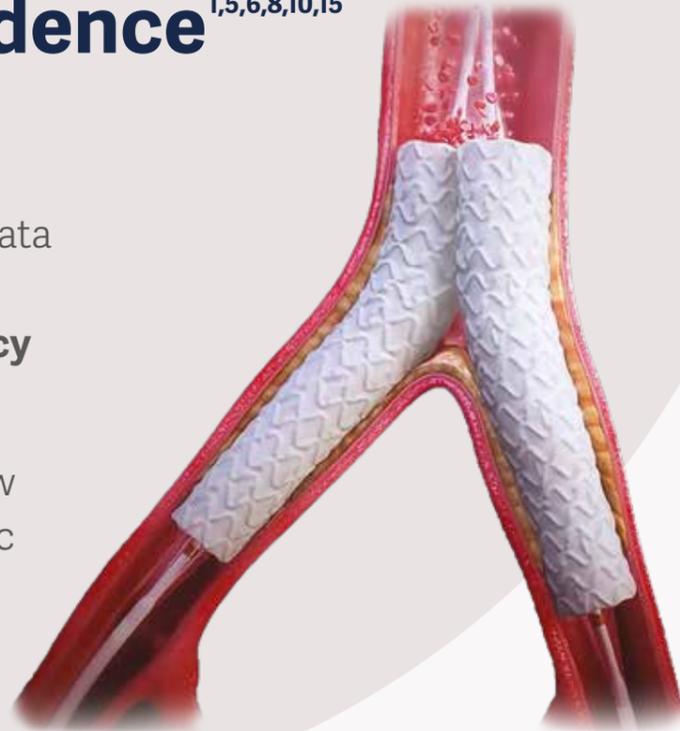

ADVANTA V12 AS A KISSING STENT IN AORTOILIAC OCCLUSIVE DISEASE

COBEST - 5-year results: Advanta V12 vs. Bare Metal Stent⁶

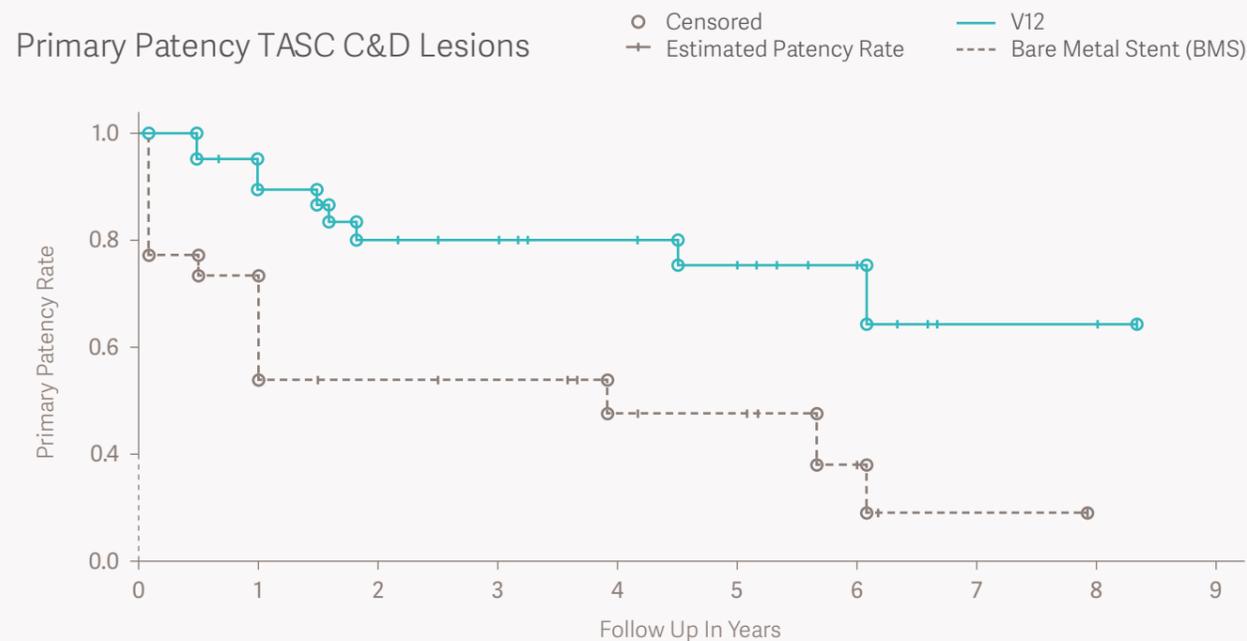

Significantly higher patency in complex TASC C&D lesions compared to bare metal stents at 5 years. (p=.003)

Designed to adapt to diverse occlusive disease treatment requirements¹

Engineered with a unique ePTFE covering¹

Full encapsulation with ePTFE:¹

- Offers a barrier to neointimal hyperplasia^{1,14}
- Helps mitigate the risks related to vessel perforation^{1,9,12,16}



12mm large diameter stents with 9Fr compatibility offers increased treatment options and direct access to aorto-iliac anatomy^{1,7}

Stent structure, cell design, and system provide versatility and flexibility in delivery and placement¹

Low profile 6 and 7 Fr compatible on 5-10 mm sizes. 6 French compatible with most common renal sizes¹

Designed for pushability and trackability through tortuous anatomy with conformance to iliac and renal arteries¹

Ordering information¹

Advanta V12

covered stent system

5-12mm stent diameter, .035" guidewire

Stent diameter/length (mm)*	Order number 80cm catheter length	Order number 120cm catheter length	Foreshortened length		Introducer compatibility (Fr)
			8 ATM nominal pressure†	12 ATM rated burst pressure†	
5 x 16	GBX05161C	GBX05162C	15.9	15.6	6
5 x 22	GBX05221C	GBX05222C	21.3	21.0	6
5 x 32	GBX05321C	GBX05322C	32.3	32.3	7
5 x 38	GBX05381C	GBX05382C	37.2	37.7	7
5 x 59	GBX05591C	GBX05592C	58.6	60.0	7
6 x 16	GBX06161C	GBX06162C	15.7	15.1	6
6 x 22	GBX06221C	GBX06222C	20.8	20.2	6
6 x 32	GBX06321C	GBX06322C	31.7	31.5	7
6 x 38	GBX06381C	GBX06382C	36.6	37.0	7
6 x 59	GBX06591C	GBX06592C	57.8	58.7	7
7 x 16	GBX07161C	GBX07162C	15.0	14.2	7
7 x 22	GBX07221C	GBX07222C	20.1	19.4	7
7 x 32	GBX07321C	GBX07322C	31.3	31.2	7

Stent diameter/length (mm)*	Order number 80cm catheter length	Order number 120cm catheter length	Foreshortened length		Introducer compatibility (Fr)
			8 ATM nominal pressure†	12 ATM rated burst pressure†	
7 x 38	GBX07381C	GBX07382C	35.8	35.7	7
7 x 59	GBX07591C	GBX07592C	57.1	57.5	7
8 x 32	GBX08321C	GBX08322C	30.0	29.6	7
8 x 38	GBX08381C	GBX08382C	34.7	34.7	7
8 x 59	GBX08591C	GBX08592C	56.0	56.5	7
9 x 32	GBX09321C	GBX09322C	28.7	29.2	7
9 x 38	GBX09381C	GBX09382C	33.7	32.7	7
9 x 59	GBX09591C	GBX09592C	54.6	54.0	7
10 x 38	GBX10381C	GBX10382C	30.8	30.9 ^{††}	7
10 x 59	GBX10591C	GBX10592C	53.3	52.5 ^{††}	7

Stent diameter/length (mm)*	Order number 80cm catheter length	Order number 120cm catheter length	Foreshortened length		Introducer compatibility (Fr)
			8 ATM nominal pressure†	10 ATM rated burst pressure†	
12 x 29	85370	85379	25.6	25.3	9
12 x 41	85371	85380	37.6	37.1	9
12 x 61	85372	85381	58.2	57.6	9

*Use in renal arteries is limited to 5 mm, 6 mm and 7 mm diameter sizes.

† Provided as general guidance based on averages observed during in vitro testing.

†† Maximum post-dilated stent diameter for 10 mm balloon diameters is 10.5 mm in kissing configuration.

References

1. Data on file.
2. Nana et al. Systematic Review and Meta-analysis of Outcomes of the Advanta V12 or iCAST Bridging Stent Graft Used for Fenestrated and Branched Endovascular Aortic Repair. *Eur J Vasc Endovasc Surg*. Jan 2025.
3. Katsargyris et al. Long Term Outcomes of the Advanta V12 Covered Bridging Stent for Fenestrated and Branched Endovascular Aneurysm Repair in 1,675 Target Vessels. *European Journal of Vascular Endovascular Surgery* (2023) 66, 313 - e321.
4. PRESERVE II trial shows robust five-year outcomes. Lee A, MD. Presentation at Vascular Annual Meeting (VAM), 2024.
5. Mwipatayi, B.P., et al., A systematic review of covered balloon-expandable stents for treating aorto-iliac occlusive disease. *Journal of Vascular Surgery*, 2020.
6. Mwipatayi, B.P., et al., Durability of the balloon-expandable covered versus bare-metal stents in the Covered versus Balloon Expandable Stent Trial (COBEST) for the treatment of aortoiliac occlusive disease. *Journal of Vascular Surgery*, 2016.
7. Laird JR, Loja M, Zeller T, et al. iCAST balloon-expandable covered stent for iliac artery lesions: 3-year results from the iCARUS multicenter study. *J Vasc Interv Radiol*. 2019.
8. Sabri SS, Choudhri A, Orgera G, et al. Outcomes of covered kissing stent placement compared with bare metal stent placement in the treatment of atherosclerotic occlusive disease at the aortic bifurcation. *J Vasc Interv Radiol*. 2010.
9. Al-Mukhaini M, Panduranga P, Sulaiman K, et al. Coronary perforation and covered stents: an update and review. *Heart Views*. 2011.
10. Grimme FAB, Reijnen MMPJ, Pfister K, et al. Polytetrafluorethylene covered stent placement for focal occlusive disease of the infrarenal aorta. *Eur J Vasc Endovasc Surg*. 2014.
11. van der Riet C, Schuurmann RCL, Verhoeven ELG, et al. Three-dimensional geometric analysis of balloon-expandable covered stents improves classification of complications after fenestrated endovascular aneurysm repair. *J Clin Med*. 2022.
12. Feldman DN, Armstrong EJ, Aronow HD, et al. SCAI guidelines on device selection in aorto-iliac arterial interventions. *Catheter Cardiovasc Interv*. 2020;96(4):915-929.
13. Harris W., et al., Covered stents convey improved performance over bare metal stents for atherosclerotic renal artery stenosis. *J Vasc Surg*, 2013.
14. Rogers C, et al., Non-GLP study of biologic responses to uncoated and PTFE coated steel stents in rabbit iliac arteries. MIT iCast IH Study, 1997.
15. Boulitrop, C., et al., From the Aortic Bifurcation to the Groin: Long-term Outcomes of Covered Kissing Stent Placement in Combination with Iliofemoral Reconstruction for Extensive Iliofemoral Occlusive Disease. *Annals of Vascular Surgery*, 2020.64: p. 11-16
16. Dubosq, M. and R. Coscas, The Iliacs and the Odyssey of Covered vs. Bare Metal Stents. *European Journal of Vascular and Endovascular Surgery*, 2023. 65(3): p. 369.



The Advanta V12 covered stent system is not available in the U.S.

The Advanta V12 covered stent system is manufactured by Atrium Medical Corporation. Getinge and GETINGE * are trademarks or registered trademarks of Getinge AB, its subsidiaries or affiliates in the United States or other countries. Copyright 2024 Atrium Medical Corporation. All rights not expressly granted are reserved. Refer to Instructions for Use for current indications, warnings, contraindications, and precautions. PN012320 RevAA 01/2026

Atrium Medical Corporation
40 Continental Blvd
Merrimack, NH 03054
USA
+1 603-880-1433
www.getinge.com



**Learn More
about
Advanta V12**