

Getinge Clean

Instrument Brightener

Instrument Brightener is designed to remove rust, corrosion, hardwater deposits and minor pitting from stainless steel instruments.



Application

Excellent for spot cleaning of scale, oxides and corrosion from stainless steel carts, trays, basins and washer chambers. This product is primarily designed for compatibility with stainless steel. Before use on other substrates, check with manufacturer of the material.

Key Features

- Enhances service life of instruments
- Reduces replacement or repair cost by loosening box locks and joints
- Restores bright, original appearance of the instrument finish

Usage Information

ALWAYS DILUTE WITH WATER BEFORE USE.

For brightening/destaining instruments, use the following procedure:

1. Preclean the instruments according to your normal standard operating procedures.
2. Set washer up to wash at 160°F (71°C) for 30 minutes.
NOTE: New washers will not allow more than 30 minutes on the wash phase.
3. Two rinses with no dry time.
4. Disable the drain function at the beginning of the wash cycle.
5. Manually add 1 to 2 quarts of Instrument Brightener into the chamber for every 8 gallons of water in the sump.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Not for use on aluminum, brass or other soft metals. Heavily stained/ corroded instruments may require more than one treatment.

Physical and Chemical Properties

- Clear liquid
- pH value: 1.6–2.7
- Characteristic odor

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean

12 1-Quart Bottles

Order Number

61301605067

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044180 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 