

Getinge Clean

PowerCon Acid Detergent Concentrate

PowerCon Acid Detergent Concentrate combines organic and inorganic acids with surfactants in a proprietary blend. Achieves excellent results in a wide variety of operating conditions.



Application

Can be used in laboratories, hospitals and other institutions on instruments, utensils, glassware and animal cages.

Key Features

- Economical
- Powerful and convenient
- High-performance cleaning
- Strong dissolving action against minerals and scale

Usage Information

Always follow manufacturer's directions for the particular automatic washer being used. Calibrate dispensing equipment to deliver a minimum of 1/6 of an ounce per gallon of water (1.30 ml/L) up to 1/2 of an ounce per gallon (3.9 ml/L). Poor water quality may require an increased product dosage. Consult your Getinge representative for appropriate chemical concentration and machine cycle information.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild, characteristic fragrance
- pH value: <1.0–1.5

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean	Order Number
2 x 1 Gallon	61301605569
2.5 Gallons	61301605570
5 Gallons	61301606233

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044165 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

