

Getinge Clean

PowerCon Alkaline Detergent Concentrate

PowerCon Alkaline Detergent Concentrate is a proprietary formula of powerful alkaline builders, water conditioners, anti-redeposition agents and surfactants.



Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

Key Features

- Economical
- Powerful and convenient
- High-performance cleaning
- Effective for removal of protein, blood, fat, grease and oil
- Cleans surgical instruments, labware, glassware and animal cages

Usage Information

Always follow manufacturer's directions for the particular automatic washer being used. Calibrate dispensing equipment to deliver a minimum of 1/6 of an ounce per gallon of water (1.30 ml/L) up to 1/2 of an ounce per gallon (3.9 ml/L). Poor water quality may require an increased product dosage. Consult your Getinge representative for appropriate chemical concentration and machine cycle information.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild fragrance
- pH value: 12.0–14.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean	Order Number
2 x 1 Gallon	61301605566
2.5 Gallons	61301605567
5 Gallons	61301605800

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044166 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 