

# Getinge Clean

## PowerCon Aluminum Safe Detergent Concentrate

PowerCon Aluminum Safe Detergent is a low foaming, nonabrasive, instrument decontamination product that rapidly removes, dissolves and disperses all types of clinical soils as well as water scale and oxide stains. It is designed to be used on a wide variety of surgical instrumentation including aluminum trays and cases.



### Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

### Key Features

- Economical
- Powerful and convenient
- High-performance cleaning
- Effective for removal of protein, blood, fat, grease and oil
- Cleans surgical instruments, labware, glassware and animal cages
- Safe for use on aluminum

**Quality Statement:** Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

## Usage Information

Always follow manufacturer's directions for the particular automatic washer being used. PowerCon Aluminum Safe Detergent should be automatically fed between 1/12 and 1/2 oz per gallon (0.7 to 3.9 ml/L) into automatic washers and washer/ disinfectors, depending upon water conditions and soil levels. All cleaned surfaces should be thoroughly rinsed with water. Instruments should be visually inspected after cleaning. If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

## Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild fragrance
- pH value: 13.0–14.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

---

## Ordering Information

### Getinge Clean

### Order Number

2 x 1 Gallon

61301606043

2.5 Gallons

61301606044

5 Gallons

61301606045

15 Gallons

61301606694

---

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044169 REVA

**Manufacturer:** Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110

**Sales Office:** Getinge, 1777 East Henrietta Road, Rochester, NY 14623

[www.getinge.com](http://www.getinge.com)

**GETINGE** 