

Getinge Clean

PowerCon Neutral pH Detergent Concentrate

PowerCon Neutral pH Detergent Concentrate is an effective combination of detergents, water conditioners and wetting agents.



Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

Key Features

- Economic “cost per wash” detergent
- Powerful and convenient product delivering high-performance cleaning
- Excellent results in a wide variety of operating conditions
- Advanced chemical technology allows a neutral pH range while delivering superior soil removal

Usage Information

Always follow manufacturer’s directions for the particular automatic washer being used. Calibrate dispensing equipment to deliver a minimum of 1/10 of an ounce per gallon of water (0.80 ml/L) up to 1/2 of an ounce per gallon (3.9 ml/L). Poor water quality may require an increased product dosage. Consult your Getinge representative for appropriate chemical concentration and machine cycle information.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Clear, colorless, thin liquid
- Mild fragrance
- pH value: 7.0–8.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean	Order Number
2 x 1 Gallon	61301605560
2.5 Gallons	61301605561
5 Gallons	61301605802
15 Gallons	61301606579

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044171 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 