

Getinge Clean

Renuzyme Plus

Renuzyme Plus is a dual-enzyme formula providing extra cleaning at lower temperatures.



Application

Designed for severe soils, cold-water applications, and when shorter contact/soak times are required. Items placed in soak tanks or ultrasonics, or that are manually cleaned benefit from the dual-enzyme formula. For use in sinks, ultrasonic baths, cart washers and washer-disinfectors.

Key Features

- Penetrates and breaks down protein and organic matter
- Quickly and effectively removes blood, tissue, mucous and other proteinaceous soils
- Effective in hard or soft water
- Neutral pH detergent
- Will not stain instruments or equipment made of glass, metal, plastic or rubber

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Usage Information

Manual cleaning:

Use 1–2 oz/gallon (8–16 ml/L) of water. Discard soak solution when visibly soiled.

Automatic Cleaning:

Inject 0.2–2 oz/gallon (2–16 ml/L) of water in prewash cycle. Always follow manufacturer's directions for the particular automatic washer being used. Amount of product used will vary depending on water quality. This product is most effective between the temperatures of 70°–140°F (21°–60°C). If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

- Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Physical and Chemical Properties

- Clear, green, thin liquid
- Green apple fragrance
- pH value: 6.0–8.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling, storage and waste disposal methods.

Ordering Information

Getinge Clean

Order Number

4 x 1 Gallon	61301605269
5 Gallons	61301600089
15 Gallons	61301600090
30 Gallons	61301602690
1 oz Packets– 50/Box	61301601308

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044130 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110

Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 