

Getinge Clean

Shine Stainless Steel Cleaner/Polish

Shine is a stainless steel cleaner/polish formulated especially for removing dirt, grease, fingerprints and other soils from stainless steel and laminated plastic surfaces.



Application

For use on and around mechanical cleaning equipment, cart washers, equipment washers, washer-disinfectors and sterilizers.

Key Features

- Soy based
- Highly effective
- Polishes as it cleans

Usage Information

Shake well before using. Spray a light mist onto stainless steel. Using a clean, absorbent cloth, polish the stainless steel with the grain. Heavily soiled areas may require more dwell time. If you have additional questions, contact your Getinge representative.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Opaque, white, thin liquid
- Characteristic odor
- pH value 6.0–8.0

NOTICE: For institutional and industrial use only. Not for home use. Refer to SDS for more information regarding safety, handling and storage and waste disposal methods.

Ordering Information

Getinge Clean

Order Number

1-16 oz Spray Bottle

61301606201

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044197 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110

Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 