

Getinge Clean

Tec Rinse

Scale fighting rinse additive designed to be injected into the final freshwater rinse of automatic washers. Reduces hard water spotting and reduces drying time for carts, cases and instruments.



Application

For use in mechanical cleaning equipment, cart washers, instrument washers and washer-disinfectors.

Key Features

- Reduces water spotting
- Speeds drying time
- Concentrated for economy

Usage Information

Tec Rinse should be used at a dose of 1/32–1/8 oz gal (0.25–1 ml/L) depending on water quality. Follow manufacturer's directions for the particular automatic washer being used. If you have additional questions, consult your Getinge representative.

- Do not add bleach, detergents, or any other chemical to this product. Added chemical may destroy the effectiveness of this cleaner.

NOTE: Some plastics and rubbers are not compatible with detergents or high water temperatures. Check with manufacturer for suitability.

Quality Statement: Confidence in Getinge is the most important quality criteria. This is the hallmark of all our external and internal commitments, activities and products. Products and services supplied by Getinge conform to the agreed terms and expectations. The achievement of these quality goals is the basis for continued competitive and successful enterprise.

Physical and Chemical Properties

- Dark blue, thin liquid
- Characteristic odor
- pH Concentrate: 2.1
- pH Use Dilution: 3

NOTICE: For institutional and industrial use only. Not for home use. Refer to Safety Data Sheet (SDS) for more information regarding safety, handling, storage and waste disposal methods.

Ordering Information

Getinge Clean	Order Number
4 x 1 Gallon	61301605280
5 Gallons	61301604156
15 Gallons	61301604157
30 Gallons	61301601299

Getinge is a registered trademark of Getinge AB, its subsidiaries, or affiliates in the United States or other countries • Copyright 2017 Getinge AB or its subsidiaries or affiliates • All rights reserved. • MCV00044177 REVA

Manufacturer: Intercon Chemical Co, 1100 Central Industrial Dr., St. Louis, MO 63110
Sales Office: Getinge, 1777 East Henrietta Road, Rochester, NY 14623

www.getinge.com

GETINGE 